Prior to use, carefully read the instructions for use. [EN]

CAPSULE MIXER CM-II

DENTAL CAPSULE MIXER

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

Mixing of glass ionomer cements and other predoesd cements in capsules

IDENTIFICATION OF PARTS

- 1. Safety cover
- 2. Capsule holder
- 3. Power switch
- 4. Mode button
- 5. Mode indicator
- 6. Timer selection button
- 7. Time display
- 8. Start button
- 9. Adapter
- 10. Power cord

SETUP

1. Remove the two black transportation screws from the bottom of the device (Fig. 1). Keep the device as horizontal as possible when unscrewing. Loosen both screws evenly.
2. Store the screws in the storage holes (Fig. 2). The screws will be necessary again if the device has to be transported in the future.
3. Firmly connect the power cord to the device and subsequently to the main power socket (100-240V AC) (Fig. 3).

SAFETY INSTRUCTIONS

1. The unit should only be used by a qualified dental professional according to the instructions.
2. The unit should not be exposed to direct sunlight, dust, a wet environment or placed near a heater.
3. Keep out of the reach of infants or children.
4. Prior to use, make sure the unit is properly connected to the appropriate power source.
5. The unit should be checked on a regular basis.
6. Do not handle the unit with wet hands.
7. Do not use the unit on a patient's mouth.
8. In case of breakdown, disconnect the unit and appropriately label the unit with details of possible fault. The repair of the unit may only be carried out by a qualified person.

DIRECTIONS FOR USE

Make sure the safety cover is tightly closed. If the cover is open, OF is shown on the time display.

1. Turn on the power switch. The mode indicator and time display will light up.
2. Set the mixing time according to the manufacturer's instructions on the encapsulated product. Two preferred time settings can be saved as mode 1 or 2.
3. Both modes are preset at the factory at 10 seconds. To change the time setting, follow the below steps.
 - a) Select mode 1 or 2 by pressing the mode button.
 - b) Press either "+" or "-" once to activate (display starts flashing).
 - c) Then press "+" or "-" button to increase or decrease the mixing time by 1 sec. Mixing time in the range of 1-30 seconds may be selected.
 - d) Press the mode button to set/save the selected mixing time (display stops flashing).

4. In the opening arm. Hold the capsule with the nozzle facing upward, install it in the swing arm of the capsule holder and press down until it fits in the fixed arm as shown in figure 4.

5. Close the cover and press the start button. Once the mixer starts, the time display starts counting down.









Note:
For safety reasons, the mixer will not start if the cover is open.

6. Once the selected mixing time has finished, the cover can be opened. Remove the mixed capsule from the capsule holder. Make sure the capsule nozzle is facing upward when removing from the holder (Fig. 5).
Note:
a) Do not attempt to start mixing without a capsule being installed in the device. This may cause breakage of the device. Breakage caused by improper use is not covered by the warranty.
b) In case of emergency, the mixing process can be stopped by opening the safety cover.
c) If a capsule leaks and material comes into contact with the device, wipe off immediately.

7. This may cause breakage of the device. Breakage caused by improper use is not covered by the warranty.
b) In case of emergency, the mixing process can be stopped by opening the safety cover.
c) If a capsule leaks and material comes into contact with the device, wipe off immediately.

MAINTENANCE AND CARE

Put the device away from direct sunlight, strong heat sources and dusty areas. Prior to routine cleaning and maintenance of the unit, disconnect the power cord from the mains socket. The device can be cleaned with a soft cloth soaked in warm water or mild detergent. Avoid using volatile solvents.

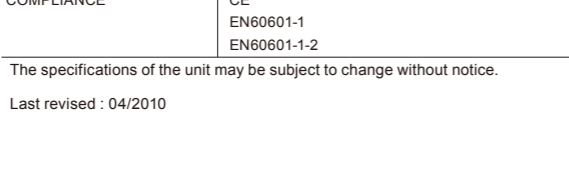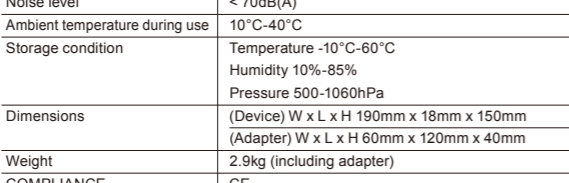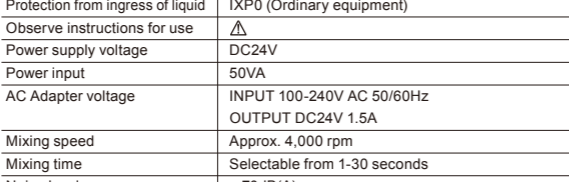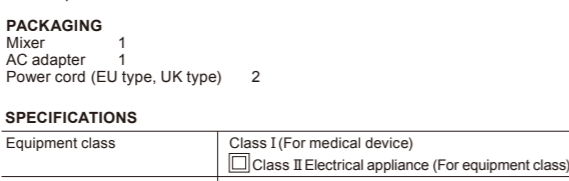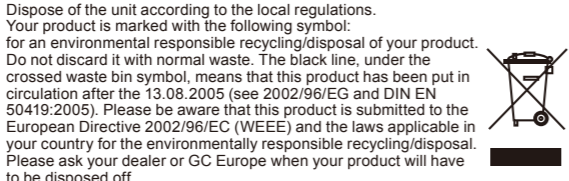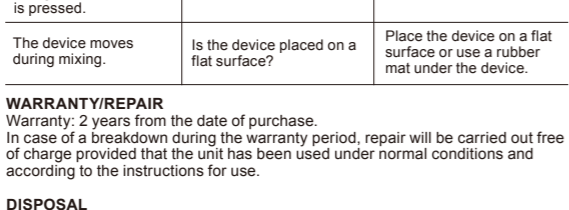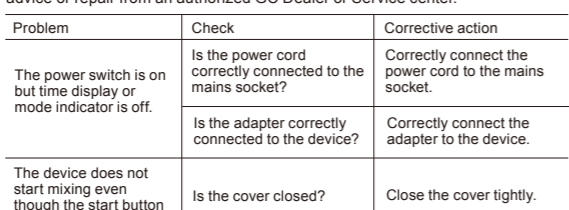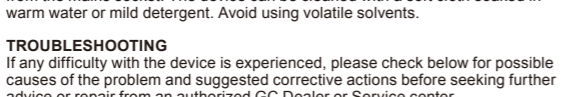DISTRIBUITY BY GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
EU: GC EUROPE N.V.
Rosaarhark Haarode-Leuven 1240, Interleuvenaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00
MANUFACTURED BY MONITEX INDUSTRIAL CO., LTD.
6F, 70 Quang-Tu Road Sec. 1, San-chung city, Taipei, Taiwan

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