**FICHA TÉCNICA**

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**TECHNICAL DATA SHEET**

<b>DUAL RESIN CEMENT PROCLINIC EXPERT</b>		
<b>REFERENCES</b>	78552: Colour dentine 78553: Translucent	
<b>MANUFACTURER</b>	CHL MEDICAL SOLUTIONS, Srl Via Felice Cavallotti, 15 20122 Milano - Italy	
<b>CLASSIFICATION</b>	Medical Device, Class IIa, Directive 93/42/EEC. Notified Body: Kiwa Cermet Italia S.p.A., No. 0476	
<b>DESCRIPTION</b>	Permanent resin cement does not require etching, priming or bonding. Self / light cured. The "Automix" system provides a rapid, consistent and homogeneous mixture which can be dispensed directly into the resation or tooth cavity.	
<b>COMPOSITION</b>	Base: methacrylate Oligomer, Triethyleneglycoldimethacrylate, Co-initiator, Photoinitiators, Treated Fumed Silica. Catalyst: Urethandimethacrylate Oligomer, Triethyleneglycoldimethacrylate, 4-Methacryloxyethyltrimellitic anhydride, Benzoyl Peroxide, Treated Fumed Silica.	
<b>SPECIFICACIONS</b>	Adhesive bonding to un-etched enamel: >20 MPa Radiopacity, % Aluminium: 250 Working time (in ambient light and temperature): 1,5-3,5 min.	
<b>PACKAGING</b>	1 automix syringe 8 g 10 mixing tips 10 intraoral angular tips, thin 10 intraoral angular tips, long XX thin 1 mixing pad	
<b>SHELF LIFE</b>	2 years from the date of manufacture	
<b>INTENDED USE</b>	Used for permanent cementation of crowns and bridges, inlays and onlays, posts and cores and ceramic crowns.	
<b>INSTRUCTIONS FOR USE</b>	<ol style="list-style-type: none"> <li>1) Remove the provisional restoration. Note: Use only temporary cements that are eugenol-free. The use of eugenol-containing temporary materials can inhibit the polymerization process of the product.</li> <li>2) Rinse thoroughly.</li> <li>3) Air dry the preparation (do not desiccate).</li> <li>4) Try in the restoration to ensure proper fit.</li> <li>5) Bleed the syringe before initial use. Place mixing tip on the Automix syringe.</li> </ol>	

	<p>6) Dispensing the material can be done as follows:</p> <p><i>Inlay/Onlay</i></p> <ul style="list-style-type: none"> <li>• Mount the supplied intra oral tip onto the mixing tip and dispense the cement directly covering all surfaces. Seat the restoration gently onto the preparation allowing the cement to flow from all sides.</li> </ul> <p><i>Crown</i></p> <ul style="list-style-type: none"> <li>• Dispense the cement directly into the restoration covering all surfaces. Seat the restoration gently onto the preparation allowing the cement to flow from all margins.</li> </ul> <p><i>Post</i></p> <ul style="list-style-type: none"> <li>• Mount the supplied XXF intra oral tip onto the mixing tip. Syringe the cement into the canal. Seat the post and vibrate slightly avoiding possibility of trapped air. Allow the cement to slowly flow from the canal space.</li> </ul> <p>7) When the restoration is properly seated, remove all excess cement. Excess cement is best removed in its gel state with a scaler or explorer. Gel state can be achieved by tack curing excess with a light for 2 seconds or allowing to self-cure for 1.5 - 2 minutes after application. For best results, leave undisturbed for 1.5 minutes before tack curing.</p> <p>8) TIMES:</p> <ul style="list-style-type: none"> <li>• Working time is 2 minutes.</li> <li>• Oral set time is 3 minutes.</li> <li>• Light-curing time: 20 seconds (see manufacturer’s recommendation).</li> <li>• After removal of excess cement, light cure all surfaces including margins for 20 seconds.</li> </ul> <p>9) Use finishing strips for proximal areas. Polish margins with polishing discs, cups, or points.</p> <p>10) Check occlusion. Note: Automix tips are designed for single patient use only.</p>
<p><b>SAFETY INFORMATION</b></p>	<p>The product requires a calcium hydroxide liner/base when working in close proximity to the pulp! Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact</p>
<p><b>HANDLING AND STORAGE CONDITIONS</b></p>	<p>Store below 24°C (68°F). Refrigerate for maximum shelf life. It is not recommended for use beyond the expiration date marked on the packaging.</p>