Universal Adhesive

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Hydrophil Methacrylate, MDP, Photoinitiatoren

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Etching precaution

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as below.
Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

a. Selective enamel etching

Apply an etching gel onto the enamel surfaces and leave in place for **at least 15 seconds**. Deciduous teeth are etched correspondingly longer. Rinse for 20 seconds with water and dry it in a water-free and oil-free airstream. The etched enamel surface should have a chalky white appearance.

b. Total-Etch Technique

Apply an etching gel onto the enamel and dentine surfaces beginning with the enamel bevels. Condition the enamel for **at least 15 seconds and the dentine for 15 seconds**. [This results in 20-30 seconds etching of enamel and 15 seconds of dentine]. Deciduous teeth are etched correspondingly longer. Rinse for 20 seconds with water. Dry it in a water-free and oil-free airstream, but do not desiccate.

The etched enamel bevel should have a chalky white appearance.

5. Application of UNIVERSAL ADHESIVE PROCLINIC EXPRT

Dispense one or two drops UNIVERSAL ADHESIVE PROCLINIC EXPERT into a mixing well and use immediately after dispensing.

Close bottle immediately after use.

Apply UNIVERSAL ADHESIVE PROCLINIC EXPERT with a brush onto the whole enamel and dentine surface in a thin and even layer. Keep the adhesive agitated for **at least 20 seconds**. Applying the adhesive on the tooth surface without agitation is inadequate. The material should build a homogeneous layer.

Air thin gently (**at least 5-10 seconds**) to remove the volatile components and to disperse the adhesive until a glossy and uniform layer results. Then light cure with a suitable dental halogen light unit or an LED (wavelength 400–500 nm, light intensity min. 1.000 mW/cm2) for **20 seconds**.

6. Application of a composite

Refer to manufacturer instructions for placement of the light, dual or self cure restorative material.

Note for direct restorations

Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

Additional informations/Warnings

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Composition

Hydrophilic methacrylates, MDP, photo initiators.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

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Limitation of Liability

Except where prohibited by law, PL Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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