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DE

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INSTRUCTIONS FOR USE

EN

SURFACE CONDITIONER FOR CERAMIC MATERIALS

INDICATIONS

Proclinc Expert Ceramic Primer is a surface conditioner for ceramic materials such as porcelain and zirconia restorations.

It is used for

- Preparation of ceramic crowns, veneers and inlays prior to cementation
- Preparation of fractured ceramic crowns for repair with resin materials

PROPERTIES

Proclinc Expert Ceramic Primer is a one bottle primer material, for surface preparation of ceramic crowns, prior to bonding with resin cements. This primer improves cementation of zirconia crowns and does not require additional conditioning such as sandblasting. Its low viscosity and excellent wetting properties promote ideal surface coverage.

SHELF LIFE OF THE PRODUCT

Shelf life: 2 years when stored below 24°C (75°F).

PREPARING CERAMIC SURFACE FOR BONDING

1. Clean ceramic surface with alcohol and dry well with oil-free air.

2. Open Proclinc Expert Ceramic Primer and dispense 1-2 drops into mixing well, replace bottle cap immediately.
3. Use applicator to apply generous amount of primer onto ceramic surfaces.
4. Wait 2 minutes.
5. Dry gently for 20 seconds with oil-free air.
6. Continue with cementation.

ESSENTIAL INGREDIENTS

Ethanol, water, organic acid, silane coupling agent, 4-META

CAUTIONS

Avoid contact with skin, oral soft tissues and eyes. If accidental contact occurs, rinse with copious amount of water and seek medical attention immediately.

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