proxeo^{AURA}

EN

Air Polisher

USE AND MAINTENANCE MANUAL

PF-60, PF-60 M, PF-60 N, PF-60 S

Copyright

© Mectron S.p.A. 2022. All rights reserved. No part of this document may be reproduced, in any form whatsoever, without the prior written consent of the copyright holder.

Trademarks

All trademarks belong to their respective owners:

- MULTIFLEX® - MULTIFLEX® is a registered trademark of Kaltenbach & Voigt (KaVo®) GmbH company.
- SIRONA® is a registered trademark of SIRONA DENTAL SYSTEMS GMBH company.
- ROTO QUICK® - ROTO QUICK® is a registered trademark of W&H® Deutschland GmbH company.
- MACHLITE® - MACHLITE® is a registered trademark of Nakanishi Corporation and Nakanishi Shika Kikai (NSK®) Ltd company.
- PHATELUS® - PHATELUS® is a registered trademark of Nakanishi Corporation and Nakanishi Shika Kikai (NSK®) Ltd company.

EN

CONTENTS

1	Introduction	1
1.1	Intended Use	1
1.2	Description of the Appliance	2
1.2.1	Eligible Patient Group	2
1.2.2	Patient Selection Criteria	2
1.2.3	Indications for Use	2
1.2.4	Users	2
1.3	Disclaimer	3
1.4	Safety Requirements	3
1.5	Symbols	4
2	Identification Data	6
2.1	Appliance Identification Data	6
2.2	Polisher Nozzle Identification Data	6
3	Delivery	7
3.1	List of components	7
4	Installation	10
4.1	Quick Couplings for Turbine	11
4.2	Safety Requirements in the Installation Phase	12
5	Use	12
5.1	Safety Requirements Before and During Use	13
5.2	Instructions for Use	14
5.3	Cleaning the Powder Container	18
6	Cleaning, Disinfection and Sterilisation	21
6.1	Preparation	22
6.2	Cleaning the Device air polisher	23
6.2.1	Required Materials	23
6.2.2	Cleaning Method	23
6.3	Disinfecting the air polisher Device	24
6.3.1	Required Materials	24
6.4	Daily Decontamination of the Dental Unit Water Circuit	26
6.5	Cleaning the Sterilisable Accessories	26
6.5.1	Manual Cleaning	27
6.5.1.1	Required Materials	27
6.5.1.2	Polisher Nozzles	28
6.5.1.3	K9 Wrench	30
6.5.1.4	Cleaning Needles	32
6.5.2	Automatic Cleaning	34
6.5.2.1	Required Materials	34
6.5.2.2	Cleaning Procedure	34
6.6	Cleaning Control	35
6.6.1	Required Materials	35
6.6.2	Procedure for Cleaning Control	35
6.7	Drying	36
6.7.1	Required Materials	36
6.7.2	Drying Procedure	36
6.8	Sterilisation	36
6.8.1	Preparation	36
6.8.2	Sterilisation Method	37
6.8.2.1	Special Information	37

7	Maintenance	38
7.1	Maintenance of the Polisher Nozzle	38
7.1.1	Routine Maintenance of the Polisher Nozzle	38
7.1.2	Clogged Polisher Nozzle	38
7.2	Maintenance of the Core Unit	39
7.2.1	Routine Maintenance of the Core Unit	39
7.2.2	Clogged Core Unit	39
7.3	Checking the Connector of the Polisher Nozzle	40
7.4	Checking the Turbine Quick Coupling	40
7.5	Routine Decontamination of the Dental Unit Water Circuit	41
8	Methods and Precautions for Disposal	41
9	Technical Data	42
10	Troubleshooting	43
10.1	Quick Troubleshooting	43
10.2	Shipping to an Authorised W&H Service Centre	46
11	Warranty	47

PAGE INTENTIONALLY LEFT BLANK

1 INTRODUCTION

Carefully read this manual before proceeding with installation operations, use, maintenance or other intervention operations on the appliance.

This manual must always be available to the operator.

Important: To avoid damages to persons or property, carefully read all the "Safety Requirements" in the manual.

Depending on the level of severity, the safety requirements are classified with the following indications:

⚠ WARNING: (Always referred to damage to persons)

⚠ CAUTION: (Referring to possible damage to property)

The aim of this manual is to make the operators aware of the safety regulations, instal-

lation procedures, instructions for proper use and maintenance of the appliance and its accessories.

Use of this manual for aims other than those strictly linked to the installation, use and maintenance of the appliance is prohibited.

The information and illustrations in this manual were updated on the edition date shown on the last page.

MECTRON is engaged in continuously updating its products with possible changes to the components of the apparatus.

In case you encounter discrepancies between the descriptions found in this manual and the equipment in your possession you can:

- check for any available updates in the *MANUALS* section of the W&H website;
- ask clarifications to Your Dealer;
- contact the W&H After Sales Service.

1.1 Intended Use

This device is a water jet, air, and powder polisher for dental use, designed for a complete supra- and subgingival prophylaxis treatment, with a specific intended use depending on the type of powder distributed by W&H.

Air-polisher for supragingival use:

- Removal of supragingival bacterial plaque;
- Removal of stains on the surface of teeth;
- Preparation of cavity for better adhesion between enamel and filling material;
- Prophylaxis on patients undergoing orthodontic therapy.

This air-polisher can be used on patients of any age and gender requiring a tooth polishing treatment. There are no contraindications for specific population segments.

Air-polisher for subgingival use:

- Removal of subgingival bacterial plaque;
- Removal of Biofilm in the prevention of Peri-implantitis.

1.2 Description of the Appliance

This air polisher is a compact water jet, air, and powder dental device used in the dental sector for prophylaxis and tooth polishing treatments.

This air polisher can be used with powders of various particle sizes and guarantees a constant, efficient jet of powder and simple, reliable use.

The operating principle of a air polisher is based on the mechanical action obtained by a mechanical jet of powders accelerated by a flow of compressed air.

The kinetic energy thus impressed on the

particles is dissipated when it hits the enamel surface, thus producing a delicate but efficient polishing action.

The action is completed by a jet of water which, by means of the vacuum created around the nozzle, forms a bell-shape around the main flow, thus producing a double effect: to prevent much of the rebound and the leakage of the powder jet and continuously wash the treated area, dissolving the powder itself.

This air polisher can be supplied with different quick couplings for connection to the turbine of the dental unit.

1.2.1 Eligible Patient Group

This medical device is designed to be used with the following patient groups:

- Children;
- Adolescents;
- Adults;
- Seniors.

This medical device can be used on patients of any age, weight, height, gender, and nationality, where applicable.

1.2.2 Patient Selection Criteria

It is not recommended to use the device in the following cases:

1. Upper respiratory tract infections, chronic bronchitis/asthma; (radiotherapy, chemotherapy, antibiotics);
2. Pregnant and breastfeeding women;
3. Patients undergoing treatment
4. Acute infectious oral lesions.

It is not recommended to use the powders in the following cases:

1. Allergy to the aroma of the powder;
2. Patients on a sodium restricted diet or who suffer from serious respiratory problems, such as chronic bronchitis, asthma, emphysema, etc., unless otherwise specified by their doctor.

All air-polishing devices are intended for professional use only. The user is therefore the only person able to decide if and how to treat their patients.

1.2.3 Indications for Use

The device is suitable for use on all eligible patients (see Chapter 1.2.1 on page 2) whose doctor has prescribed a professional tooth polishing treatment, as part of the intended use of the device (see Chapter 1.1 on page 1).

1.2.4 Users

The device must be used exclusively by specialised and properly trained personnel, specifically the dental surgeon/dentist or dental hygienist, who must be able-bodied, adult, of any weight, age, height, gender, and nationality.

1.3 Disclaimer

The manufacturer Mectron disclaims all liability, express or implied, and cannot be held responsible for direct or indirect personal injury and/or property damage, occurring as a result of incorrect procedures linked to the use of the appliance and its accessories.

The manufacturer Mectron cannot be held liable, expressly or implicitly, for any type of personal injury and/or property damage, inflicted by the user of the product and its accessories and which occurs in the following cases:

- Misuse or use during procedures other than those specified in the destination of use of the product;
- The environmental conditions for preservation and storage of the device are not complying with the requirements indicated in *Chapter 8 on page 41*;
- The appliance is not used in accordance with all the instructions and requirements described in this manual;
- Assembly operations, extensions, re-adjustments, upgrades and repairs of the device are carried out by personnel not authorised by Mectron or by W&H;
- Misuse, abuse, abnormal use, negligent use, intentional misconduct or use exceeding the limits of the appliance indicated and allowed and/or normal wear or deterioration, ill-treatment and/or incorrect interventions;
- Any attempt to tamper with or modification of the appliance under every circumstance;
- Shortage of stock material (handpiece, nozzles, wrenches) to be used in case of a standstill due to faults or problems;
- Incorrect/omitted maintenance compared to what is stated in *Chapter 7 on page 38* of this manual;
- Breach of the requirements and the information contained in *Chapter 6 on page 21* of this manual;
- Unauthorised repairs in accordance with the indications contained in *Chapter 10.2 on page 46* this manual.

1.4 Safety Requirements

⚠ CAUTION: It is prohibited to make any changes to this appliance.

⚠ WARNING: The appliance must be used in a dental practice or clinic, or in professional oral hygiene and preventive care centres.

⚠ WARNING: Qualified and specialist personnel.

The appliance should only be used by qualified personnel with adequate medical culture; for the use of the device are not requested special training activities. Use of the device does not produce any side effects if used correctly. Misuse may cause soft tissue damage and tissue emphysema.

⚠ WARNING: Intended use.

Use the device only for the destination of use for which it is provided (see *Chapter 1.1 on page 1*). Failure to observe this regulation can cause serious injury to the patient, the operator and damage/faults on the device.

⚠ WARNING: Contraindications.

Patients wearing contact lenses must remove them prior to receiving treatment with the jet polisher.

⚠ WARNING: Contraindications.

Do not direct the air/powder/water jet towards the eyes.

Failure to comply with this indication may cause serious eye damage. It is advisable for all involved persons to wear safety goggles (patient, hygienist, and dentist).

⚠ WARNING: Contraindications.
Do not aim the air/powder/water jet towards composites, fillings, and bridges. Failure to comply with this indication may cause damage thereto.

⚠ WARNING: Contraindications.
Do not aim the air, specific supragingival powder, or water jet onto the soft tissues or inside the gingival sulcus. Failure to comply with this prescription can cause a gingival tissue emphysema (emphysema of the mucous and/or subcutaneous).

⚠ WARNING: Contraindications. Do not use the device near areas subject to recent dental extraction and in traumatised/damaged areas (or areas nearby) due to the risk of emphysema.

⚠ WARNING: Contraindications.
It is advisable for the dentist and hygienist to use a protective face mask to limit the inhalation of powder.

⚠ WARNING: Checking the condition of the device before treatment.
Before every treatment always check the perfect operation of the appliance and efficiency of the accessories. If operation anomalies are observed do not carry out the treatment. If the faults concern the appliance, contact an Authorised W&H Service Centre.

⚠ WARNING: Cleaning, disinfection, sterilisation of new or repaired products.
All new or repaired products are delivered non-sterile. Before the treatment, any new or repaired products must be cleaned, disinfected and sterilised, strictly following the instructions provided in *Chapter 6* on page 21.

⚠ WARNING: Infection control.
For utmost patient and operator safety, use only clean, disinfected and sterilised accessories. Strictly follow the instructions provided in *Chapter 6* on page 21.

⚠ CAUTION: Contraindications.
After having sterilised the polisher nozzles and any other sterilisable accessory, wait for them to completely cool down before reusing them.

⚠ WARNING: Use only original W&H accessories and spare parts.

⚠ WARNING: In case of knocks, damage, or dents on the clear air polisher core unit, send it to an Authorised W&H Service Centre for analysis and inspection.

⚠ WARNING: In case of an adverse event and/or accident attributable to the device during correct use and in accordance with the intended use, a report must be made to the Competent Authority and manufacturer indicated on the product label.

NOTE: To check and guarantee the perfect operation of the device and prolong its service life, maintenance should be carried out by an Authorised W&H Service Centre on an annual basis.

1.5 Symbols

Symbol	Description	Symbol	Description
	Device compliant with Regulation (EU) 2017/745. Notified body: IMQ S.p.A.		Medical device
	Caution! Read the instructions for use		Operating instructions


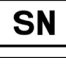
Symbol	Description	Symbol	Description
	Manufacturer		Date of manufacture
	Serial number		Batch number
	Product code		Disposable
	Expiry date		Sterilised with Ethylene Oxide (EO)
	Not sterile		The sterilisable materials must be autoclaved and can resist up to a maximum temperature of 135 °C
	Treatable in thermal-disinfector		The appliance and its accessories must not be disposed of or treated as municipal solid waste
	Generic warning sign ^{a)}		Temperature limits for transport and storage
	Moisture limits for transport and storage		Atmospheric pressure limits for transport and storage
QTY.1	Quantity in package: 1		Do not use if the package is damaged
Rx only	For US market only CAUTION: US federal law restricts sale to orders by licensed dentists or dental hygienists only.		

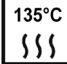Table 1 – Symbols

a) The symbol is represented by a yellow triangle and black graphic symbol.

2 IDENTIFICATION DATA

A correct description of the model and of the serial number of the appliance will allow the After Sales Service to provide fast and effective answers. Always provide this information every time you contact a W&H Service Centre.

2.1 Appliance Identification Data

The appliance name, serial number and other information (Figure 1) is laser engraved on each device. The complete technical specifications are provided in Chapter 9 on page 42.

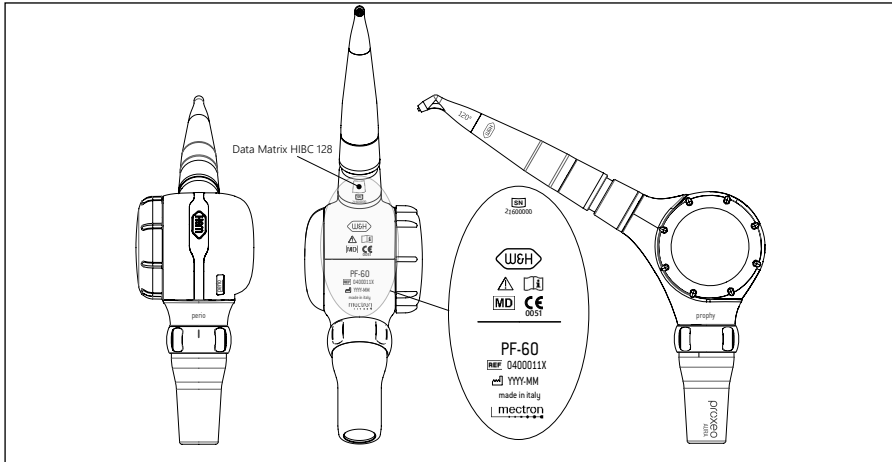
Figure 1 – Appliance Identification Data

2.2 Polisher Nozzle Identification Data

The name of the polisher nozzle itself (Figure 2 - Ref. 1), W&H logo (Figure 2 - Ref. 2), the batch number of the polisher nozzle (Figure 2 - Ref. 3), the HIBC data matrix (Figure 2 - Ref. 4) and other symbols described in Chapter 1.5 on page 4 are laser engraved on each polisher nozzle.

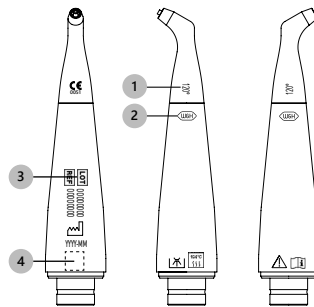
Figure 2 – Polisher Nozzle Identification Data

3 DELIVERY

3.1 List of Components

The air polisher is supplied in two base sets:

- Prophy-Set;
- Paro-Set.

Prophy-Set Supply			
Item	Code	Description	Ref. ^{b)}
Device core unit ^{c)}	04000123	W&H® coupling / PF-60 (ROTO QUICK®)	A
	04000124	KaVo® coupling / PF-60 M (MULTiflex® - MULTiflex® LUX)	
	04000125	NSK® coupling / PF-60 N (PHATELUS® - MACHLITE®)	
	04000126	Sirona® coupling / PF-60 S (SIRONA® R/F - SIRONA® B)	
Cleaning needle Ø 0.4 mm	02900050		B
Cleaning needle Ø 0.8 mm	02900121		C
Pipe cleaner	02900131		D
Use and Maintenance Manual	02150396		E
Information: Online Documentation	02150660		
O-ring replacement KIT	02900141-001		F
Cap and cap O-ring replacement KIT	02900142-001		G
Jar of specific supragingival powder	03140030	Prophylaxis Powder: SOFT + (single package) ^{e)}	L
TERMINALE PULITORE LUNGO 120°	03340017	120° long air-polishing nozzle	H5

b) See image on front cover.

c) The air polisher device can be connected to the dental unit turbine using different couplings (see Chapter 4.1 on page 11). A different type of device core unit will be supplied depending on the requested coupling.

Paro-Set Supply			
Item	Code	Description	Ref. ^{b)}
Device core unit ^{o)}	04000123	W&H [®] coupling / PF-60 (ROTO QUICK [®])	A
	04000124	KaVo [®] coupling / PF-60 M (MULTIflex [®] - MULTIflex [®] LUX)	
	04000125	NSK [®] coupling / PF-60 N (PHATELUS [®] - MACHLITE [®])	
	04000126	Sirona [®] coupling / PF-60 S (SIRONA [®] R/F - SIRONA [®] B)	
Cleaning needle Ø 0.4 mm	02900050		B
Cleaning needle Ø 0.8 mm	02900121		C
Pipe cleaner	02900131		D
Use and Maintenance Manual	02150396		E
Information: Online Documentation	02150660		
O-ring replacement KIT	02900141-001		F
Cap and cap O-ring replacement KIT	02900142-001		G
Jar of specific supragingival powder	03140030	Prophylaxis Powder: SOFT + (single package) ^{e)}	L
Jar of specific subgingival powder	03140024	Prophylaxis Powder: SENSITIVE + (single package) ^{e)}	K
TERMINALE PULITORE LUNGO 120°	03340017	120° long air-polishing nozzle	H5
TERMINALE PULITORE LUNGO PERIO	03340018	PERIO air-polishing nozzle	H4
SUBGINGIVAL PERIO TIPS	02900139-001	40 pc set of SUBGINGIVAL PERIO TIPS	I
K9 Wrench	02900140-001		J

Accessories			
Item	Code	Description	Ref. ^{b)}
TERMINALE PULITORE CORTO PERIO	03340021	PERIO short air-polishing nozzle ^{d)}	H1
TERMINALE PULITORE CORTO 120°	03340020	120° short air-polishing nozzle ^{d)}	H2
TERMINALE PULITORE CORTO 90°	03340019	90° short air-polishing nozzle ^{d)}	H3
TERMINALE PULITORE LUNGO PERIO	03340018	PERIO air-polishing nozzle ^{d)}	H4
TERMINALE PULITORE LUNGO 120°	03340017	120° long air-polishing nozzle ^{d)}	H5
TERMINALE PULITORE LUNGO 90°	03340016	90° long air-polishing nozzle ^{d)}	H6
SUBGINGIVAL PERIO TIPS	02900139-001	40 pc set of SUBGINGIVAL PERIO TIPS ^{d)}	I
K9 Wrench	02900140-001	^{d)}	J
Jar of specific subgingival powder	03140024	Prophylaxis Powder: SENSITIVE + (single package) ^{e)}	K
	03140023	Prophylaxis Powder: SENSITIVE + (2x160g package) ^{e)}	
Jar of specific supragingival powder	03140030	Prophylaxis Powder: SOFT + (single package) ^{e)}	L
	03140029	Prophylaxis Powder: SOFT + (4x250g package) ^{e)}	
	03140016	Prophylaxis Powder: SMOOTH (single package) ^{d)}	
	03140015-001	Prophylaxis Powder: SMOOTH (4x250g package) ^{d)}	
Cleaning kit	02900143-001	Includes: <ul style="list-style-type: none"> • Cleaning needle Ø 0.4 mm; • Cleaning needle Ø 0.8 mm; • Pipe cleaner.^{d)} 	M
Cap and cap O-ring KIT	02900142-001	^{d)}	G

d) Produced by Mectron.

e) Distributed by Mectron.

NOTE: Both the items included in the standard supplies and all accessories can be ordered separately by the client.

The device packaging is sensitive to impact, therefore caution must be exercised during its transport and deposit.

Do not stack multiple boxes so as not to crush the underlying packages.

All material sent by Mectron is checked at the time of dispatch.

The device is dispatched with suitable protection and packaging.

When the device is received, check the condition of the packaging and in case of damage, deterioration, or signs of tampering, submit a claim to the carrier.

Keep the packaging in case any items need to be sent to an Authorised W&H Service Centre and to store the appliance during long periods of non-use.

4 INSTALLATION

The air polisher device, in order to operate must be connected to a dental unit equipped for water and air distribution.

NOTE: The device does not require any electrical and/or electronic connections to be made.

The air polisher device is supplied with a specific adapter for the main couplings to the dental unit turbine (see Chapter 4.1 on page 11).

⚠ CAUTION: Use the air polisher device only with the specific coupling for the relative turbine.

Connecting the air polisher device to a connector not intended for use in combination with the dental unit may damage the device.

The dental unit to which air polisher is connected must be a medical device that complies with general and applicable product regulations for the specific category of dental unit, and in general, satisfy the following standards:

- ISO 7494 Dentistry – Stationary dental units and dental patient chairs;
- ISO 9168 Dentistry – Hose connectors for air driven dental handpieces;
- ISO 60601-1 Medical electrical equipment – Part 1: General requirements for basic safety and essential performance.

To ensure the correct operation of the device, it is necessary to check:

- The “spray” water supply coming from the turbine;
- The air supply coming from the turbine;
- The presence of powder.

⚠ CAUTION: Check that the dental unit to which air polisher is connected satisfies the specified requirements.

The device must be installed in a suitable place allowing its comfortable use.

For the air polisher device to operate correctly, the air and water pressure values must be within the intended range of use, specifically:

- For water, from 1.0 to 3.0 bar (from 1000 hPa to 3000 hPa), max 40 °C;
- For air, from 2.2 to 3.0 bar (2200 hPa - 3000 hPa).

⚠ CAUTION: To ensure proper and long-lasting performance, ensure the air flowing from the dental unit is thoroughly dry and filtered.

Moreover, it is essential to ensure that the air pressure is between 2.2 and 3.0 bar inclusive, measured at the point of the connection between the device and the dental unit.

NEVER use excessive air pressure.

Increasing the air pressure does not correspond to an increase in performance; to the contrary, the device may become damaged or fail to work properly.

4.1 Quick Couplings for Turbine

The quick couplings with which the air polisher device can be supplied are listed in *Table 2* on page 11.

To connect the device to a quick coupling, fully insert the connector until you hear a “click”.

To disconnect the device, follow the instructions in *Table 2* on page 11.

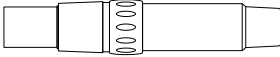
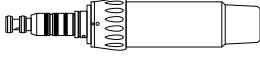
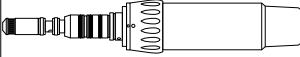
Quick Coupling	Description	Extraction Procedure
	W&H® ROTO QUICK®	On the quick coupling, make the ring slide towards the cord and disconnect the device by rotating it slightly. Do not pull or grasp the cord.
	NSK® MACHLITE® / PHATELUS®	
	KaVo® MULTIflex®	Firmly grasp the quick coupling and disconnect the device by rotating it slightly. Do not pull or grasp the cord.
	KaVo® MULTIflex® LUX	
	SIRONA® B	
	SIRONA® R/F	

Table 2 – Quick Couplings for Turbine

4.2 Safety Requirements in the Installation Phase

⚠ **CAUTION:** Check that the turbine connector is clean and that its O-rings are in good condition.

⚠ **WARNING:** Do not press the turbine control pedal while checking the connector.

⚠ **WARNING:** Do not press the turbine control pedal while making the connection with the device.

⚠ **CAUTION:** In case of a turbine with light, switch it off.

⚠ **CAUTION:** The polisher nozzle and turbine connector must be perfectly dry. Any humidity may cause the obstruction of the air/powder circulation channels.

⚠ **CAUTION:** Do not expose the appliance to direct sunlight or sources of UV light.

⚠ **CAUTION:** The appliance can be transported but must be handled with care when moved.

EN

5 USE

Suitable powder and polisher nozzles must be used depending on the type of treatment to be carried out:

- For applications with specific supragingival powders distributed by W&H, it is necessary to:
 1. Set the adjustment ring to prophy;
 2. Fill the powder container with the appropriate powder without exceeding the indicated maximum level;
 3. Insert the 120° polisher nozzle on the handpiece.
- For applications with specific subgingival powders distributed by W&H, it is necessary to:
 1. Set the adjustment ring to perio;
 2. Fill the powder container with the appropriate powder without exceeding the indicated maximum level;
 3. Insert the desired polisher nozzle on the handpiece (120° or PERIO);
 4. In case of subgingival applications and therefore when using the PERIO polisher nozzle, the single-use SUBGINGIVAL PERIO TIP must be inserted.

⚠ **CAUTION:** The SUBGINGIVAL PERIO TIP can only be used with the PERIO polisher nozzle.

⚠ **WARNING: The SUBGINGIVAL PERIO TIPS are single-use sterile.**

The SUBGINGIVAL PERIO TIPS are supplied sterile and for single use. They must be used on one patient only during a single application, after which they must be removed and disposed of.

⚠ **CAUTION:** Use only and strictly the powders intended for 'subgingival' use with the PERIO polisher nozzle.

⚠ **CAUTION:** Before filling the powder container with the selected powder, make sure it is clean and free of any powder residue from the previous treatment.

5.1 Safety Requirements Before and During Use

⚠ WARNING: Contraindications. Patients on a sodium restricted diet must not receive treatment with sodium bicarbonate powders.

⚠ WARNING: Contraindications. Patients who suffer from serious respiratory problems such as chronic bronchitis, asthma, emphysema, etc., must not receive prophylaxis treatment unless otherwise indicated by their doctor.

⚠ WARNING: Contraindications. Patients wearing contact lenses or glasses must remove them prior to receiving treatment with the air polisher.

⚠ WARNING: Contraindications. Do not direct the air/powder/water jet towards the eyes. Failure to comply with this indication may cause serious eye damage. It is advisable for all involved persons to wear safety goggles (patient, hygienist, and dentist).

⚠ WARNING: Contraindications. It is advisable for the dentist and dental hygienist to use a protective face mask to limit the inhalation of powder.

⚠ WARNING: Contraindications. Do not aim the air/powder/water jet towards composites, fillings, and bridges. Failure to comply with this indication may cause damage thereto.

⚠ WARNING: Contraindications. Do not use the device near areas subject to recent dental extraction and in traumatised/damaged areas (or areas nearby) due to the risk of emphysema.

⚠ WARNING: Contraindications. Do not aim the jet of air/powder/water onto the soft tissues or inside the gingival sulcus. Failure to comply with this prescription can cause a gingival tissue emphysema (emphysema of the mucous and/or subcutaneous).

⚠ WARNING: Checking the condition of the device before treatment.

Before every treatment always check the perfect operation of the appliance and efficiency of the accessories. If operation anomalies are observed do not carry out the treatment. If the faults concern the appliance, contact an Authorised W&H Service Centre.

⚠ WARNING: Use only original W&H accessories and spare parts.

⚠ WARNING: Infection, water and air circuit control.

First use: All parts and reusable accessories (new or returned from an Authorised W&H Service Centre) are delivered in NON-STERILE conditions and must be treated before each use, according to the instructions provided in *Chapter 6 on page 21*.

Subsequent uses: After each treatment, clean, disinfect and sterilise all parts and reusable accessories according to the instructions in *Chapter 6 on page 21*.

⚠ CAUTION: Do not attempt to unscrew the cap of the powder container when it is pressurised.

⚠ CAUTION: Make sure the powder container is perfectly dry. Any humidity may cause the channels to become clogged.

⚠ WARNING: During the intervention on the patient, do not perform any maintenance tasks on the system.

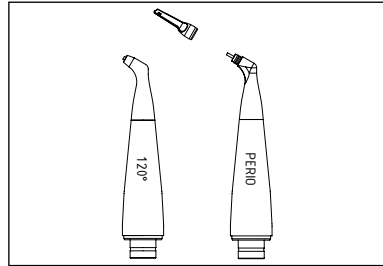
5.2 Instructions for Use

Select the type of polisher nozzle depending on the treatment to be carried out;

EN

⚠ **CAUTION:** The SUBGINGIVAL PERIO TIP can only be used with the PERIO polisher nozzle.


1

Grab the air polisher device, slide the ring down and hold it in this position, insert the selected nozzle until it comes into contact, then release the ring;

2

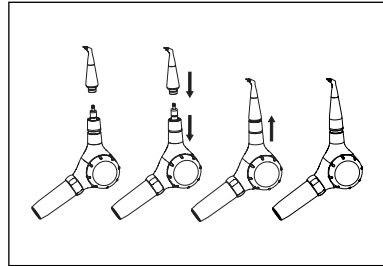⚠ **WARNING:** Before performing the subsequent operations, check that the polisher nozzle is properly inserted on air polisher (the ring and tip must be in contact) and that the ring has returned to the initial position.

If the PERIO polisher nozzle has been selected, insert the single-use SUBGINGIVAL PERIO TIP on the front part, pushing it all the way in until it comes into contact.

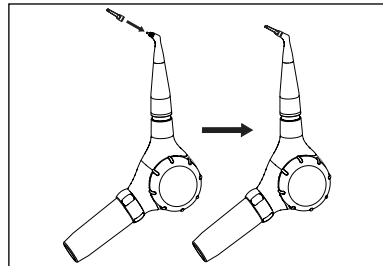
3

⚠ **WARNING:** Make sure the single-use SUBGINGIVAL PERIO TIP is properly inserted on the polisher nozzle; the two pieces must be in contact.

⚠ **CAUTION:** Use only original W&H accessories.


⚠ **CAUTION:** Handle the single-use SUBGINGIVAL PERIO TIP with care.

⚠ **CAUTION:** With the PERIO polisher nozzle, only specific subgingival powders distributed by W&H must be used.


4 Select the desired function, either prophy or perio, using the adjustment ring of the air polisher device.

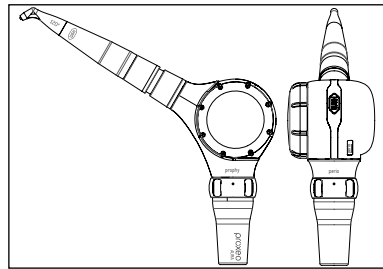
- Prophy function: use the specific supragingival powders distributed by W&H;
- Perio function: use the specific subgingival powders distributed by W&H.

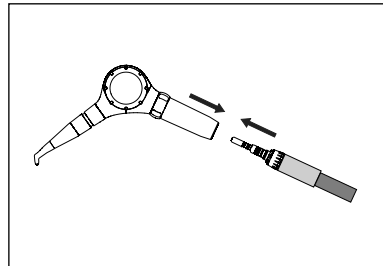
5 Connect air polisher to the turbine of the dental unit;

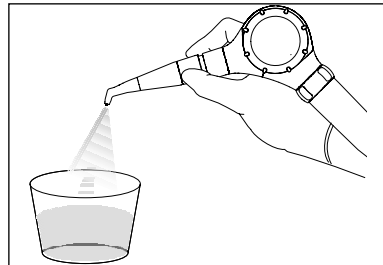6 Adjust the water flow rate based on the dental unit to obtain a consistent jet of water. The adjustment is easier to make in the absence of powder;

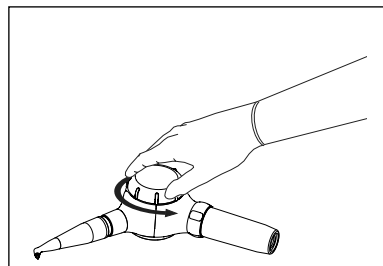
7 Unscrew the cap of the powder container;

Pour the selected powder for the specific application into the powder container.

⚠ **CAUTION:** Always use the powder, selecting the correct prophy or perio function.

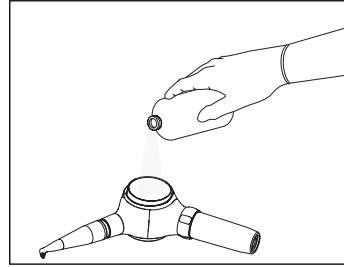
8

Two maximum powder levels are shown on the powder container:

- one for the specific supragingival powders distributed by W&H (prophy)
- one for the specific subgingival powders distributed by W&H (perio).

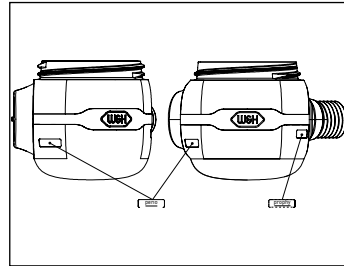9


⚠ **CAUTION:** Do not exceed the recommended levels for the different types of powders.

Screw the cap of the powder container back on;

⚠ **CAUTION:** Remove any powder residue from the thread of the cap before screwing it in.

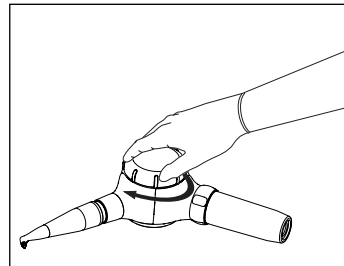
10

Grab the air polisher device properly, making sure not to tilt it too much ($> 45^\circ$) either lengthways or sideways. To work with greater inclinations, rotate the polisher nozzles without tilting or rotating the entire device;

⚠ **CAUTION:** Excessive or incorrect inclination may produce a less efficient jet.

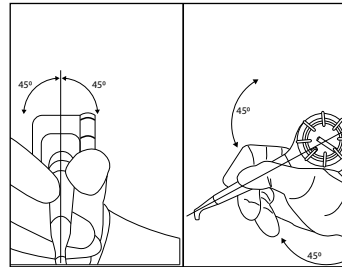11


EN

Press the pedal of the turbine to perform the treatment;

⚠ **CAUTION:** During the treatment use the suction tube to eliminate the air/powder/water mix from the patient's mouth.

⚠ **CAUTION:** When the foot is lifted off the control pedal, the air/powder/water jet continues for a few seconds. End the treatment considering these additional seconds.

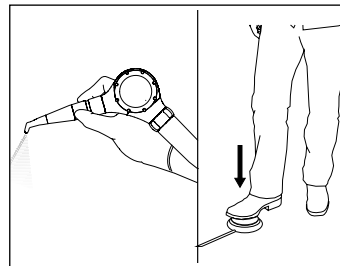12


5.3 Cleaning the Powder Container

⚠ **CAUTION:** The operations described in this chapter must be carried out each time the powder is changed and before cleaning, disinfection, and sterilisation.

EN


Disconnect the air polisher device from the turbine of the dental unit;

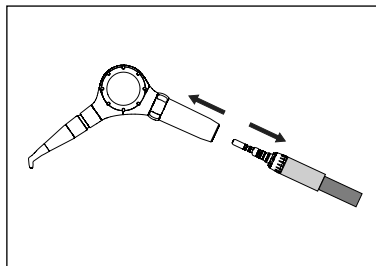1


Unscrew the cap and empty the powder container;

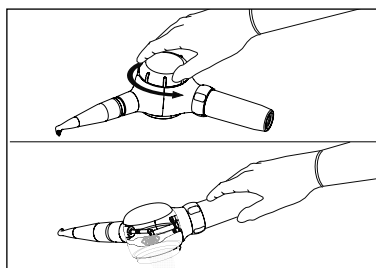2


Grab the air polisher device, slide the ring down and hold it in this position, extract the nozzle, then release the ring;

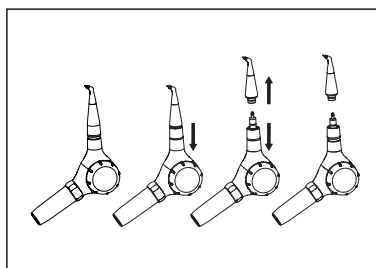3


Blow compressed air into the powder container, on the cap, and in the cavity of the handpiece connector, so as to remove any powder residue;

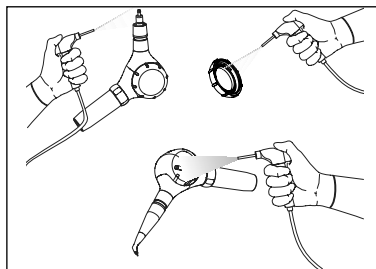4

Connect the air polisher device to the turbine of the dental unit;

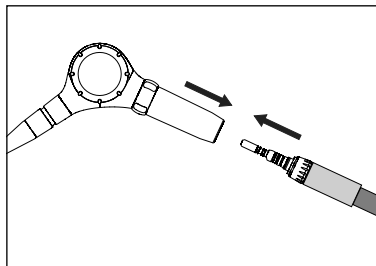
5

Take the supplied brush and brush the holes on the diffuser in the powder container, holding the pedal pressed.

NOTE: When the pedal is pressed, water will flow out from the diffuser. Beware of the environment in which this procedure is performed.

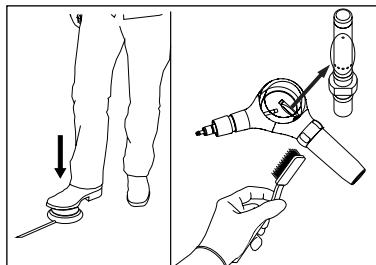
6

Screw the cap of the powder container back on;


① **CAUTION:** Remove any powder residue from the thread of the cap before screwing it in.

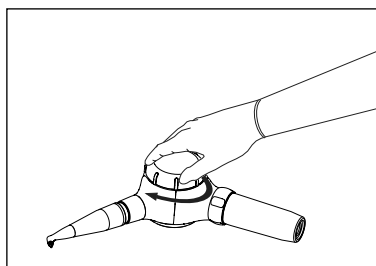7


Grab the air polisher device, slide the ring down and hold it in this position, insert the selected nozzle until it comes into contact, then release the ring;

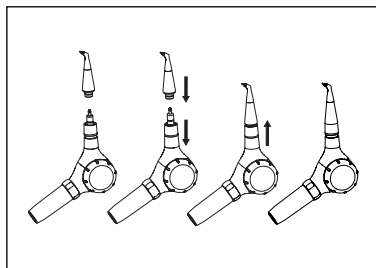8

Operate the pedal for 8-10 seconds to allow any powder residue to flow out from the handpiece;

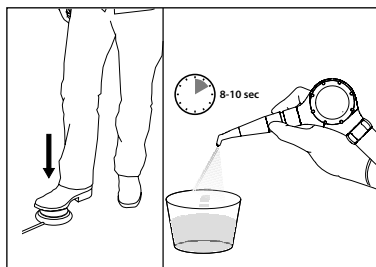9

6 CLEANING, DISINFECTION AND STERILISATION

The following tables are for indicative purposes only.

For the full procedures for cleaning and sterilisation of individual parts refer to the Chapters referenced in the tables.

⚠ **CAUTION:** It is prohibited to use methods not covered in the tables below.

⚠ **CAUTION:** The cleaning and sterilisation operations described below must be carried out at the time of first use and at all subsequent uses.

⚠ **WARNING: The SUBGINGIVAL PERIO TIPS are single-use sterile.**

The SUBGINGIVAL PERIO TIPS are supplied sterile and for single use. They must be used on one patient only during a single application, after which they must be removed and disposed of.

EN

air polisher device		
Phase	Chapter	Procedure
I	6.1	Preparation
II	6.2	Manual cleaning with detergent solution
III	6.3	Disinfection with isopropyl alcohol-based disinfectant
IV	6.4	Daily decontamination of the water circuit of the dental unit


NOTE: To clean the accessories, select a procedure (manual or automatic) from among those validated and proposed in this manual.

Accessories (Polisher Nozzles, K9 Wrench, Cleaning Needle)					
Phase	Chapter	Procedure	Polisher Nozzles	K9 Wrench	Cleaning Needle
V	6.5.1	Manual cleaning	x	x	x
	6.5.2	Automatic cleaning	x	x	
VI	6.6	Check cleaning	x	x	x
VII	6.7	Drying	x	x	x
VIII	6.8	Sterilisation	x	x	x

NOTE: Repeated reconditioning has a minimal effect on the devices and their accessories. The end of the service life of the accessories is generally determined by wear or damage resulting from use. Mectron guarantees the integrity of its sterilisable polisher nozzles for up to 250 reconditioning cycles.


6.1 Preparation

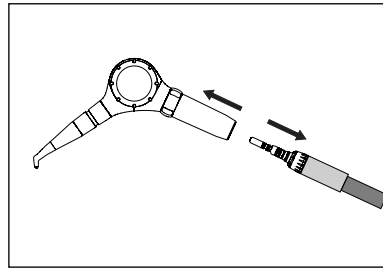Disconnect the air polisher device from the turbine of the dental unit;


If the PERIO Polisher nozzle was used, remove the single-use SUBGINGIVAL PERIO TIP using the supplied K9 wrench and proceed with disposal (Chapter 8 on page 41);

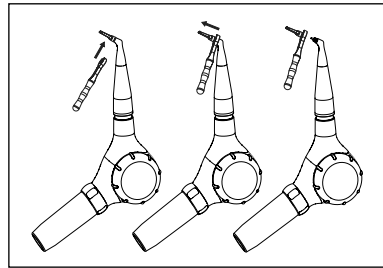
NOTE: Insert the K9 wrench exactly in the position shown in the figure.

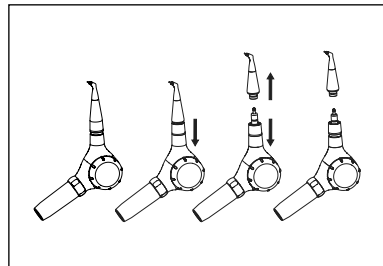Grab the air polisher device, slide the ring down and hold it in this position, extract the nozzle, then release the ring of the air polisher;

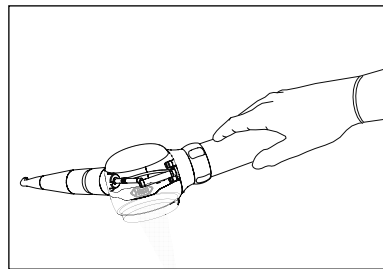Empty the powder container of the air polisher device following the instructions provided in Chapter 5.3 on page 18.

6.2 Cleaning the Device air polisher

The following procedure must be carried out on the air polisher device.

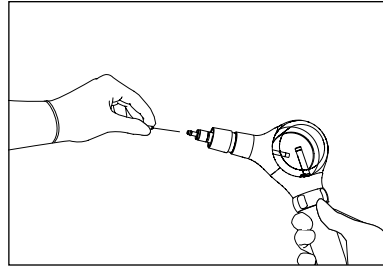
6.2.1 Required Materials

- Clean, soft, low-lint cloth;
- Cleaning solution (pH 6-9);
- Demineralised water.

6.2.2 Cleaning Method


Free the channel of the air polisher device of any powder residue using the supplied Ø 0.4 mm cleaning needle.

1


Clean the surface of the air polisher device using a clean, dry, low-lint cloth, dampened with a detergent solution (pH 6-9);

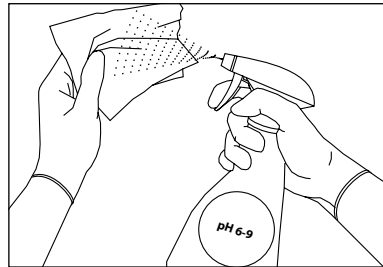2


Clean the surface of the air polisher device using a clean, dry, low-lint cloth, dampened with demineralised water to eliminate any detergent solution residue;

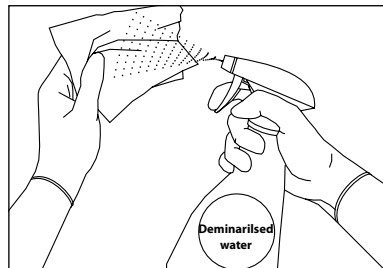3

Dry the air polisher device with a clean, soft, and low-lint cloth;

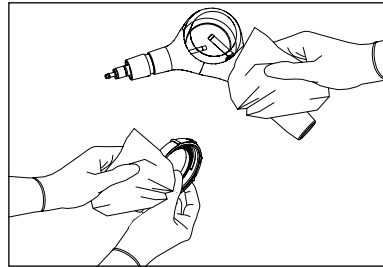4

⚠ **CAUTION:** Do not sterilise air polisher in an autoclave. It may be damaged and cause personal injury and/or property damage.

⚠ **CAUTION:** Do not immerse the air polisher device in various liquids and/or solutions as it may be damaged.

⚠ **CAUTION:** Do not immerse the air polisher device in the ultrasonic tank.

⚠ **CAUTION:** Do not use running water to clean the air polisher device.

6.3 Disinfecting the air polisher Device

The following procedure must be carried out on the air polisher device.

⚠ **CAUTION: DO NOT USE** as disinfectants::

- Very alkaline products (pH > 9);
- Products containing sodium hypochlorite;
- Products containing hydrogen peroxide;
- Products containing abrasive substances;
- Very acid products (PH < 4);
- Products containing aldehyde, amine and/or phenols;

- Acetone;
- Methylethylketone;

since they may discolour and/or damage the plastic materials.

The manufacturer disclaims all liability for any damage caused by the substances mentioned above. In case of damage caused by those substances, the Warranty will be void.

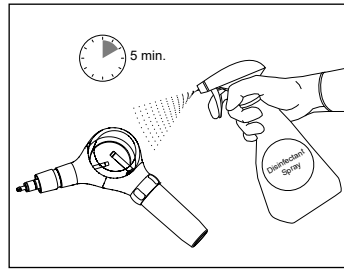
6.3.1 Required Materials

- Disinfectant made with isopropyl alcohol (> 60% v/v) and quarternary ammonium compounds^{b)};
 - Clean, dry, and low-lint cloths^{c)}.
- f) Disinfectant with tuberculocidal declaration (pH 6-9). The disinfection procedure has been validated by an independent body using the following disinfectants for surfaces: Discide Ultra - Disinfecting Spray - Manufacturer: Palmero Healthcare LLC.
 - g) The disinfection procedure has been validated by an independent body using the following product: Discide Ultra Soaked Wipes - Manufacturer: Palmero Healthcare LLC.

CLEANING, DISINFECTION AND STERILISATION


Completely wet the surface of the air polisher device with at least 12 ml of disinfectant spray^{b)} and leave wet for 5 minutes.

1

Dampen a clean, dry, low-lint cloth with 16 ml of disinfectant^{b)}. Rub the surface of the air polisher device for 5 minutes.

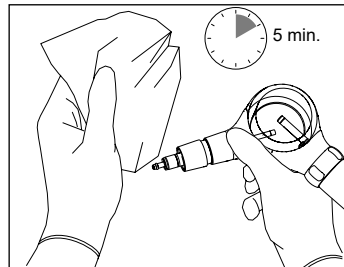2


⚠ CAUTION: Disinfect the device taking care not to damage the turbine quick coupling and silicone O-rings.

Repeat points 1 and 2.

3

Dry the air polisher device with a clean, soft, and low-lint cloth;

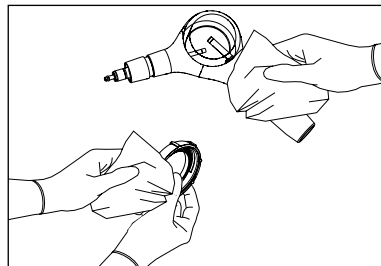4

EN

6.4 Daily Decontamination of the Dental Unit Water Circuit

This procedure must be carried out:

- at the start of the day;
- after each patient;
- at the end of the day, before stowing the device.

To prevent the spread of bacteria in the water circuit of the dental unit to which the air polisher device is connected, fill the dental unit water circuit tank with a chlorine dioxide solution at 0.1% to treat the dental unit lines, diluted with tap water at a ratio of at least 1:10 (as per the manufacturer's instructions^{d)}).

- h) Procedure validated using the product MicroCLEAR® Dental Unit Waterline Cleaner.

1. Prepare the decontamination solution by adding the required volume of concentrate to the dental unit water tank (volume required as per the manufacturer's instructions).
2. Fill the rest of the tank with tap water;
3. Operate the pedal and run the solution through the device for at least two minutes.

⚠ WARNING: Prepare the decontamination solution, carefully following the manufacturer's instructions.

It is advisable to empty and fill the dental unit tank with a fresh solution every day.

6.5 Cleaning the Sterilisable Accessories

The sterilisable parts of the appliance are:

- Polisher Nozzles;
- K9 Wrench;
- Cleaning Needles.

Before proceeding with operations to check cleanliness (*Chapter 6.6 on page 35*), drying (*Chapter 6.7 on page 36*) and then sterilisation (*Chapter 6.8 on page 36*), one of the three possible cleaning methods explained in detail in the following sub-chapters must be selected.

ⓘ CAUTION: The instructions provided below have been validated by the manufacturer of the device to allow their reuse. The person responsible for the maintenance, cleaning and sterilisation process, must ensure the repeated processes are effectively carried out using the equipment, materials and staff in the reprocessing structure, able to achieve the desired result. This generally requires the validation and systematic monitoring of the process. Similarly, all deviations by the person responsible for the processes, with respect to the supplied instructions, should be properly evaluated in order to determine their effectiveness and any potentially undesirable consequences.

6.5.1 Manual Cleaning

Manual cleaning can be carried out as an alternative to the automatic cleaning described in Chapter 6.5.2 on page 34.

6.5.1.1 Required Materials

- Enzymatic detergent at pH 6-9;
- Water;
- Container for immersion in the enzymatic liquid;
- Ultrasonic tank;
- Clean, soft, low-lint cloths;
- Brush with soft nylon bristles;
- 25 ml syringe;
- Demineralised water.

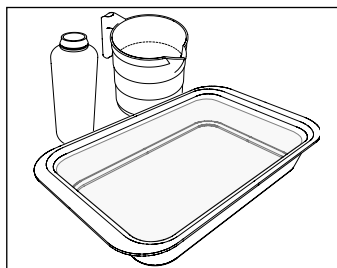
Obtain a bucket and ultrasonic tank large enough to hold the accessories needing to be cleaned.

Prepare an enzymatic detergent solution^{a)} at pH 6-9 according to the manufacturer's instructions, in a sufficient quantity to fill the bucket and ultrasonic tank. Use warm water for the bucket (30 °C - 35 °C) and water at ambient temperature (22 °C ± 2 °C) for the ultrasonic tank.

⚠ CAUTION: Once used, dispose of the enzymatic detergent correctly, do not recycle it.

- a) Process validated by independent bodies with Mectron ENZYMEC enzymatic detergent, 0.8% v/v.

1

6.5.1.2 Polisher Nozzles

Free the channel of the polisher nozzle of any powder residue using the supplied Ø 0.4 mm cleaning needle, from both sides. Blow compressed air into the central hole in the tip, from both ends;

⚠ **CAUTION:** Clean the channel of the nozzle using only the Ø 0.4 mm cleaning needle supplied with the appliance.

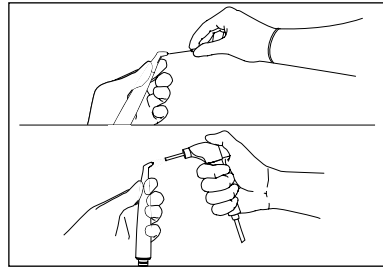
⚠ **CAUTION:** If the SUBGINGIVAL PERIO TIP is on the Polisher nozzle, remove and dispose of it.

Place the polisher tip horizontally in a clean container and add enzymatic detergent solution until the polisher tip is covered completely;


Leave to soak for 10 minutes at 22 °C ±2 °C. This procedure reduces the amount of blood, protein and mucous present on the polisher nozzle;

After 10 minutes of immersion in the enzymatic solution, gently brush the surface of the polisher nozzle using the brush with soft nylon bristles for at least 20 seconds, and eliminate all visible residue. Thoroughly clean the hard-to-reach areas such as the edges, recesses, and joints;


2


3

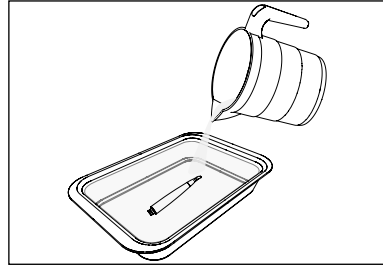4

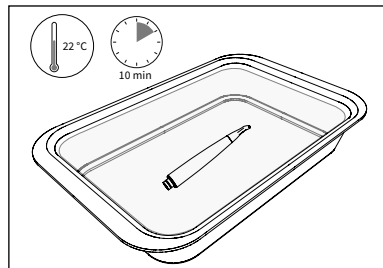
5

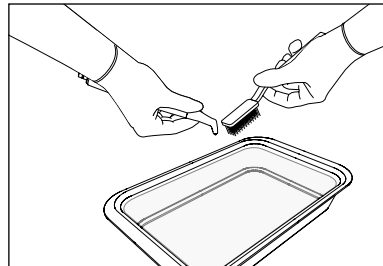CLEANING, DISINFECTION AND STERILISATION


Rinse the polisher nozzle under hot running water ($40\text{ }^{\circ}\text{C} \pm 5\text{ }^{\circ}\text{C}$) for about 10 minutes, moving it slightly to allow the water to reach the entire surface;

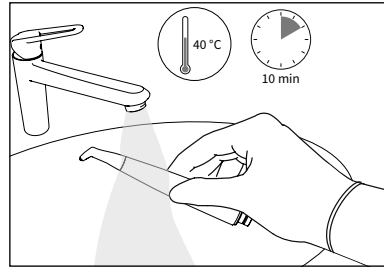6


Place the polisher nozzle in the ultrasonic tank immersed in the enzymatic detergent solution for at least 10 minutes at $22\text{ }^{\circ}\text{C} \pm 2\text{ }^{\circ}\text{C}$;

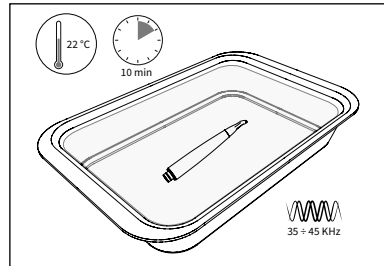7


Gently brush the surface of the polisher nozzle again using the brush with soft nylon bristles;

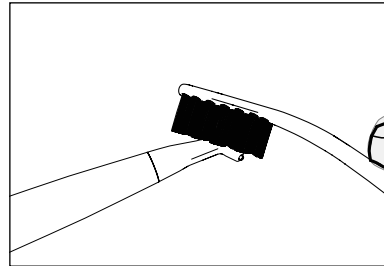8

Rinse the polisher nozzle under hot running water ($40\text{ }^{\circ}\text{C} \pm 5\text{ }^{\circ}\text{C}$) for about 10 minutes, moving it slightly to allow the water to reach the entire surface;

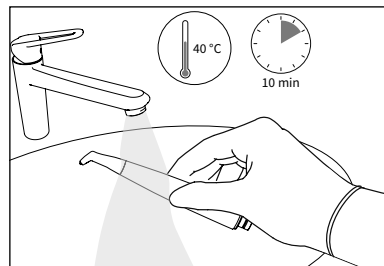
9

EN

Place the polisher nozzle in the ultrasonic tank immersed in demineralised water for at least 10 minutes at 40 °C ± 2 °C;

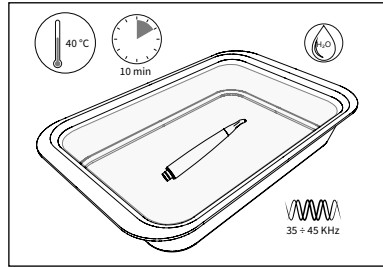
10

EN

After 10 minutes of immersion in the ultrasonic tank with demineralised water, rinse the internal channels of the polisher nozzle by injecting 20 ml of deionised water at ambient temperature (20 °C – 25 °C) using a syringe with needle, five times in a row.

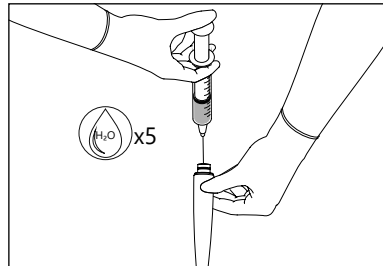
11

6.5.1.3 K9 Wrench

Place the K9 wrench in a clean container. Add enough enzymatic detergent solution to completely cover it. Leave the K9 wrench to soak in the enzymatic detergent solution for 10 minutes at ambient temperature (22 °C ± 2 °C);

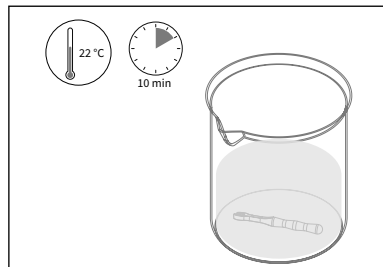12


While soaking in the enzymatic solution, gently brush all the surfaces until all visible soil has been removed.

Use a clean brush with soft nylon bristles for the external surfaces, and a clean pipe cleaner with soft nylon bristles for the internal cavities and grooves.

NOTE: Thoroughly brush all the following parts of the K9 wrench for about 20 seconds:

- Through-holes and internal channels;
- Hard-to-clean areas such as sharp edges and in particular, the interstices between the cutting cusps;
- External metal ring;
- Internal cavities, grooves and fissures.

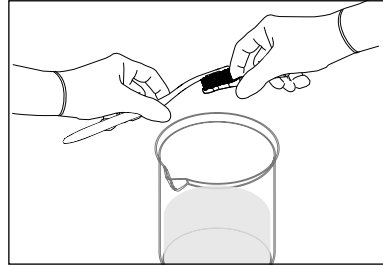
13

14


Remove the K9 wrench from the enzymatic detergent solution.

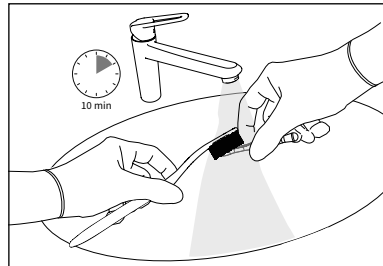Thoroughly rinse and brush all the surfaces of the K9 wrench (see previous point) under running water for at least 10 minutes;


15

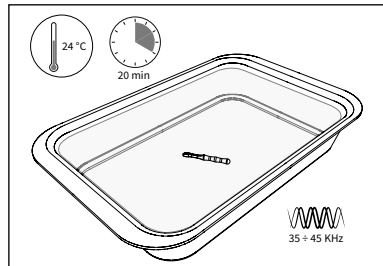Place the K9 wrench in the ultrasonic tank immersed in the enzymatic detergent solution at 24 °C \pm 2 °C and run a cycle for at least 20 minutes;

Remove the K9 wrench from the ultrasonic tank and rinse it under running water so as to eliminate all detergent residue. Brush the internal and external surfaces of the K9 wrench with a clean brush with soft nylon bristles, under running water.

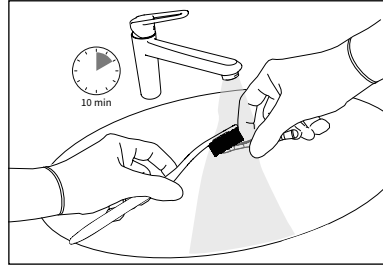16


EN

6.5.1.4 Cleaning Needles


Place the cleaning needle in a clean container, in a horizontal position, and add enough enzymatic detergent solution to completely cover the cleaning needle;

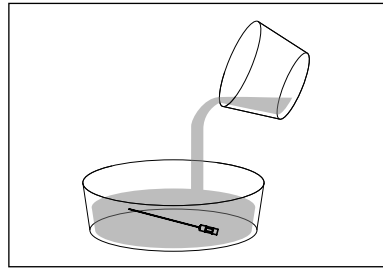17


Leave to soak for 10 minutes. This procedure reduces the amount of blood, protein, and mucus on the cleaning needle;

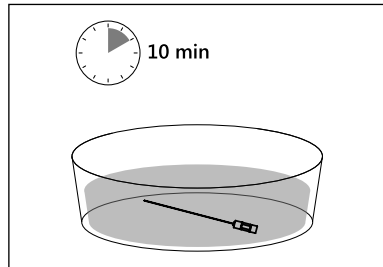18

During immersion in the enzymatic solution, delicately brush the surface of the cleaning needle using the brush with soft nylon bristles to eliminate all traces of visible soil. Thoroughly clean the hard-to-reach areas such as sharp edges and, in particular, the gaps between the shearing cusps;

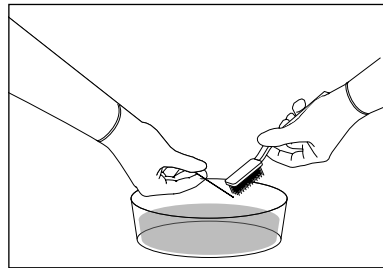
19

CLEANING, DISINFECTION AND STERILISATION


Delicately brush the surface of the cleaning needle under running water using the brush with soft nylon bristles;

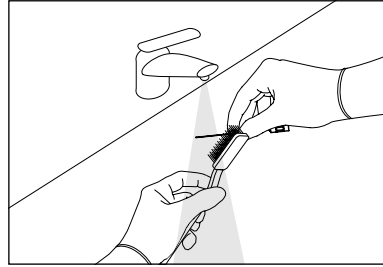20


Place the cleaning needle in the ultrasonic tank immersed in the enzymatic detergent solution for at least 10 minutes;

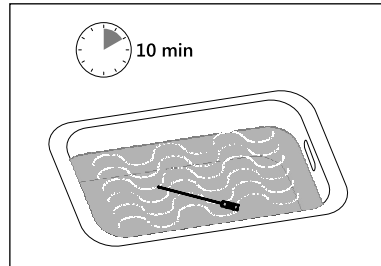21


Remove the cleaning needle from the ultrasonic tank and rinse with demineralised water;

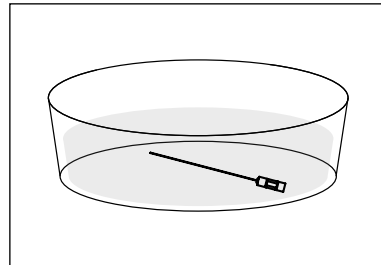22

Gently brush the surface of the cleaning needle again using the brush with soft nylon bristles.

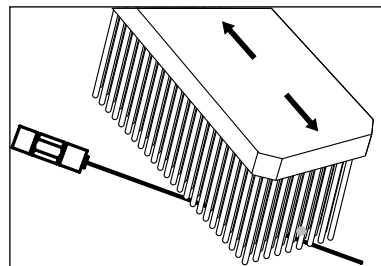23

EN

6.5.2 Automatic Cleaning

Automatic cleaning can be carried out as an alternative to the manual cleaning described in Chapter 6.5.1 on page 27.

NOTE: Procedure validated with:

- Miele PG8536 washer/disinfector;
- Miele DES-VAR-TD program;
- Alkaline detergent: neodisher® FA

(0.2% v/v);

- Neutralising liquid: neodisher® Z (0.1 % v/v)

EN

6.5.2.1 Required Materials

- Alkaline detergent;
- Neutralising liquid;
- Water;
- Metal basket;
- Adapters;
- Thermal-disinfector.

ⓘ **CAUTION:** Do not place the air polisher device in a thermal-disinfector.

ⓘ **CAUTION:** Do not place the cleaning needle in a thermal-disinfector.

⚠ **WARNING:** Avoid overloading the thermal-disinfector as this could compromise the effectiveness of cleaning.

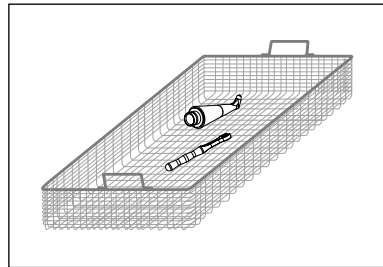
NOTE: Make sure that the accessories are appropriately blocked in the basket and cannot move during washing. Any blows could damage them. Position the instruments in a way that the water can flow through all the surfaces, even internal.

6.5.2.2 Cleaning Procedure

Position the accessories in a metal basket.

NOTE: If the Miele adapters are available, for their use refer to the leaflet "Instructions for maintenance and cleaning" and/or to the IFU provided with the adaptor.

1


Sequence and parameters applicable to the cycle:

- 1 min, rinse with cold water;
- 5 min, wash with alkaline detergent at 55 °C \pm 2 °C;
- 1 min, neutralisation with neutralising liquid, 1/3 cold water and 2/3 hot water;
- 1 min, rinse with water (1/3 cold water, 2/3 hot water);
- 5 min, thermal disinfection at 93 °C with demineralised water.

Thermal disinfection has not been tested experimentally. In compliance with ISO 15883-1, Table B.1 [4] thermal disinfection at a temperature of 90 °C for 5 minutes determines a value of A0 3000.

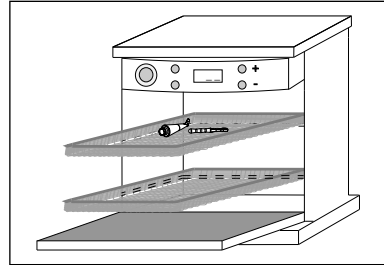2

EN

6.6 Cleaning Control

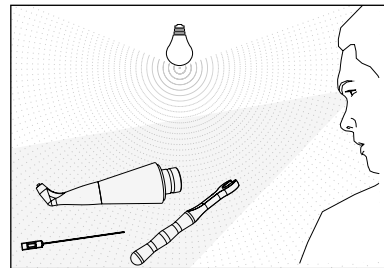
6.6.1 Required Materials

- Light source;
- 2.5x Magnifier.

6.6.2 Procedure for Cleaning Control

Once the cleaning operations have been completed, check the polisher nozzles, K9 wrench, and cleaning needle under an adequate source of light, if necessary using a magnifying glass 2.5X, paying attention to any parts that may conceal residual soil (threading, cavities, grooves) and, if need be, repeat the cleaning cycle if soil is still visible. Finally, check the integrity of any parts and elements that may have deteriorated during use.

1

6.7 Drying

6.7.1 Required Materials

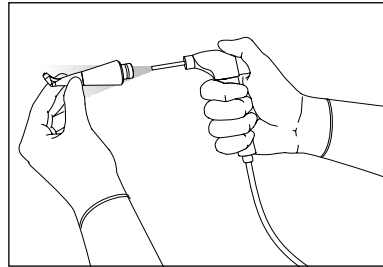
- Compressed air;
- Soft, low-lint cloth.

EN

6.7.2 Drying Procedure

Thoroughly dry all parts of the polisher nozzle, both internally and externally. To do so, blow compressed air both externally and through the internal through-hole. This will prevent the appearance of stains, streaks on the surface, or oxidation inside the polisher nozzle;


1

Dry the K9 wrench using a soft, low-lint cloth.

Thoroughly dry all parts of the cleaning needle by blowing them with compressed air.

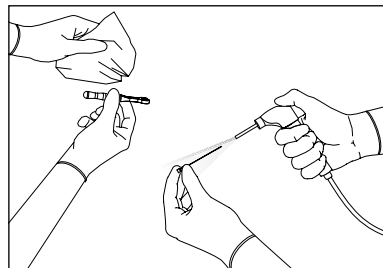2


6.8 Sterilisation

6.8.1 Preparation


Seal the polisher nozzles individually inside a disposable bag for sterilisation.

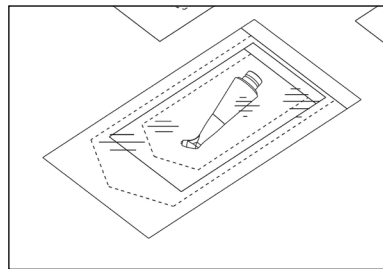1

Seal the cleaning needle and K9 wrench individually inside a disposable bag for sterilisation.

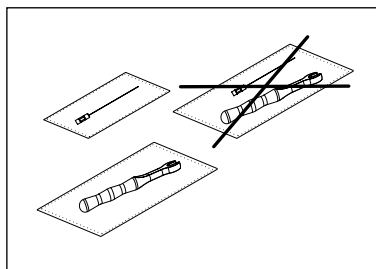2

EN

6.8.2 Sterilisation Method

The sterilisable parts are manufactured with materials that resist a maximum temperature of 135 °C for a maximum time of 20 minutes. Once individually bagged, carry out the steam autoclaving process.

⚠ CAUTION: Use sterilisation bags compliant with standard UNI EN ISO 11607-1.

The sterilisation process validated by Mectron S.p.A., in an autoclave with steam, guarantees a SAL 10⁻⁶ by setting the parameters indicated below:

- **Type of cycle:** 3 times Pre-vacuum (pressure min. 60 mBar).
- **Minimum sterilisation temperature:** 132 °C (interval 0 °C ÷ +3 °C).
- **Minimum sterilisation time:** 4 minutes.
- **Minimum drying time:** 20 minutes.

6.8.2.1 Special Information

Sterilisation parameters, in steam autoclave, used in Great Britain:

- Temperature: 134° C;
- Time: 3 minutes.

All stages of sterilisation must be carried out by the operator in compliance with the most current version of standards: UNI EN ISO 17665-1, UNI EN ISO 556-1 and ANSI/AAMI ST:46.

⚠ CAUTION: Do not sterilise the air polisher device.

⚠ WARNING: Infection control - Sterilisable parts

Carefully remove any residual organic matter before sterilisation.

⚠ CAUTION: Carry out the sterilisation by using only an autoclave with steam from water. Do not use any other sterilisation procedure (dry heat, irradiation, ethylene oxide, gas, low temperature plasma, etc.).

⚠ CAUTION: Do not exceed the permissible load of the steam-steriliser.

7 MAINTENANCE

In order to guarantee safety and proper operation, the device requires periodic maintenance.

Moreover, if faults occur during use, it is helpful to be familiar with some of the procedures

that may prevent said conditions and restore the device's proper operation.

Below are the main procedures and operations to be carried out as part of normal operations.

EN

7.1 Maintenance of the Polisher Nozzle

Following are the main operations to be carried out on the polisher nozzle.

7.1.1 Routine Maintenance of the Polisher Nozzle

In addition to the cleaning and sterilisation described in *Chapter 6 on page 21*, to be carried out at first use and at every subsequent use, the following operation should also be regularly performed:

- To avoid the build-up of residual powder, each time the polisher nozzle is removed, blow compressed air into the central rear duct of the polisher nozzle.

7.1.2 Clogged Polisher Nozzle

If the polisher nozzle is clogged, check and carry out the following procedures.

If the air passage of the polisher nozzle is clogged, free the polisher nozzle duct of any powder residue using the supplied \varnothing 0.4 mm cleaning needle in the front part, and \varnothing 0.8 mm cleaning needle in the rear part.

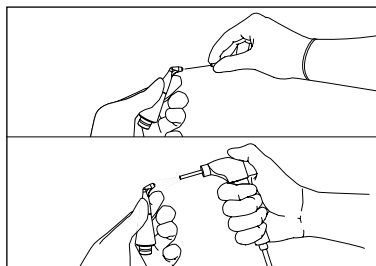
Blow compressed air into the central duct of the nozzle, from both ends;

⚠ CAUTION: Clean the channel of the polisher nozzle using only the cleaning needles supplied standard with the appliance.


⚠ CAUTION: If the SUBGINGIVAL PERIO TIP is on the Polisher nozzle, remove and dispose of it.

If the polisher nozzle is clogged due to excessive limescale in the water, it is recommended to immerse it in a 2% acetic acid solution for a few minutes.

1

2

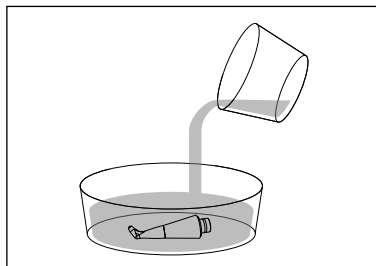7.2 Maintenance of the Core Unit

Following are the main operations to be carried out on the device core unit.

7.2.1 Routine Maintenance of the Core Unit

In addition to the cleaning and sterilisation described in *Chapter 6 on page 21*, to be carried out at first use and at every subsequent use, the following operations should also be regularly performed:

1. As explained in *Chapter 5.3 on page 18*, empty and clean the powder container each time the powder is changed and before cleaning, disinfection, and sterilisation.
2. Each time the polisher nozzle is removed, use the syringe to blow compressed air into the central duct of the connector of the nozzle on the device.

7.2.2 Clogged Core Unit

If the device core unit is clogged, check and carry out the following procedures.

If the air passage of the air polisher device is clogged, use the supplied cleaning needle to free the channel of any powder residue.

NOTE: The duct and powder chamber must be properly dried with a clean cloth and compressed air before reuse. Remember that too much moisture or liquid inside the powder chamber may block the device.

1

7.3 Checking the Connector of the Polisher Nozzle

EN

Regularly check the condition of the O-rings on the nose cone coupling. The o-rings are needed to separate the powder and water outlets.

The o-rings must always be in good condition without any powder residue or cuts.

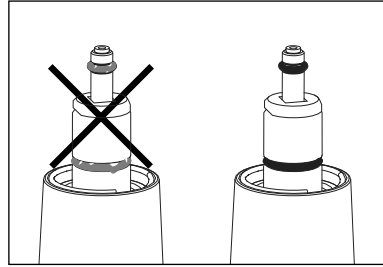
⚠ WARNING: Contraindications.

The nose cone may become clogged in the presence of O-rings that are broken or worn, or with powder residue.

⚠ WARNING: Use only original W&H accessories and spare parts.

In case of worn or broken O-rings, use only the kit supplied by W&H.

1

7.4 Checking the Turbine Quick Coupling

Regularly check the condition of the O-rings on the turbine quick coupling.

The O-rings are needed to separate the air and water outlets.

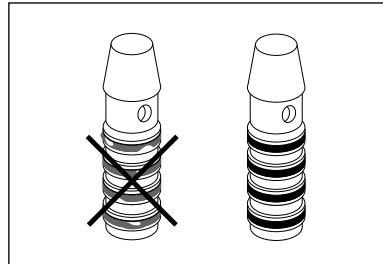
The O-rings must always be in good condition without any dirt or cuts.

⚠ WARNING: Contraindications.

The air polisher device may become clogged or fail to work properly in the presence of broken, worn, or dirty O-rings.

⚠ WARNING: Use only original accessories and spare parts supplied by the manufacturer of the turbine.

1

7.5 Routine Decontamination of the Dental Unit Water Circuit

To decontaminate the water circuit of the dental unit to which the air polisher device is connected after long periods of non-use or at first use, use troclosen sodium dihydrate ($\geq 1\%$) and sodium chlorite ($\geq 4\%$) tablets to treat the dental unit water circuits. Dilute the tablet in tap water as per the manufacturer's indications^{b)}.

b) Procedure validated using the product Vistatab™ Dental Waterline Cleaner Tablets by Hue-Friedy.

1. Fill the dental unit water tank with a volume of tap water sufficient to fill the entire water circuit.
2. Dissolve a suitable number of decontamination tablets in the water tank of the dental unit (as per the manufacturer's instructions^{b)});

3. Start the water circuit of the dental unit to which the air polisher device is connected until it is completely filled. The decontamination solution must flow out from the air polisher device;
4. Leave the decontamination solution to work inside the air polisher device and the water circuit to which it is connected;
5. Rinse the water circuit of the dental unit to which the air polisher device is connected with at least 500 ml of tap water.

⚠ WARNING: Prepare the decontamination solution, carefully following the manufacturer's instructions.

EN

8 METHODS AND PRECAUTIONS FOR DISPOSAL

⚠ WARNING: Hospital waste.

Treat the following objects as hospital waste:

- Polisher nozzles, when worn or broken;
- Cleaning needles, when worn or broken;
- K9 wrench, when worn or broken;
- Single-use SUBGINGIVAL PERIO TIP, after each use.

Disposable materials and materials that carry biological risk must be disposed of according to local regulations in force regarding hospital waste.

The device must be disposed of and treated as waste subject to separate collection. The purchaser is entitled to deliver the device to be disposed of to the dealer who supplies them with new equipment; at Mectron, instructions for proper disposal are available.

9 TECHNICAL DATA

EN

Device compliant with Regulation (EU) 2017/745	Class IIa
Appliance for intermittent operation:	60 sec. ON - 10 sec. OFF
The standard water supplied by the dental unit for a turbine is:	from 1.0 to 3.0 bar (from 1000 hPa to 3000 hPa) max 40 °C
The standard air supplied by the dental unit for a turbine is:	2.2 - 3.0 bar (2200 hPa - 3000 hPa)
Maximum water flow	71.5 mL/min
Maximum air flow	8.3 L/min
Operating Conditions	from +10 °C to +35 °C Relative humidity from 30% to 75% Air pressure P: 800hPa/1060hPa
Transport and Storage Conditions (Excluding Powder)	from -10 °C to +60 °C Relative humidity from 10% to 90% Air pressure P: 500hPa/1060hPa
Weights and dimensions	~ 155 g ~ 200 x 90 x 50 mm (L x W x H) ^{a)} Weights and dimensions may vary depending on the different configurations

Table 3 – Technical Data

a) **W = width ; L = length ; H = height**

10 TROUBLESHOOTING

10.1 Quick Troubleshooting

If the appliance does not seem to work properly, read the instructions again and then check the following table.

Problem	Possible Cause	Solution
The cap of the powder container doesn't unscrew	The air polisher device is connected to the turbine and the powder container is pressurised even if the pedal isn't pressed	Disconnect the polisher nozzle from the air polisher device to depressurise the internal circuit. If the problem persists, free the channel of the polisher nozzle of any powder residue using the cleaning needle (read <i>Chapter 6.5.1.2 on page 28</i>)
Powder leak through the powder container cap	Cap not correctly tightened	Screw the cap correctly
	Cap seal not properly positioned in its seat or worn	Check the position of the cap seal, or replace it if worn
	Powder residue in the thread	Blow compressed air onto the thread of the powder container (read <i>Chapter 5.3 on page 18</i>)

Problem	Possible Cause	Solution
<p>No powder flows out from the polisher nozzle during operation</p>	<p>Dental unit not connected to the air circuit</p>	<p>Check the connection between the dental unit and the air circuit</p>
	<p>The air polisher container is empty</p>	<p>Fill the powder container without exceeding the maximum level (read <i>Chapter 5.2 on page 14, point 9</i>)</p>
	<p>Polymer nozzle clogged due to an excessive amount of moisture in the powder or insufficient cleaning/maintenance</p>	<p>Read <i>Chapter 5.3 on page 18, Chapter 6.1 on page 22, Chapter 6.5 on page 26 and Chapter 7 on page 38</i>. Remove the powder from the container and clean it using a dry cloth</p>
	<p>air polisher device channel clogged due to an excessive amount of moisture in the powder or insufficient cleaning/maintenance</p>	<p>Read <i>Chapter 7 on page 38</i></p>
	<p>The amount of powder in the container exceeds the maximum level</p>	<p>Restore the correct level of powder in the container (see <i>Chapter 5.2 on page 14, point 9</i>)</p>
<p>No water flows out from the polisher nozzle during operation</p>	<p>Dental unit not connected to the water circuit</p>	<p>Check the connection between the dental unit and the water circuit</p>
	<p>Too much limescale in the water</p>	<p>Immerse the polisher nozzle in a 2% acetic acid solution for a few minutes. For the complete cleaning procedure see <i>Chapter 6.5 on page 26</i></p>
	<p>Dental unit water filter clogged</p>	<p>Check the connection to the dental unit water circuit</p>

Problem	Possible Cause	Solution
Inefficient therapy	Insufficient pressure in the air supply circuit	Check the pressure of the dental unit air circuit
	Insufficient or excessive powder level in the container	Restore the correct level of powder in the container (see <i>Chapter 5.2 on page 14, point 9</i>)
	Polisher nozzle clogged due to an excessive amount of moisture in the powder or insufficient cleaning/maintenance	Read <i>Chapter 5.3 on page 18, Chapter 6.1 on page 22, Chapter 6.5 on page 26 and Chapter 7 on page 38</i> . Remove the powder from the container and clean it using a dry cloth
	Irregular and/or intermittent water/powder flow	Check the O-ring of the polisher nozzle coupling (see <i>Chapter 7.1 on page 38</i>)
	The air polisher device is tilted more than 45° with respect to the correct position	Correctly reposition the grip of air polisher and use the dedicated polisher nozzles for 360° rotation without rotating the entire device

Table 4 – Quick Troubleshooting

10.2 Shipping to an Authorised W&H Service Centre

If technical assistance is required on the machine, contact an Authorised W&H Service Centre or your Dealer. Do not try to repair or modify the appliance and its accessories.

Clean and sterilise all parts that need to be sent to an Authorised W&H Service Centre by following the instructions in *Chapter 6* on page 21.

Leave the sterilised parts in the bag which certifies the sterilisation process.

The cleaning and sterilisation demands comply with the mandatory requirements for workplace health and safety protection pursuant to Legislative Decree 81/08 and Italian laws.

If the client fails to comply with the requirements, W&H reserves the right to charge them the cost of cleaning and sterilisation or to refuse goods that arrive in unsuitable conditions, returning them at the client's own expense for proper cleaning and sterilisation. The appliance must be returned suitably packaged and accompanied by all the accessories and a form including:

- Owner's data with telephone number
- Product Name

- Serial number and/or batch number
- Reason for return / description of failure
- Photocopy of the bill or invoice for the purchase of the appliance

⚠ CAUTION: Packaging

Pack the appliance in its original packaging to prevent damage during transport. If the appliance is returned in non-original or otherwise inadequate packaging with consequent damage caused by transport, W&H reserves the right to refuse assistance and to return the appliance at the expense of the end client.

Once the material is received at the Authorised W&H Service Centre, the qualified technical personnel will evaluate the problem. The repair will be made only upon acceptance by the end client. For further details please contact your nearest Authorised W&H Service Centre or dealer.

Unauthorised repairs can damage the system and void the warranty, releasing MECTRON of all liability for direct or indirect damages to persons or property.

11 WARRANTY

Before being marketed, all MECTRON appliances, are subjected to a thorough final check that verifies the full functionality.

MECTRON warrants the proxeo^{AURA} air polisher device, purchased new from a W&H dealer or importer, against defects in material and workmanship for a period of 1 (ONE) YEAR from the date of purchase.

During the warranty period, W&H undertakes to repair (or, at its discretion, to replace) the parts of products free of charge, which, according to its judgement, are proven to be defective.

Full replacement of MECTRON products is not covered by the warranty.

The manufacturer disclaims all liability for direct or indirect damage to persons or property, in the following cases:

- The appliance is not used in accordance with the intended use.
- The appliance is not used in accordance with all the instructions and requirements described in this manual.
- Assembly operations, extensions, re-adjustments, modifications, replacements and repairs are carried out by personnel not authorised by MECTRON/W&H in breach of what is provided in this manual also in regard to the origin of the authorised material;
- The environmental conditions for preservation and storage of the device are not complying with the requirements indicated in *Chapter 9 on page 42*;
- The installation or transport of the appliance is not performed as specified in this manual or in other documentation provided by W&H, or in any case available on the website of the latter;
- The appliance or component thereof is purchased from a subject not authorised by W&H;
- The appliance, including its subcomponents, parts or assemblies, are altered or modified with respect to the indications in this manual;
- Accidents, misuse, abuse, abnormal use, negligent use, intentional misconduct or use exceeding the allowed and recommended limits of the appliance, or in case of its normal wear or deterioration;
- If the defect or non-conformity has not been promptly and readily communicated in writing to W&H as set out in this manual;
- If the damage, costs or expenses are caused by events of force majeure.

In any case, W&H will not recognise indemnity or compensation for loss of use, inconvenience, loss of profits, loss of business, loss of business opportunities, damage to reputation, and any incidental or consequential damages arising out of or relating to the appliance.

The expected service life of the device is minimum 2 years.

The service life / duration does not establish a limit of use; the service life of the device defines the period of time after installation and/or commissioning, during which the original performance, or in any case performance suited to the intended use, is guaranteed without there being any degradation such as to compromise functionality and reliability.

The service life is a minimum qualitative objective of the design, therefore, individual parts or components may guarantee superior performance and reliability with respect to that declared by the manufacturer.

The service life assumes compliance with the maintenance schedules set out in this manual, does not include components normally subject to "wear", and is not linked to the warranty period: the service life does not establish any implicit or explicit extension of the warranty period.

CAUTION

The warranty starts from the date of purchase of the device, which evidence is given by the delivery note/purchase invoice issued by the Dealer / Importer or, in case of device with activation code, from the date of activation of the same.

In order to avail of the warranty service, the client must return the appliance, at their own expense, to the W&H dealer/importer from which they purchased the product.

The appliance must be returned together with the original packaging, accompanied by all the accessories and by a form containing:

- The data of the owner and telephone number;
- The data of the Dealer / Importer;
- Photocopy of the delivery note/purchase invoice of the appliance by the owner where are reported the date, the name of the appliance and the serial number;
- Description of the failure.

The transport and the damage caused by transport are not covered by the warranty.

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© Mectron S.p.A. 2022. Tutti i diritti riservati. Nessuna parte di questo documento può essere riprodotta, in qualsiasi forma, senza il consenso scritto del proprietario del copyright.