
DENTAL AMALGAMATOR

SYG-200

USER MANUAL.....	2
MANUAL DEL USUARIO.....	8
MANUEL D'UTILISATION.....	14
INSTRUKCJA OBSŁUGI.....	20
MANUAL DO UTILIZADOR	26
BEDIENUNGSANLEITUNG	32
MANUALE D'USO	38
HANDLEIDING	44
РУКОВОДСТВО ПОЛЬЗОВАТЕЛЯ.....	50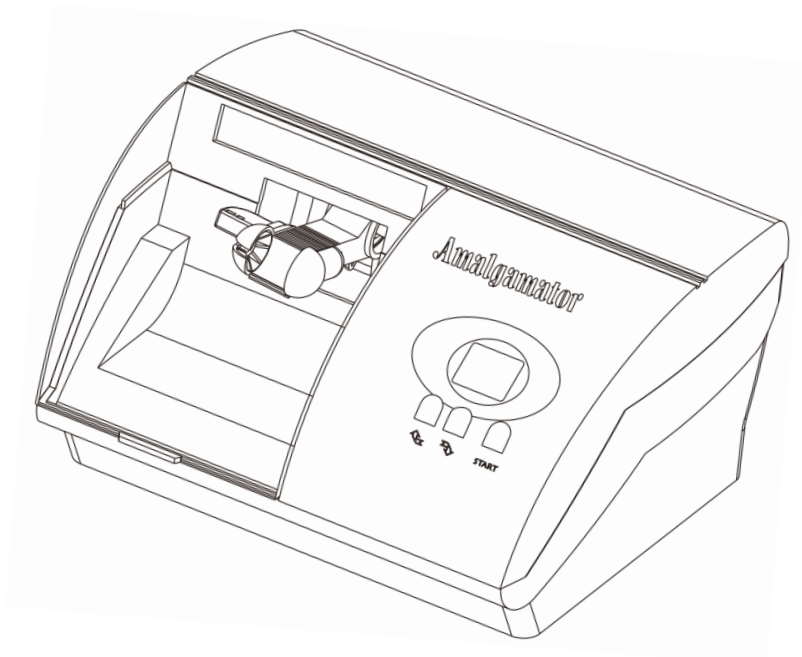

Warning:

Do not modify this equipment without authorization of the manufacturer.

1. Preface

Thank you for your confidence in the SYG-200 dental amalgamator. This device is controlled by a microprocessor; it has a function to set the running time exactly and is operated very easily. Please read this manual carefully before you begin to use the mixer and keep it close to the mixer.

SYG-200 shall only be operated by qualified dental personal.

Intended use

This device is used to mix silver with mercury in a capsule presentation and to make alloy in the best manner. So the quality of teeth recovery level is elevated. It is used instead of the former manual method, not only creating easier mixing conditions but also decreasing the mercury pollution in the dental practice. So the health of the human body is preserved.

2. Product diagram

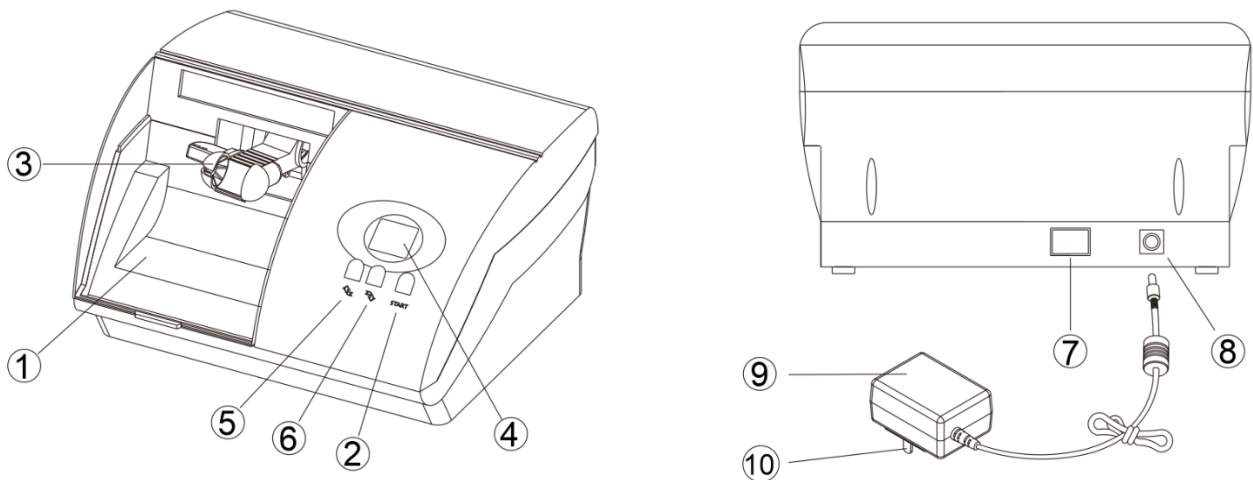
Fig.1

- | | | | |
|----------------|-----------------|------------------|--------------|
| ① Cover | ② Start | ③ Capsule holder | ④ Display |
| ⑤ Extend time | ⑥ Decrease time | ⑦ Power ON/OFF | ⑧ Power jack |
| ⑨ Power supply | ⑩ Power plug | | |

3. Technical specifications

1) Power supply

Power input: $\sim 100-240V$, 1.0-0.5A, 50/60Hz; Output: $24V \text{---} 1.5A$; Fuse: T2A/250V;

Model: EM1024RR; Manufacturer: EDACPOWER ELECTRONICS CO.,LTD.

2) Main unit power input: $24V \text{---} 1.5A$

3) Oscillation frequency: 4600rpm

- 4) Noise level: $\leq 65\text{dB}$
- 5) Time setting: 01s-99s
- 6) Intermittent cycle: 3mins running @1mins stopping
- 7) Overvoltage category: Class II
- 8) Enclosure: IPX0
- 9) Operating conditions
 - Temperature: $+10^{\circ}\text{C} \sim +40^{\circ}\text{C}$; Relative Humidity: 10% to 90%
 - Ambient pressure 850 hPa to 1060 hPa
- 10) Storage and transport conditions
 - Temperature: $-10^{\circ}\text{C} \sim +40^{\circ}\text{C}$; Relative Humidity: 10% to 90%
 - Ambient pressure 500 hPa to 1060 hPa

4. Installation instructions

- 1) Take the main unit out of the packaging box, turn the bottom up, and place it on the desktop with a protective pad. As shown in Fig.2, remove the four red shipping screws with a screwdriver.
- 2) Turn the main unit upside down and place on a flat table.
- 3) Connect the power supply to the main unit, and the other end to the power source.

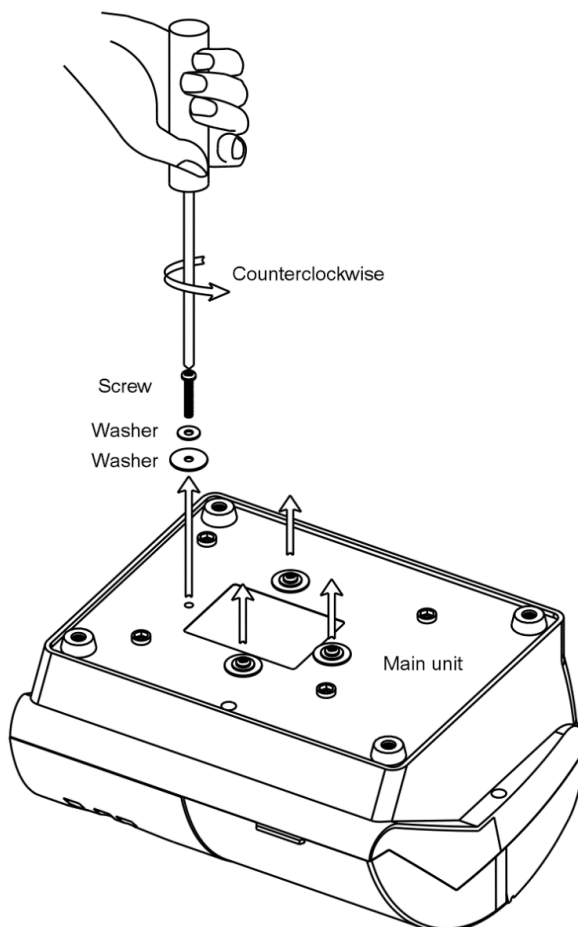
Fig 2

5. Operation method

- 1) As shown in Fig.1, turn the power switch on the rear side of the main unit to the “-” position to turn on the power. At this time, the screen will display the initial working time (40 seconds).
- 2) Choose fine capsules of silver with mercury.
- 3) Press the cap on the top of the capsule into the capsule, allowing the inner silver powder and mercury to be in close contact.
- 4) Open the transparent protecting shield and fix the capsule onto the mixer’s holder, then close the cover, As shown in Fig.3.

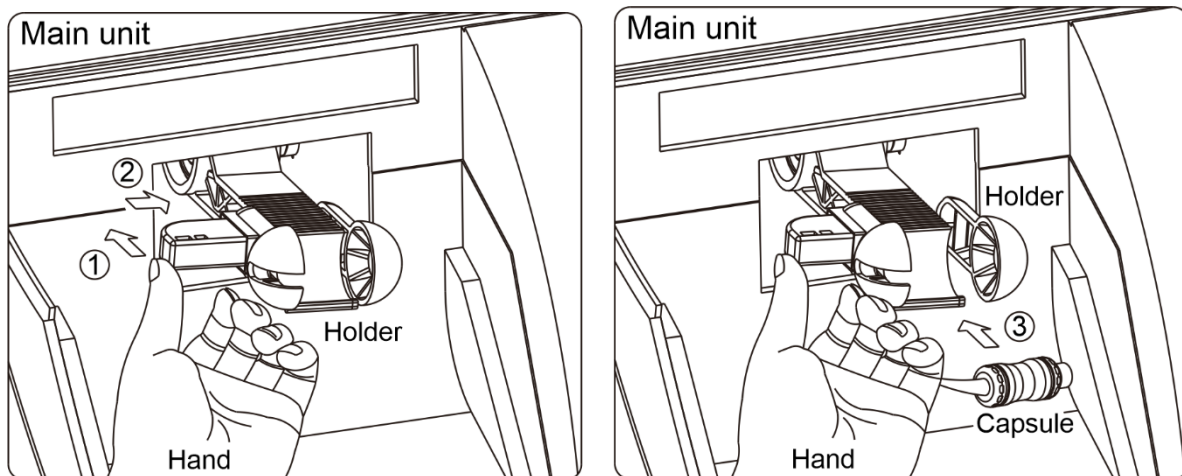
Fig 3

- 5) Time set – Press the "↺" key to extend the time, press the "↻" to decrease the time.
 - 6) Press the "START" key, the mixer starts running and stops at the end of the set time. If you want to stop a running mixer, press the "START" key again.
 - 7) When the mixer has stopped, you can open the cover and retrieve the mixed capsule that can now be used.
 - 8) After the work is completed, please turn the power switch on the rear side of the main unit to the “O” position to turn off the power.
- In order to fully lubricate the transmission mechanism of the machine, it is recommended that when the user first uses the machine every day, let the machine run without load for 60 seconds first to make the machine reach the best working condition.

6. Precautions, maintenance and disposal

1) Precautions

- a) The main four red transport fixing screws on the bottom of the device must be removed before first use.
- b) Put the device on the flat desktop when in use, and maintain good ventilation.
- c) Make sure that the power plug is firmly attached, and the power plug is easily accessible.

- d) Please close the transparent protection shield before pressing the "Start" button to running the amalgamator.
- e) Please pay attention to rest the device to avoid overheating.
- f) The device is non-AP-type equipment, and non-APG-type equipment.
- g) Please be strictly in accordance with the operation instructions, such as violation of normal operation which will cause damage to the equipment or operator injury.
- h) Do not position the mixer so that it is difficult to operate the disconnection device.
- i) Plug of the power adapter is used to isolate the ME equipment from the supply mains.
- j) Put the device on the flat desktop, and check the product labels and make sure that the product is properly connected according to user manual.
- k) When the mixer stops working, do not press the "start" button before the holder completely stops swinging to avoid damage to the device structure.

2) Maintenance

- a) Keep equipment appearance clean, the device enclosure and capsule holder can be cleaned by alcohol, Cleaning operations should be carried out at the end of every day.
- b) If the device has not been used for a long time, please disconnect the power and place it into the box for storage.

3) Disposal

- a) The product must not be discarded in domestic household waste.
- b) Should you wish to definitively dispose of the product, please comply with the regulations which apply in your country.

7. Troubleshooting and repair


1) Troubleshooting

No.	Symptom	Solution
a)	The LCD has no numeric display	Check whether the power cord is loose.
b)	Too loud operating noises	Check if the screws under the bottom of the device have been removed or not. Please remove before first use.

- Remark:
To prepare for the above operation, be sure to disconnect the power and unplug the power cord.
- 2) The Device has been tested in compliance to the standard IEC 60601-1-2: 2014 Electromagnetic compatibility restrictions. But do not rule out potential weak electromagnetic interference to other devices or receiving sensitive chance of interference to other equipment, if the above problems occur, it is recommended to correct the interference by the following measures.
 - a) Connect the device to an outlet of a receiver circuit different from the one used by the sensitive equipment.
 - b) Increase the distance between the device and the receiver of sensitive equipment.
 - c) Restart the device.

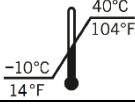- 3) If you cannot define the problem, please return it to service. Non-professional maintenance personnel must not disassemble it as wrong operation may cause more damage.
- 4) Repair
 - a) The equipment should be maintained by professionals.
 - b) To ensure quality, please use the repaired parts which the company provides.

8. Symbols

Symbols	instructions
	Alternating current
	Direct current
	Class II
	ISO 7000-0434: Caution, consult ACCOMPANYING DOCUMENTS
	ISO 7010-M002: Follow instructions for use
	CE Mark
	Manufacturer
	AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY
	Recommended storage and transport temperature
	Protect against moisture
	Fragile, handle with care
	Date of manufacture

	Do not dispose of as household waste
	EU Medical Device

9. Warranty

The distributor gives a 12-month warranty period if the device cannot be used as intended due to quality defects. This warranty begins on the day of purchase. The distributor reserves the right to either repair the device or replace individual parts or the entire device at its discretion. This warranty does not cover wearing parts or fuses.

10. Scope of delivery

- | | |
|-----------------|------|
| 1) Main Unit | 1pcs |
| 2) Power supply | 1pcs |
| 3) Power plug | 1pcs |
| 4) User manual | 1pcs |

11. Reporting obligation

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's condition and a serious risk to public health that arise or could have arisen in association with the use of SYG-200 must be reported to the manufacture and the competent authority of the Member State in which the user and/or patient is established.

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E.M.S. Electro Medical Systems (Hangzhou) Co., Ltd.