
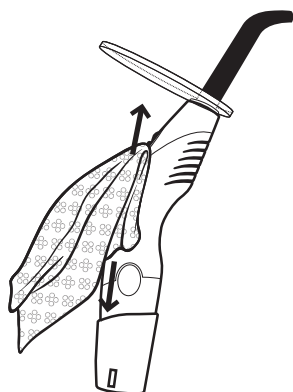



# Demi<sup>+</sup>Plus

LED Light Curing System  
Powered by Demetron™ Technology

## Product Care Tips

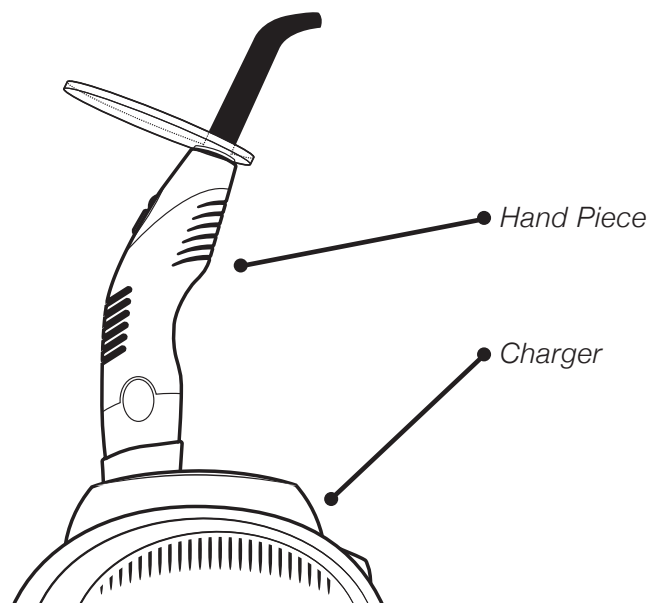


- **Do** wipe handpiece.

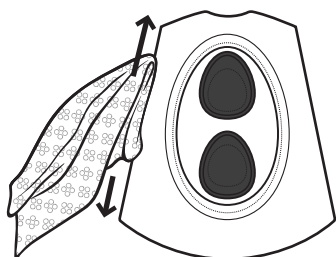

- **Do** wipe charger base.

- **Do** let handpiece air dry completely before placing into charger base.
- **Do** use CaviWipes™ or any disinfectants containing less than 20% alcohol.
- **Do** wipe handpiece separately from charger base.

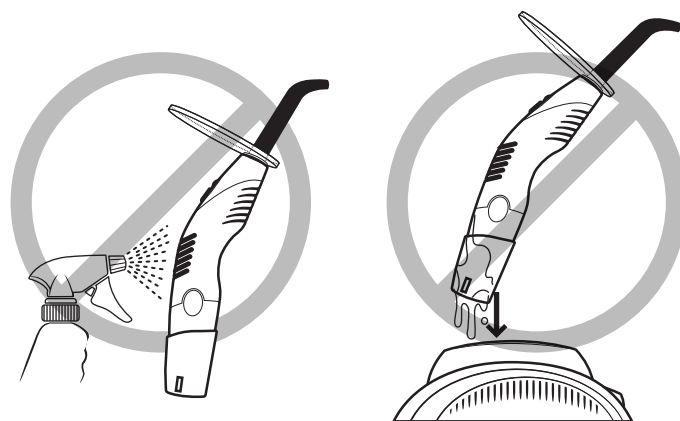


- **Do not** spray handpiece.



- **Do not** place wet handpiece in charger.



- **Do not** spray handpiece while in charger base.

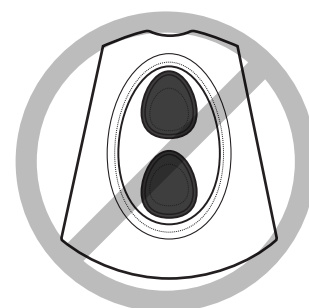

- **Do not** allow sockets in charger to get wet.

- **Do not** use denatured alcohol, Lysol® phenol, ammonia, or iodine solutions.

**FOR COMPLETE INSTRUCTIONS ON HOW TO MAINTAIN YOUR DEMI PLUS, PLEASE REFER TO THE DIRECTIONS FOR USE.**