# Multisteril

**Ref: DM401ALP**

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## IMPORTANT

**Dear Sir,**

Thank you for purchasing our new liquid Multisteril CD Ctrl + Alt. The new liquid has certified cleansing and disinfectant action which is increasing device performance.

**Multisteril CD Ctrl+Alt has been expressively designed for MULTISTERIL SYSTEM with the support of important boards as the University of Dresden which has tested and certified system efficiency; For more information please visit [http://www.tecnogaz.com/tecnogaz/product\\_top.aspx?id=1218](http://www.tecnogaz.com/tecnogaz/product_top.aspx?id=1218)**

If you are using this liquid for the first time, make sure to empty tank completely before putting it into the inside of the machine, following the directions on the back.

Unless you do this, you will be mixing two different liquids and you will notice that the liquid in the tank will take on a milky appearance.

Please also be advised that this liquid must have a concentration of only 1%, thus enter the machine's menu, section CHANGE CYCLES and change the voice INLET CLEANER, this will allow you to save 50%.

We remain at your disposal for any information,

**TECNO GAZ S.p.A.**

# USE INSTRUCTIONS

## Detergent reservoir draining

During the appliance use, when injection of detergent is required, in order to drain completely the detergent reservoir, press and hold down button **SELECT** and at the same time hold down button **OK**.

After 5 seconds the peristaltic pump starts, during this phase an intermittent acoustic signal is sent out which stops only when the phase is over, the detergent is drained directly into the tank through the nozzle on the lid.

After about 10 min slightly open the lid (without letting the lid opening sensor intervene) and make sure no detergent comes out of the nozzle.

Once the draining is completed, stop the process using button **STOP**

## Cycle changes

Select the cycle to be modified.

Press OK to display the MENU.

Available parameters:

Description	Parameters	Step
Standby	YES - OFF	
Detergent infeed	0 ÷ 6 %	1%
Heating	YES - NO	
Degas function	YES - NO	
SWEEP function	YES - NO	
Ultrasound cleaning	0 ÷ 30 min	1 min
Drying	YES - NO	

Press **OK** to scroll the cycle parameters.

Press **SELECT** to modify the value of the selected parameter..

**The menu of the parameters and setup scrolls circularly.**

Once the setup is completed select EXIT and press **SELECT** to quit the MENU.

**Changes are saved.**

**YOU CAN FIND MORE INFO IN THE USER'S MANUAL INCLUDED IN THE DEVICE**

**Ref: DM401ALP**  
BOX OF 6 BOTTLES OF 1 LT

Liquid concentrate for the non protein-fixing cleaning and disinfection of dental/medical instruments and rotating precision instruments in MultiSteril ultrasonic device by Tecno-Gaz.

The Institute of Hygiene and Medical Microbiology, Faculty of Medicine at the Technical University of Dresden performed between May and July 2014 technical and microbiological tests on the use of multifunctional vessel Multisteril by Tecno-Gaz in association with the liquids Ctrl Alt (Multisteril CD). The tests showed the effectiveness of the cleaning and disinfecting of the instruments when using the device Multisteril and the Multisteril CD. This according to the EN 15883 European Standard Norm which is specifying the particular requirements for washer and disinfection devices (WDS) developed to achieve the disinfection level that can be achieved by a thermal cleaning and disinfection.

Recommended settings with device

**Multisteril Tecno-gaz**

**Recommended content: 1%**

	(according to type of load)
Standby	
Detergent infeed	1 %
Heating	SI
Degas function	SI
SWEEP function	NO
Ultrasound cleaning	20 min
Drying	SI

**Composition (100 g):**

7,0 g Laurylpropylenediamine,

3,5 g Dodecyldipropylenetriamine,

2,2 g Alkylpropylenediamineguanidine;

Rest ad 100 g: emulsifying agents, complexing agents, tensides, deionized water

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