

Ai-Dental Quick Installation Guide

Preparation before use

Check the attachments and data

Device connection

- Place the device on a flat place and avoid shaking
- Take out the power line and connect the device

Install and run Ai-Dental software on the computer

First-time installers are recommended to use their browsers to scan the code to watch related videos.



i-Scan Scanner Video

The videos include: i-Scan Main Unit Installation, i-Scan Client Installation, i-Scan Operation Demonstration Video.

Installation key points

- Insert the software installation disk into your computer and run the software installation package.
- Click “Next” and a Software License Agreement will pop up. Select [Agree] to enter the installation interface.
- Main unit installation: Select the server during installation.
Client installation: Unselect the server during installation.
- After the client is installed, click [Search Server] to select the server.

Notes

In the process of installation or use, if you encounter the problem of unsuccessful client connection or unsuccessful mobile phone connection, please scan the QR code below to download the information package.



i-Scan Scanner
Information Package