MC2 & Isolite

Uniquement la calotte amovible est stérilisable / Ne pas lubrifier
Only the removable sleeve can be sterilized / Do not lubricate

- FRA** Mode d'emploi
- DEU** Anleitung
- ENG** Instruction
- ESP** Instrucciones de uso
- ITA** Istruzioni
- RUS** Инструкция по эксплуатации
- JPN** 取扱説明書

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English

Intended use

Medical devices entirely made in Switzerland by Bien-Air Dental SA.

Type

MC2 IR, MC2 GTAV, Isolite 100 and 300 electric dental micromotor with brushes. Non-sterilizable, protected to the oil of the hand-pieces. Removable, sterilizable sleeve. Interchangeable brushes.

MC2 LED description:

micromotor with internal spray and light with LED and switch integral with the motor.

MC2 IR description:

without light, internal spray.

MC2 GTAV description:

micromotor with higher torque, self-ventilating.

Isolite 100 description:

micromotor with Isolux lighting system with electric contacts to supply the bulb situated in the contra-angle.

Isolite 300 description:

micromotor with internal spray and light with bulb and switch integral with the motor.

Product intended for professional use only. Use in dentistry for prophylaxis, general dentistry and endodontic work, and in the field of body care and health treatment. The device is not designed for use in an explosive atmosphere (anaesthetic gas).

Any use other than that for which this device is intended is prohibited and may prove dangerous.

Technical data and assembly

Classification

Class IIa in accordance with European Directive 93/42/EEC concerning medical devices. This medical device is in compliance with the legislation in force.

Coupling

The most commonly used coupling in the world as per ISO Standard 3964.

Dimensions

MC2 GTAV: 24x92.6 mm / Isolite: 23.3x86 mm / MC2 IR: 23.5x81 mm / MC2 LED: 23.3 x 86 mm.

Weight

MC2 GTAV: 111 g / Isolite: 97 g / MC2 IR: 75 g.

Speed

Maximal 40.000 rpm.

Recommended rotation speed

60 rpm (minimum is variable, depending on the electronic power supply used) up to 40,000 rpm \pm 10%; rotation in both directions.

Nominal voltage

In accordance with ISO 11498:
0 – 24 Vdc.

Continuous operation

MC2 GTAV: 1.5 A; approx. 0.8 Ncm
Isolite: 1 A; approx. 0.6 Ncm
MC2 IR: 1 A; approx. 0.6 Ncm
MC2 LED: 1 A; approx. 0.6 Ncm.

Intermittent operation

According to CEI 34-1, Type S3. 3 A for 60 seconds; 5 A for 10 seconds; air cooling in 3 min. or without air in 20 min.

Mechanical power

MC2 GTAV: 74.2 W / Isolite: 54 W / MC2 IR: 56 W, MC2 LED 54 W.

Torque

MC2 GTAV: 2.1 Ncm / Isolite: 2.3 Ncm / MC2 IR: 2.3 Ncm, MC2 LED 2.3 Ncm

Idle current

0.3 A - 24 Vdc

Current at max. power

MC2 GTAV: 5 A / Isolite: 4 A
MC2 IR: 4 A / MC2 LED : 4A

Internal resistance

MC2 GTAV: 2.15 Ω / Isolite: 2.61 Ω / MC2 IR: 2.73 Ω / MC LED: 2.61 Ω

Noise

In accordance with ISO 7785-1, less than 62 dBA at 45 cm (17.72 in).

Bulb

Isolite 300: 3.5 V; 0.74 A.

LED

variable from 2.5 to 4.0 Vdc, 0.1 to 0.5 A, 15-38 klux.

Comments

The performance of the micromotor varies, depending on the type of electronic power supply used. We recommend that you use Bien-Air Dental electronic control units for optimum micromotor performance.

Electromagnetic compatibility

Corresponds to the electromagnetic compatibility in accordance with IEC 60601-1-2. Declaration by the manufacturer regarding electroma-

gnetic compatibility: refer to the tables on pages 41-42-43-44.

Electrical safety

According to IEC 60601-1 standard (General safety for Medical Electrical Equipment), the device shall be classified as a class II type B device.

Information

The technical specifications, illustrations and dimensions contained in these instructions are given only as a guide. They may not be the subject of any claim. The manufacturer reserves the right to make technical improvements to its equipment, without amending these instructions. For all additional information, please contact Bien-Air Dental SA at the address indicated on the back cover.

Assembly

Connecting the hoses

Connection recommended using hoses **see table page 17**. Check the rear of the motor and the joint on the hose are clean. Line up the connection components on the motor with those of the plug and insert until stop position.

Fully tighten tube retainer. Replace sleeve **fig. 2-4**.

Cooling

Via compressed air from the unit. Place the flow-meter on the coupling and regulate the flow at 8 norm-liter/min. for the Isolite and MC2 LED micromotor, and at 3 norm-liter/min. for the MC2 micromotor **fig. 1**, except for the MC2 GTAV micromotor, which is self-ventilating.

Important

Liquid or spray must not be allowed to penetrate inside the micromotor, due to risk of irreversible damage. Never connect an instrument with a micromotor that is running.

Changing the bulb

Isolite 300 **fig. 5**. Box of 5 bulbs. Wear rubber gloves when carrying out this changing operation.

Changing the LED

MC2 LED: The LED must only be changed by a Bien-Air Dental-approved repair centre.

Changing the brushes

Note: fine work; keep the brushes dry,

never in contact with the lubricant, otherwise there is a risk of irreversible damage. Before removing the two carbon brushes, wipe the back of the micromotor with a dry cloth.

Remove the sleeve

Micromotor MC2 IR **fig. 7**

Micromotor MC2 GTAV **fig. 8**

Micromotors Isolite 300 and 100 **fig. 6**

To change the two carbon brushes, remove the used carbon brushes. Blow dry, clean air across the apertures at the rear of the body of the micromotor **fig. 9**.

To reassemble, put the brushes in place and secure them **fig. 10** with the spring.

Replace the sleeve

Take care not to damage the O ring when replacing the sleeve.

Micromotor MC2 IR **fig. 7**

Micromotor MC2 GTAV **fig. 8**

Micromotors Isolite 300 and 100 **fig. 6**

Seals replacement

To change the O rings REF 1300145-010 **fig. 11**. Only valid for motors with internal spray.

Changing of the O ring REF 1301646-010 **fig. 12**, not valid for the Isolite 100 and 300 micromotors. Changing of the O ring REF 1300992-010 **fig 8 + 13**, not valid for the GTAV micromotors.

Disposal

This device must to be recycled. Electrical and electronic equipment may contain dangerous substances which constitute health and environmental hazards. The user must return the device to its dealer or establish direct contact with an approved body for treatment and recovery of this type of equipment (European Directive 2002/96/EC).

Maintenance

Important
Never lubricate the electric brush motor.

Only use maintenance products and components from Bien-Air Dental. The use of other products and components can void the guarantee.

Precautions for maintenance:

The universal precautions, in particular wearing of personal protective equipment (gloves, goggles, etc.), should be complied with by medical personnel using or performing maintenance on medical devices that are contaminated or potentially contaminated.

Pointed and sharp instruments should be handled with great caution.

Check that the steriliser and the water that is used are clean. After each sterilisation cycle, remove the device from the sterilising apparatus immediately, in order to reduce the risk of corrosion.

Suitable detergent

For manual cleaning-disinfection:

- Detergent or detergent-disinfectant (pH 6-9.5) recommended for cleaning-disinfection of dental or surgical instruments. Quaternary ammonium- and/or enzyme-based surfactants.
- Do not use detergents that are corrosive or contain chlorine, ace-

tone aldehydes or bleaches.

- Only use original Bien-Air Dental maintenance products and parts or those recommended by Bien-Air Dental. Using other products or parts may cause operational failure and/or void the guarantee.

❶ Manual cleaning/ disinfection only

⚠ Attention :

Electric motors with brushes are not suitable for automatic cleaning/disinfection (in a washer/disinfector machine).

Clean and disinfect using a clean cloth soaked in a suitable product.

- Never immerse in disinfectant solutions.
- Do not soak in physiological liquid (NaCl).
- Do not immerse in an ultrasonic bath.

❷ Sleeve sterilisation

⚠ Only for motors with removable sleeve fig. 6.

Conditioning process

Wrap the removable sleeve in approved packaging for steam sterilisation.

Important

The quality of the sterilisation depend very much on the cleanliness of the device. Only perfectly clean devices may be sterilised.

Do not use a sterilisation procedure other than the one described below.

Procedure: Fractionated pre-vacuum steam sterilisation, Class B cycle acc. to EN13060. The procedure has been validated according to ISO 17664. All Bien-Air Dental straight handpieces are sterilisable in an autoclave up to 135°C (273.2°F). Duration: 3 or 18 min., depending to the national requirements in force.

Overhaul

Never disassemble the device. For any modification and repair, we recommend that you contact your regular supplier or Bien-Air Dental directly. Bien-Air Dental asks the user to have its dynamic instruments checked or inspected at least once a year.

Transport and storage conditions

Temperature between -40°C (-40°F) and 70°C (158°F), relative humidity between 10% and 100%, atmospheric pressure 50 kPa to 106 kPa (7.3 to 15.3 psi).

Other precautions for use

The device must be used by a qualified professional in compliance with the current legal provisions concerning workplace safety, health and accident prevention measures, and these working instructions. In accordance with these requirements, the operators:

- must only use operating devices that are in perfect working order; in the event of irregular functioning, excessive vibration, abnormal heating or other signs indicating malfunction of the device, the work must be stopped immediately; in this case, contact a repair centre that is approved by Bien-Air Dental;
- must ensure that the device is used only for the purpose for which it is intended, must protect themselves, their patients and

third parties from any danger, and must avoid contamination through the use of the product.

Guarantee **Terms of guarantee**

Bien-Air Dental grants the operator a guarantee covering all functional defects, material or production faults. The device is covered by this guarantee for 18 months from the date of invoicing. Bien-Air Dental offers a 24-month guarantee for the glass-bar light conductors.

In the event of justified claim, Bien-Air Dental or its authorised representative will fulfil the company's obligations under this guarantee by repairing or replacing the product free of charge. Any other claims, of whatever nature, in particular in the

form of a claim for damages and interest, are excluded.

Bien-Air Dental shall not be held responsible for damage or injury and the consequences thereof, resulting from:

- excessive wear and tear
- improper use
- non-observance of the instructions for installation, operation and maintenance
- unusual chemical, electrical or electrolytic influences
- poor connections, whether of the air, water or electricity supply.

The guarantee does not cover flexible "fibre optic" type conductors,

or any parts made of synthetic materials.

The guarantee shall become null and void if the damage and its consequences are due to improper manipulation of the product, or modifications to the product carried out by persons not authorised by Bien-Air Dental.

Claims under the terms of the guarantee will be considered only on presentation, together with the product, of the invoice or the consignment note, on which the date of purchase, the product reference and the Serial No. should be clearly indicated.

Designation micromotor	MC2 IR	MC2 GTAV	MC2 Isolite 300	MC2 Isolite 100	MC2 LED
REF micromotor	1600073-001	1600075-001	1600078-001	1600079-001	1600681-001
Designation hose	REF hose				
Grey hose ECO with fixed connector	1600120-001	x	(x)	x	x
Hose Technobox	1600136-001		x		x
Black hose ECO with fixed connector	1600297-001	x	(x)	x	x
Grey hose, revolving connector	1600298-001		x	x	x
Black hose, revolving connector	1600299-001		x	x	x
White hose ECO, with fixed connector	1600314-001	x	(x)	x	x
Black coiled hose with fixed connector	1600315-001	x	(x)	x	x

(X) Do not connect supply of water and air spray (risk to damage the micromotor)

REF	Legend
-----	--------

1600681-001	MC2 LED micromotor, with internal spray and light LED
1600073-001	MC2 IR micromotor, with internal spray, without light
1600075-001	Micromotor MC2 GTAV self-cooling, without internal spray, without light
1600078-001	Isolite 300 micromotor, with internal spray and light
1600079-001	Isolite 100 micromotor, with internal spray, without light
1302447-001	Sleeve for MC2 Isolite 100
1500246-001	Sleeve for MC2 Isolite 300
3300168-001/007.22.05-001	Extraction tongue for the brush springs
1500241-010	Pack of 10 brushes for MC2 and Isolite micromotors
1500007-005	Pack of 5 bulbs, for micromotors
1300145-010	O-Ring
1301646-010	O-ring for GTAV
1300992-010	O-ring for Isolite
1600307-001	Fow-meter for MC2 micromotors

a Set supplied

See cover page.

b Optional accessories

See cover page.

List of Bien-Air Dental SA registered trade mark products ®:

Aquilon®	Eolia®	Lubrifiuid®	Prestilina®
Bora®	Gyro®	Lubrimed®	Spraynet®
Boralina®	Gyrolina®	MX®	
ChiroPro®	Isolite®	PowerCare®	

In these instructions, “Device” corresponds to the product described in the heading “Identification”. For example, turbine, contra-angle, handpiece, micromotor, tube, electronics, connectors, station etc.

Symbols

Manufacturer.

CE Marking with number of the notified body.

Attention.

Recyclable electrical and electronic material.

Wear rubber gloves.

Sterilisable at the specified temperature.

Move in the direction indicated.

Light.

Move fully to the stop, in the direction indicated.

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EMC Precautions

Medical electrical equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the user's manual. Essential performance was defined as the maintaining of the visual lightning intensity when powered at 3.5VDC.

1. Use of MC2

⚠ Attention : The MC2 should not be used adjacent or stacked with other equipment. If adjacent or stacked use is necessary, the MC2 should be observed to verify normal operation in the configuration in which it will be used.

2. Use of Accessoires

⚠ Attention : The use of accessories, transducers and cables other than those specified, with the exception of transducers and cables sold by Bien-Air as replacements parts for internal components, may result in increased emissions or decreased immunity of the MC2.

3. Radio Transmitting Equipment

⚠ Attention : The MC2 motors comply with the EMC requirements according to IEC 60601-1-2. Radio transmitting equipment, cellular phones, etc. shall not be used in the close proximity of the device since this could influence the performance of the device. Particular precaution must be considered during use of strong emission sources such as High Frequency surgical equipment and similar so that e.g. the HF cables are not routed on or near the device. If in doubt, please contact a qualified technician or Bien-Air Dental.

Guidance and manufacturer's declaration – electromagnetic emissions

The MC2 motors are intended for use in the electromagnetic environment specified below. The customer or the user of the MC2 motor should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The MC2 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The MC2 is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations/flicker emissions IEC 61000-3-3	Not applicable	

Guidance and manufacturer's declaration – electromagnetic immunity

The MC2 motors are intended for use in the electromagnetic environment specified below. The customer or the user of the MC2 motor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±2 kV contact ±4 kV contact ±6 kV contact ±2 kV air ±4 kV air ±8 kV air	±2 kV contact ±4 kV contact ±6 kV contact ±2 kV air ±4 kV air ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	Not applicable	Not applicable
Surge IEC 61000-4-5	±0.5 kV line to line ±1 kV line to line ±0.5 kV line to earth ±1 kV line to earth ±2 kV line to earth	Not applicable	Not applicable
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % <i>UT</i> (>95 % dip in <i>UT</i>) for 0,5 cycle 40 % <i>UT</i> (60 % dip in <i>UT</i>) for 5 cycles 70 % <i>UT</i> (30 % dip in <i>UT</i>) for 25 cycles <5 % <i>UT</i> (>95 % dip in <i>UT</i>) for 5 sec	Not applicable	Not applicable
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical industrial location or hospital environment.

NOTE *UT* is the a.c. mains voltage prior to application of the test level.

Guidance and manufacturer's declaration – electromagnetic immunity

The MC2 motors are intended for use in the electromagnetic environment specified below. The customer or the user of the MC2 motors should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	10 V	Portable and mobile RF communications equipment should be used no closer to any part of the MC2, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter Recommended separation distance $d = \left[\frac{3,5}{V_1} \right] \sqrt{P}$ $d = \left[\frac{3,5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[\frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	10 V/m	where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range. ^b Interference may occur in the vicinity of equipment marked with the following symbol: 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the MC2 is used exceeds the applicable RF compliance level above, the motor MC2 should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the MC2 motor.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the MC2

The MC2 motors are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the MC2 motors can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the MC2 as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of the transmitter	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$	800 MHz to 2.5 GHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$
W			
0.01	0.035	0.035	0.07
0.1	0.11	0.11	0.22
1	0.35	0.35	0.7
10	1.1	1.1	2.2
100	3.5	3.5	7

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.