SOPIRA® Carpulse®

Sterile Single-Use Dental Needles

- DE

Gebruichsanweisung
- GB

Instructions for use
- FR

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Instrucciones de uso
- IT

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Bruksanvisning
- DK

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Bruksanvisning
- FI

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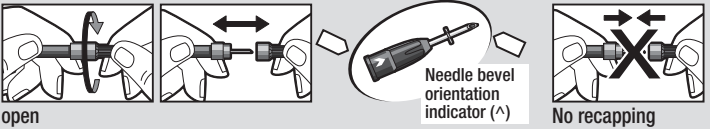
KULZER
MITSUBI CHEMICALS GROUP

Manufacturer:
Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)

9900087/11

SOPIRA® Carpulse®

Instructions for use

GB

● 100 sterile, individually packed disposable dental needles for single use

- Silicone coated for easier penetration and sterile.
- Plastic hub with thread and needle bevel orientation indicator for optimum injection direction and reduced insertion pain.
- Thin-wall SOPIRA Carpulse Free Flow cannula for low injection pressure.

Use

1. Protect needles against direct sunlight, high temperature and moisture.
2. Only use needles if the seal on the packaging is intact.
3. Needles may be used with all commercial syringes and syringe systems in common use in dental anaesthesia. Please choose appropriate length and thickness of needles for specific indication in dental local anaesthesia.
4. Before use, unscrew the short, transparent protective cap (removing the long protective cap just before injection).
5. Insert the short needle into the syringe and screw onto the thread of the syringe with the long protective cap on. The needle is properly attached to the syringe when the two parts of the thread (on the syringe and needle) fit together flush (i.e. with no protrusions).
6. Only remove the long protective cap just before injection.
7. Do not bend the needles.
8. Each needle may only be used for one single patient.
9. Avoid all contact with the used needle (risk of infection, do not manually recap!). To avoid cross infection, items intended for single use must be disposed of immediately after initial use.
10. Dispose of needles in special safe containers (puncture-resistant/lockable), which are available commercially.
11. Needles should be collected and protected against human contact (i.e. in a suitable protective container) and then disposed of in accordance with valid regulations.
12. Only for professional use.

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SOPIRA Carpulse Needles					
Available with metric thread	Indication	Available as Free-Flow	Available as Blister	Hub colour (indicating diameter)	Label colour (indicating length) *
0.4 x 08 mm 27 G	Intraseptalis			grey	violet
0.4 x 08 mm 27 G short bevel	Intraligamentalis		Blistered only	grey	violet
0.4 x 25 mm 27 G	Infiltration	✓	✓	grey	orange
0.4 x 35 mm 27 G	Mandibularis		Blistered only	grey	red
0.4 x 38 mm 27 G	Mandibularis	✓		grey	dark red
0.3 x 12 mm 30 G short bevel	Intraligamentalis	✓	✓	yellow	blue
0.3 x 12 mm 30 G	Infiltration			yellow	blue
0.3 x 16 mm 30 G short bevel	Intraligamentalis	✓	✓	yellow	green
0.3 x 16 mm 30 G	Infiltration		✓	yellow	green
0.3 x 21 mm 30 G	Infiltration		✓	yellow	yellow
0.3 x 25 mm 30 G	Infiltration	✓	✓	yellow	orange
Available with inch thread					
0.4 x 35 mm 27 G	Mandibularis			grey	red
0.3 x 12 mm 30 G short bevel	Intraligamentalis			yellow	blue
0.3 x 16 mm 30 G short bevel	Intraligamentalis			yellow	green
0.3 x 21 mm 30 G	Infiltration			yellow	yellow

* In addition to the hub colour indicating the diameter, the coloured paper label is used to code the length of the needles. Please find further information on www.kulzer-dental.com

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