MM-MTA™ – MM-MTA™ Caps
Ciment de réparation endodontique

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V. APPLICATION**5.1. Repair of root perforations**

Place rubber dam and clean the root canal system using intracanal instruments and irrigate with NaOCl. Dry the root canal with paper points and isolate the perforation.

Fill the canal space from the apical part up to the perforation.

Mix MM-MTA™ as described under point 4 and extrude it on a clean glass plate.

Apply MM-MTA™ with suitable instruments into the perforation site and condense it.

Check the position of MM-MTA™ in the root canal by an X-ray. If an adequate barrier has not been created, rinse MM-MTA™ out of the canal and repeat the procedure.

After 20 minutes, obturate the remaining canal space. MM-MTA™ repair material remains as a permanent part of the root canal filling.

If the treatment is made in 2 sessions, take a damp cotton pellet, remove excess moisture from the pellet and place it in the canal. Seal the access preparation with a temporary restoration.

5.2. Root-End Fillings

Create an access to the root-end and resect the root with a surgical bur.

Use an ultrasonic tip to prepare a class I root-end cavity preparation to a depth of 3-5 mm.

Isolate the area and dry the root end cavity with paper points.

Mix MM-MTA™ as described under point 4 and extrude it on a clean glass plate.

Apply MM-MTA™ with suitable instruments and condense it using a small plunger.

Remove excess cement and clean the surface of the root with a moist piece of gauze.

Confirm placement of the MM-MTA™ repair material with an X-ray. The MM-MTA™ repair material remains as a permanent part of the root canal filling.

5.3. Pulp Capping

Place rubber dam and prepare the access cavity. If caries is present, remove it. Rinse cavity and exposed pulpal areas with 3-5% NaOCl.

Mix MM-MTA™ as described under point 4 and extrude it on a clean glass plate.

With a suitable instrument apply a small amount of MM-MTA™ over the exposed pulp and remove excess moisture with a dry cotton pellet.

Place a small amount of a light cure liner and light cure.

Etch the remaining cavity walls according to the total-etch and apply a suitable bonding agent according to the corresponding instructions.

Place a light cure composite according to the instructions and light cure.

Pulp vitality and status should be checked by X-ray at regular intervals.

VI. PRECAUTIONS FOR USE

• Store MM-MTA™ in the sealed packaging at a dry place prior to use.

• Intraoral application of MM-MTA™ must be done immediately after mixing to prevent dehydration during setting.

• MM-MTA™ can cause discoloration. Use MM-MTA™ only in the root canal and/or the pulp chamber.

VII. STORAGE

Store MM-MTA™ at a dry place at 10 – 25°C. **Do not store below 10°C!**

Do not use after expiry date.

MM-MTA™ Caps are for single use only.

VIII. WARRANTY

MICRO-MEGA® warrants this product will be free from defects in material and manufacture. MICRO-MEGA® makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and MICRO-MEGA®'s sole obligation shall be repair or replacement of the MICRO-MEGA® product.

IX. LIMITATION OF LIABILITY

Except where prohibited by law, MICRO-MEGA® will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Attention:

Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting which may impair or prevent application of the material. The material must be extruded within 10 seconds after the end of mix. To prevent dehydration during setting, apply MM-MTA™ intraorally immediately after mixing.

Working time of MM-MTA™ is approx. 2 minutes (at 23°C).

Setting time of MM-MTA™ is about 20 minutes.

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For activation of the MM-MTA™ Cap press the plunger on a hard and plane surface to the end into the MM-MTA Cap.

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Insert the MM-MTA™ Cap into the MM-MTA™ Gun. Pull the lever 2 times (2 clicks) to prime the MM-MTA™ Cap.

Extrude the mixed material on a glass plate. Unlock the gun (push button A) and remove the MM-MTA™ Cap.

Only with the MM-MTA™ Gun the optimal amount of mixed material is guaranteed.

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