

CE0473


## Reliance Bonding Resins WITH OR WITHOUT FLUORIDE BONDING SYSTEM FOR DIRECT AND INDIRECT BONDING

V3 3/16

**INDICATIONS FOR USE:** Reliance Bonding Resins are intended for use in direct and indirect bonding of orthodontic brackets and appliances.

**INSTRUCTIONS FOR USE:**

1. Prepare the tooth surface (Prophy, etch, rinse, dry)
2. Mixing: Dispense equal amounts of Bonding Resin Part A&B onto a mixing pad. Using a brush or sponge pellet, mix the two resins thoroughly (5-10 seconds).
3. Application: Apply a thin coat of mixture onto the etched surface. The working time of the Bonding Resin is 70 seconds when measured from the start of mixing. It is not necessary to wait for the bonding resin to set before proceeding to the application of the adhesive.

 **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.



**TEMPERATURE LIMITATION:** Store at room temperature (15-30 °C / Not to exceed 86 degrees° F) or can be refrigerated (36-46 degrees° F).

**TO REORDER:** Ref# S7AF – 7 gm Bonding Resin A w/Fluoride, Ref# S7B – 7gm Bonding Resin B, Ref# S20AF – 20 gm Bonding Resin A w/Fluoride, Ref# S20B – 20 gm Bonding Resin B


**WARRANTY:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**Rx Only:** Federal law restricts this device to sale by or on the order of a dental professional.



*Reliance Orthodontic Products, Inc.*

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 · [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)

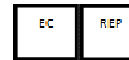

EMERGO EUROPE  
Molenstraat 15  
2513 BH, The Hague  
The Netherlands



Australian Sponsor:  
Emergo Australia  
201 Sussex St. Darling Park  
Tower II, Level 20,  
Sydney, NSW 2000 Australia

Scan for more information