Liquid Strip

English

Instructions for Use
Gebrauchsinformation
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções de Uso
Bruksanvisning
Brugsanvisning
Käyttöohjeet
Bruksanvisning
Productinformatie
Οδηγίες Χρήσεως

- Water-soluble glycerine paste
- Wasserlösliche Paste auf Glycerinbasis
- Pâte soluble dans l'eau, à base de glycérine
- Pasta su base di glicerina idrosolubile
- Pasta en base a glicerina soluble en agua
- Pasta de glicerina solúvel em água
- Vattenlöslig glycerinpasta
- Vandopløselig glycerinpasta
- Vesiliukoinen glyseriinipasta
- Vannløselig pasta på glyserolbasis
- Wateroplosbare pasta op basis van glycerine
- Υδατοδιαλυτή πάστα γλυκερίνης

For dental use only!
Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

557681/0503/4/WE3/G

ivoclar
vivadent
clinical

Instructions for Use

Description

Glycerine gel impermeable to oxygen, as an aid for the try-in of composite and ceramic restorations (try-in paste) and for the processing of composites.

Composition

Glycerine gel

Indication

1. Try-in paste: Try-in for composite and ceramic restorations

The glycerine gel can be used to fill the cement gap during the try-in and thus simulate the influence of the bonding composite on shading. This permits realistic evaluation of the restoration's shading. For an even more accurate simulation of the Variolink®II luting composite system, shaded glycerine pastes (Variolink II Try-In pastes) are also available.

2. Prevention of oxygen inhibition during polymerization

Composites are subject to oxygen inhibition, i.e. the surface layer of the composite that is not covered with a matrix and comes in contact with oxygen during polymerization is not polymerized (approx. 50 µm). If this layer is covered with Liquid Strip and thus protected from oxygen, the composite is thoroughly polymerized.

Contraindication

None known to date

Side Effects

None known to date

Application

1. Try-in of composite and ceramic restorations

- Application of Liquid Strip on cavity or restoration
- Try-in, evaluation of shade and fit
- Removal of the restoration from the cavity
- Rinsing off Liquid Strip with water and drying with blower
- Cleansing the restoration with 37 % phosphoric acid (e.g. Total Etch)
- Thoroughly rinsing off phosphoric acid and drying with blower
- Further procedure according to information of the luting composite manufacturer

2. Prevention of oxygen inhibition with composites

a) Inlay/Onlay luting

- Placement of ceramic or composite restoration with a luting composite (e.g. Variolink II)
- Removal of excess composite
- Application of Liquid Strip on restoration margins
- Light-curing of the luting composite through Liquid Strip
- Rinsing off Liquid Strip with water
- Finishing/polishing of the restoration

b) Freely built-up composite restorations

- Build-up of restoration
- Covering restoration surface with Liquid Strip
- Light-curing of the composite through Liquid Strip
- Rinsing off Liquid Strip with water
- Finishing/polishing of the restoration

Note

With self-curing composites, glycerine gel has to be left on restoration until end of setting time.

Important

If a low-viscosity luting composite is used, Liquid Strip has to be applied carefully, to avoid mixing with, or extruding of, the composite.

Storage

- Do not use Liquid Strip after expiration date
- Storage conditions: room temperature (2–28 °C / 54–82 °F)
- Shelf life: see expiration date

Keep out of the reach of children!

For use in dentistry only!

Date information prepared

05/2003

Manufacturer

Ivoclar Vivadent AG

FL-9494 Schaan / Liechtenstein

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the material for its suitability and use for any purpose not explicitly stated in the Instructions.

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Ivoclar Vivadent Ltd
Ivoclar Vivadent S.A.
Ivoclar Vivadent AB
Ivoclar Vivadent, Inc.