Prior to use, carefully read the instructions for use.

EN

## COCOA BUTTER

PROTECTIVE COATING

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

- For the prevention of water and saliva contamination to exposed surfaces of glass ionomer cements during the first 24hours after placement.
- To prevent cement from sticking to adjacent teeth and instruments.

### CONTRAINDICATIONS

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

### HOW TO USE

- Place glass ionomer cement according to the manufacturer’s instructions.
- Extrude the required amount of material from the tube. Using a spatula or cotton pellet, apply uniformly to the area to be moisture proofed / isolated.
- After use, immediately replace cap.

### STORAGE

Store at temperature of 4-25°C (39.2-77.0°F).  
(Shelf life: 5 years from date of manufacture)

### PACKAGE

Tube of 10g

### CAUTION

In case of contact with eyes, flush immediately with water and seek medical attention.

Last revised : 08/2014

**GC**

MANUFACTURED by  
GC DENTAL (SUZHOU) CO., LTD.  
Qing Qiu Street, Suzhou Industrial Park, Suzhou, 215126 P. R. China

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

 EU: GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.  
11 Tampines Concourse, #03-05 Singapore 528729  
TEL: +65 6546 7588

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