Prior to use, carefully read the instructions for use.

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ES

## BITE COMPOUND

IMPRESSION COMPOUND FOR BITE REGISTRATION

For use only by a dental professional in the recommended indications.

### RECOMMENDED INDICATIONS

Checking occlusal registrations.

### DIRECTIONS FOR USE

Soften over a flame or in a warm water bath and adapt in this plastic state. The right moment to take the impression is when BITE COMPOUND does not show any more threads when pulled apart, but can still easily be formed. Note : Do not use on patients whilst too hot (over 45°C).

### STORAGE

Recommended for optimal performance, store at normal room temperature (4-25°C) (39.2-77.0°F). (Shelf life : 3 years from date of manufacture)

### PACKAGE

Standard package : 120 g (15 sticks)

### CAUTION

- In case of contact with eyes, remove and flush with water.
- In rare cases the product may cause sensitivity to some persons. If such reactions are experienced, discontinue the use of the product and refer to a physician.
- Personal protective equipment (PPE) such as gloves, face masks and safety eyewear should always be worn.

Some products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the safety data sheets available at:

<http://www.gceurope.com>

or for The Americas

<http://www.gcamerica.com>

They can also be obtained from your supplier.

Last revised : 07/2017

Rx Only

CE

GC

DISTRIBUTED by  
GC CORPORATION  
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.  
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,  
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.  
TEL: +1-708-597-0900  
[www.gcamerica.com](http://www.gcamerica.com)

GC ASIA DENTAL PTE. LTD.  
11 Tampines Concourse, #03-05 Singapore 528729  
TEL: +65 6546 7598

MANUFACTURED by  
NIMI CHEMICAL INDUSTRIAL CO., LTD.  
4-13 Yokoyama-cho, Kiryu, Gunma 376-0055, Japan

RESPONSIBLE MANUFACTURER IN CANADA:  
GC AMERICA INC.  
3737 West 127th Street, Alsip, IL 60803 U.S.A.

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