

Now with
new cannula



Calcicur®

READY-TO-USE CALCIUM HYDROXIDE PASTE

Calcicur® – the tried and tested calcium hydroxide paste is now even more practical in its application

Calcium hydroxide products play an important role in the day-to-day work of the dental practice. The highest product quality and an application-friendly consistency are indispensable properties for application near the pulp or in the root canal. With Calcicur, VOCO offers a product which fulfils all of the requirements placed on the material – and, with the new cannula, has now ensured even easier handling, too. The market leader* Calcicur, with its familiar creamy consistency, is now available in a new syringe with an optimised cannula.

Calcicur is the ideal material for direct and indirect capping, for temporary root canal fillings and for lining cavities as a protective measure against exposure to acid from cement materials. Dentists who have been using Calcicur in their practices for some time were impressed when introduced to the new cannula. Its high pH value of > 12.5 gives Calcicur an antimicrobial effect, and around 90 % of users rated the consistency of the calcium hydroxide paste as “good”.

With its slim, pliable design and high flexibility, the new cannula is ideal for introducing Calcicur into hard-to-reach cavities as well as root canals. The new, transparent attachment is 20 mm long and has a diameter of 0.6 mm at its tip, which makes for a far more comfortable application. On the strength of the new cannula, well over half of the users would change their application method, applying the material directly from the syringe and only using a Lentulo to move it further towards the tip of



the root. This noticeably simplifies the application of a calcium hydroxide preparation.

For almost all dental professionals using the cannula it has already proven itself in their day-to-day practice since it “saves time and effort” and facilitates “fast, clean working”. With the aid of the lateral wings, the new cannula can easily be locked into place on the new 2.5 g syringe using a Luer lock. What is more, the ease with which the material can be expressed from the syringe is perceived as positive in every respect.

Conclusion: Thanks to the new application cannula, the tried and tested Calcicur now guarantees even simpler working – helping day-to-day practice run effectively.

Source: VOCO in-house test, 2012

* in the calcium hydroxide product syringes segment, according to the market research company GfK, Germany 2011.

