Addition duplicating silicone

Instructions for use

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EN Instructions for use.

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DE

ADDITIONSVERNETZENDES DUBLIERSILIKON

VERWENDUNG

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ADDITION DUPLICATING SILICONE

USE

High precision poly-addition silicone rubber used in dental laboratory models. It has an excellent compatibility with phosphate coatings, polyurethane resins and plasters. It is biocompatible and non-toxic.

INSTRUCTIONS

Make sure that the model to duplicate is perfectly clean and dry. Before using, shake well the bottles of part A and B. Weigh the same quantity of part A and B. Mix for a minute until a homogeneous colour is obtained, then pour the mixture on the model to duplicate. Strain from 20/30 cm in height at the side of the model to remove any air bubbles. After about 30' demould the model using a jet of compressed air. It is recommended to work at 23°C.

WARNINGS

Some inhibitions to vulcanization process may occur when the silicone rubber gets in contact with amines, plasticines, metal salts, sulphur, tin catalyst and poly-condensation silicone rubbers.

STORAGE

It is suggested to keep the product in their original packaging, well-closed at a temperature between +7°C and +27°C, in well-aired places. When stored under appropriate conditions, it is stable and usable for 12 months. Do not reverse the cap.

TECHNICAL DATA

Hardness (24h)	16 shore	22 shore	30 shore
Mix ratio	100:100	100:100	100:100
Colour	light blue	dark blue	green
Viscosity	3000	5000	5000
Specific gravity	1.09	1.14	1.16
Working time	5-10'	5-10'	5-10'
Setting time	30'	30'	30'
Detail reproduction	20µm	20µm	20µm

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Proclinc, S.A.