4DESIGN 4DISKS

DIGITAL PRECISION SOLUTIONS PMMA

DE PMMA-Disc. Gebrauchsanweisung.

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Acrylat Kunststoff Rohling.

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EN PMMA disk. Instructions for use.

Intended use:

Acrylic polymer disk particularly suitable for making removable or temporary dental structures such as crowns and bridges using milling technology using CAD/CAM.

Indications:

- Temporary anterior and posterior crowns
- Temporary anterior and posterior bridges with up to two adjacent pontics
- Implant supported temporary restorations

Maximum recommended usage without checkup by dentist: 12 months

- Removable structures for dentures (dental bases)
- Removable structures for therapeutic restorations (bite splints or occlusal splints)

Contraindications:

- Use for fixed permanent restorations
- Use for implant supported permanent restorations
- Bridge designs involving more than two adjacent pontics
- Patients suffering from parafunctional habits such as bruxism
- Restorations should not be made with Iodentine in case of a known allergic reaction to any of its chemical constituents.

Instructions:

- Can be used with any regular system of milling CAD/CAM (disc 98.5 mm diameter).

- Fix the disk in the milling machine as usual. Consider the composition of colors to properly allocate the enamel side of the blank.

To process, tungsten carbide burs for metal can be used. Milled structures can be separated using burs or cutting discs for resin.

Finishing and polishing:

The structure should be finished and polished in the usual manner for PMMA dental material.

Veneering:

After milling, 4DESIGN can be faced using conventional veneering materials. MMA based materials are preferred. When veneering composite, Iodentine must be bonded following the corresponding composite's instructions for use.

Specifications on design:

The dental structures for restorations are designed in CAD Software.

- A 5-Axis milling machine is required, to reach undercut areas.
- Minimum wall thickness: 1,5mm for occlusal faces and 0,9 mm for peripheral sides.
- A maximum of one unsupported end unit (cantilever unit) is permitted. A maximum of 6 units in multi-unit bridges for temporary fixed restorations is permitted.

- In posterior region, no more than one pontic may be positioned between two abutment crowns.

Composition:

Polymer based on polymethyl methacrylate (PMMA) > 99 % wt. / Pigments < 1 % wt.

PRECAUTIONS

- Consider the physical properties of the material to use.
- Dental bridges in posterior region: Not more than two interdental connectors in acrylic resin.
- During polishing avoid breathing in dust.

Storage conditions: No special measures required.

Disposal: The remains of the block processing and its packaging are not considered hazardous waste (plastic and cardboard) and must be disposed of according to local or state legislation.

Side effects:

In individual case, rash, sensitivity or allergies may occur by using the PMMA dental prosthesis, please stop using and consult the physician.

Interactions:

Potential interactions with other products used in the treatment are very unlikely because the product is a totally polymerized material based on PMMA. In order to ensure that the selected restoration can be placed safely and durably, the dentist should consider known interactions and cross reactions of the product with other materials already present in the oral environment of each patient before using the product.

Re-use:

The device can be reused as long as there is space available to mill the structures.

Clean the device after each use and store it properly identified in its box. **Caution:** Reuse dentures and other used milled dental structures in other persons can infect diseases.

INFORMATION FOR THE PATIENT:

The patient should be informed of the following points:

- a) Normal oral hygiene must be maintained. Direct the patient to adopt daily hygiene protocols*, suggest him to avoid excessive brushing which could cause abrasive wear, and in case of removable prosthesis not to immerse it in alkalis, acids or other substances than can damage acrylic materials.
- b) The material is not visible at x ray

- c) Material is resistant to abrasion but heavy bites should be avoided.

- d) In case of an allergic reaction, seek medical advice.

* **Cleaning protocol:** Patients should clean dental prosthesis at least daily. Brush them with a softbristled brush and nonabrasive denture cleanser to remove food, plaque and other deposits.

- Disclaimer: Material for dental use. The manufacturer is not liable for any damage resulting from use other than or not in accordance with the instructions for use. Any serious incident involving the device must be reported to the manufacturer and to the competent authority of the Member State in which the incident occurred.

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Unión Dental, S.A. unidesa-odi