Face mask

ADJUSTABLE

Instructions for use

- DE **Gebrauchsanweisung.**
- ES **Instrucciones de uso.**
- FR **Mode d'emploi.**
- IT **Istruzioni per l'uso.**
- PT **Instruções de uso.**

INSTRUCTIONS FOR USE EN

FULLY-ADJUSTABLE FACEMASK Orthodontic Class III Malocclusion Treatment

FACEMASK USE

Always use the facemask according to your medical practitioner's instructions. Be sure to keep your mouth and the facemask clean. Changing the protection pads is recommended as they become dirty over time. Do not use the facemask if its structure has altered due to an accident. The facemask cannot be used while playing sport. Advise your medical practitioner immediately if any irritation appears or if you feel discomfort.

DECLARATION OF COMPLIANCE - European Commission

ORTHOCOMFORT states that the product known as FULLY ADJUSTABLE FACEMASK complies with the requirements defined in Directive 93/42/CEE and its revisions. As such, it also complies with the Spanish legislation in the RD 1591/2009, relating to sanitary products. Throughout the manufacture process, aspects relating to the quality of the product are specified and controlled in order to ensure compliance to the defined specifications.

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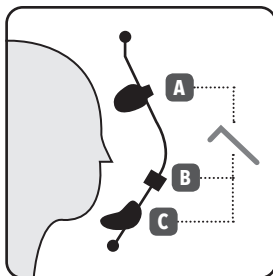
EN In order to adjust the mask to the patient's face, the Chin cup **C**, Forehead support **A** and Crossbar **B** can be moved vertically up or down using the hexagon key provided in the pack.

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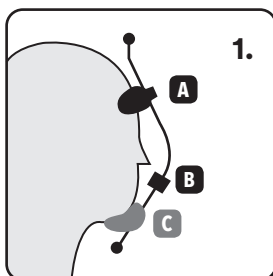
EN Center the chin cup **C** over the patient's chin and tighten the screw using the hexagon key.

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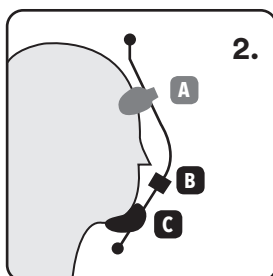
EN Adjust the forehead support **A** placing it slightly above the patient's eyebrows and tighten hexagonal screw.

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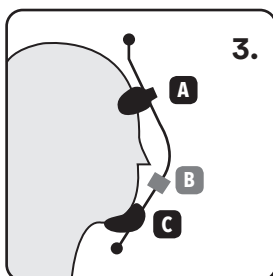
EN Adjust position of the crossbar **B** sliding it vertically to the desired position and tighten the screw using the hexagon key.

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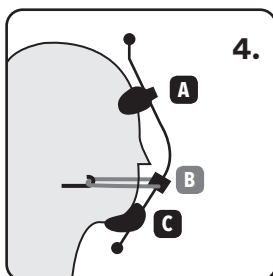
EN Attach the elastics either on brackets or wire hooks and stretch forward to place on the crossbar **B**.

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