

CE0473

Light Bond™ Sealants

V5 3/16


With or Without FLUORIDE


LIGHT CURE SEALANT RESIN

INDICATIONS FOR USE: Light Bond™ sealants are intended for use as a light cure sealant resin for bonding under an orthodontic adhesive.

INSTRUCTIONS FOR USE:

Dispense one to two drops of Light Bond sealant resin onto a mixing pad. With a brush, apply a thin, uniform layer of sealant to the etched enamel. It is not necessary to cure the sealant at this time, however, a cured sealant can prevent bracket floatation. To further enhance bond strength with some fine mesh brackets, a thin coat of sealant can be painted on the metal bracket base. Cure the sealant for 10 seconds before applying adhesive to the bracket base.

 **PRECAUTIONS:** Unpolymerized resins may cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water.

 **TEMPERATURE LIMITATION:** Store at room temperature (15-30 °C / Not to exceed 86 ° F°) or can be refrigerated (2-8 °C/ 36-46° F°).

TO REORDER: Ref# LBS3F – 3cc Light Bond Sealant w/Fluoride, LBS7F – 7cc Light Bond Sealant w/Fluoride, LBS3 – Non Fluoride, 3cc Light Bond Sealant, LBS7 – Non-Fluoride 7cc Light Bond Sealant, LBFS7F – Light Bond Filled Sealant 7cc w/Fluoride, LBFSF – Light Bond Filled Sealant 3cc w/ Fluoride

WARRANTY: Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

Rx Only: Federal law restricts this device to sale by or on the order of a dental professional.



Reliance Orthodontic Products, Inc.

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.
1-800-323-4348 · www.RelianceOrthodontics.com



Australian Sponsor:
Emergo Australia
Level 20, Tower II, Darling Park
201 Sussex St.
Sydney, NSW 2000 Australia



EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands



Scan for more information