

SR PHONARES® II

TOOTH MOULD ARTISTRY



TOOTH MOULD CHART

- SR PHONARES® II
- SR PHONARES® II *Typ*
- SR PHONARES® II *Lingual*




SELECTION CRITERIA

ANTERIOR TEETH

A range of tooth moulds designed to match the age and characteristics of the individual patient

A straightforward selection procedure

The tooth moulds are arranged into easy-to-identify groups according to an innovative new concept. The teeth that most accurately match the individual characteristics of the patient can be easily selected.

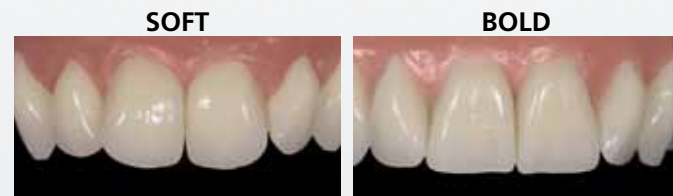

- Unabraded incisal edge
- Pronounced facial curvature

- Slightly abraded incisal edge
- Reduced facial curvature

- Heavily abraded incisal edge
- Flat facial curvature

In addition to the shade, the shape (form) and size of anterior teeth have to be matched to the individual characteristics of the patient. The SR PHONARES II line of teeth offers 18 upper and 6 lower tooth moulds for the anterior region.

If possible, the anterior teeth should be selected directly on the patient according to the patient's anatomical and facial characteristics. The following methods are recommended to facilitate the selection procedure:

CHAIRSIDE

LABSIDE

FormSelector

The SR PHONARES II FormSelector allows fast, easy and targeted selection of the appropriate anterior tooth mould.

- Step 1** Determine the interalar width of the nose with the help of the FacialMeter. Select an appropriately sized tooth mould from the INTER-ALA table.
- Step 2** After having identified the correct size category, select the desired tooth form, soft or bold, in line with the shape of the patient's face.
- Step 3** Select the appropriate age group of the teeth according to the incisal and facial curvature of the SR PHONARES II teeth.

Model analysis

Determine the following reference points with the help of a model analysis:

a) Position of canines

Identify the first large pair of rugae; the midpoint of the labial surface of the upper canines is aligned at a distance of 9 mm.

b) Intersection point of central incisors

The labial surface of the central incisors is aligned at a distance of 7 mm to the centre of the incisive papilla.

The length of the dental arch measured across these three points can be used as a reference to select a set of anterior teeth with an appropriate width from the tooth mould chart.

SELECTION CRITERIA POSTERIOR TEETH

The SR PHONARES® II product family incorporates two posterior tooth lines

SR PHONARES® II *Typ*

This tooth mould is suitable for a wide range of applications and is therefore the classic mould for partial, complete and hybrid dentures.

Conventional occlusion




One-to-two setup



SR PHONARES® II *Lingual*

This tooth line is based on a lingualized occlusion concept. In addition to universal applications, these teeth are therefore particularly suitable for implant-borne removable restorations as they offer an ideal static alignment.

Lingualized occlusion

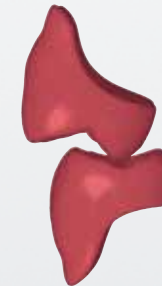

One-to-one setup



or

One-to-two setup



The following requirements should be met to ensure that dentures function appropriately in the oral cavity in the long term:


- Fabrication in the dental laboratory according to the principles of best practice
- Application of dental materials according to the manufacturer's instructions. In particular, denture teeth should not be excessively weakened as a result of adjustments by grinding.
- Planning and fabrication of the dentures should be performed in close collaboration with the dentist.
- Dental technician and dentist should ensure a balanced occlusion and articulation.

Hands-on BPS training courses offer users an opportunity to learn the relevant basic skills easily and confidentially in a practice-oriented setting.

ANTERIOR TEETH

SR PHONARES® II

SOFT

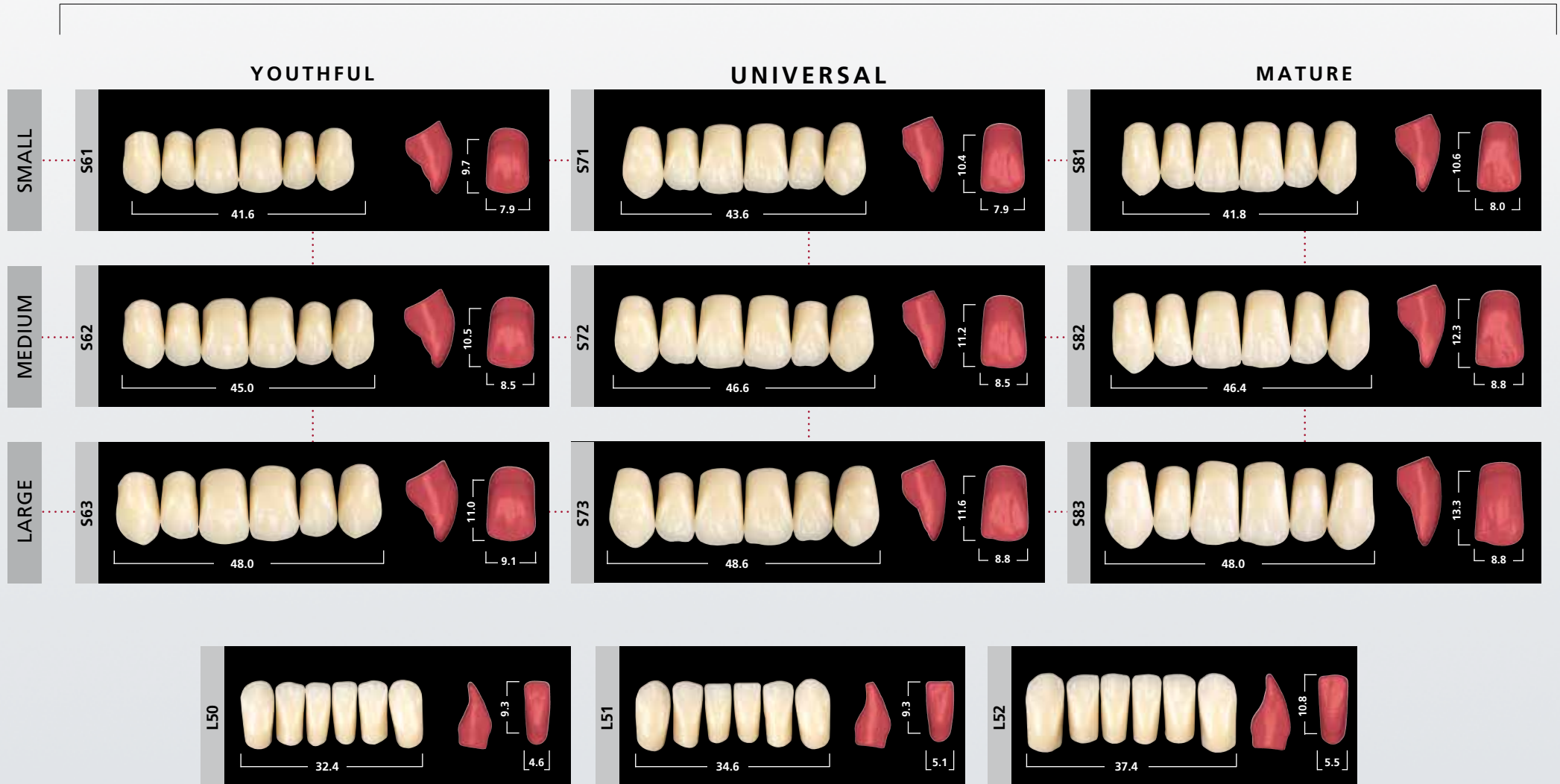

SR PHONARES® II

BOLD




POSTERIOR TEETH

SR **PHONARES® II**
Typ



SR **PHONARES® II**
Lingual

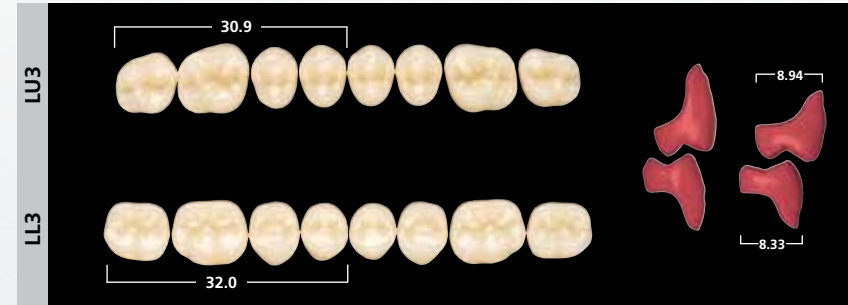

COMBINATION TABLE

		SR PHONARES® II		SR PHONARES® II	SR PHONARES® II
		upper	lower	<i>Typ</i>	<i>Lingual</i>
Small	soft	S61	L50, L51	NU3 / NL3	LU3 / LL3
		S71	L50, L51	NU3 / NL3, NU5 / NL5	LU3 / LL3, LU5 / LL5
		S81	L50, L51	NU3 / NL3	LU3 / LL3, LU5 / LL5
	bold	B61	L50, L51	NU3 / NL3	LU3 / LL3
		B71	L50, L51	NU3 / NL3, NU5 / NL5	LU3 / LL3
		B81	L50, L51	NU3 / NL3	LU3 / LL3
Medium	soft	S62	L52, L53	NU3 / NL3, NU5 / NL5	LU5 / LL5
		S72	L52, L53	NU5 / NL5	LU5 / LL5, LU6 / LL6
		S82	L52, L53	NU5 / NL5	LU5 / LL5, LU6 / LL6
	bold	B62	L52, L53	NU3 / NL3, NU5 / NL5	LU5 / LL5, LU6 / LL6
		B72	L52, L53	NU3 / NL3, NU5 / NL5	LU5 / LL5, LU6 / LL6
		B82	L52, L53	NU3 / NL3, NU5 / NL5	LU3 / LL3, LU5 / LL5
Large	soft	S63	L54, L55	NU5 / NL5	LU6 / LL6
		S73	L54, L55	NU5 / NL5, NU6 / NL6	LU6 / LL6
		S83	L54, L55	NU5 / NL5, NU6 / NL6	LU6 / LL6
	bold	B63	L54, L55	NU6 / NL6	LU6 / LL6
		B73	L54, L55	NU6 / NL6	LU6 / LL6
		B83	L54, L55	NU5 / NL5, NU6 / NL6	LU6 / LL6

This combination table is recommended as a guideline. In case of particular anatomical conditions, deviations are possible.

SR PHONARES® II

TOOTH MOULD ARTISTRY

Shade selection

with the SR PHONARES II shade guide

Shade selection should be performed on the patient under defined light conditions (5500K colour temperature) or in daylight. An accompanying shade guide enables a consistent shade selection procedure according to the A–D shade system. The tooth samples of the shade guide are based on the same layering pattern and materials as the original SR PHONARES II teeth. Shade deviations are therefore minimized.



Shades

- 16 A–D
- 4 Bleach

Ivoclar Vivadent ICDE



Being consistently on the latest standard of knowledge is a sign of your competence. Ivoclar Vivadent supports you in achieving this objective by providing you with both knowledge about the latest products and know-how on practice-oriented application and working techniques.

The Ivoclar Vivadent ICDE continuously offers basic and advanced training courses tailored to your needs.

Ask for our Course Programme!



This product forms part of our Implant Esthetics competence area. All the products of this area are optimally coordinated with each other.

CE 0123

ISO 22112:2005

Descriptions and data constitute no warranty of attributes and are not binding.

Printed in Germany

© Ivoclar Vivadent AG, Schaan/Liechtenstein
642684/0212/e/W

Ivoclar Vivadent AG
Bendererstr. 2
FL-9494 Schaan
Principality of Liechtenstein
Tel. +423 / 235 35 35
Fax +423 / 235 33 60
www.ivoclarvivadent.com

ivoclar
vivadent
passion vision innovation