FICHA TÉCNICA

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TECHNICAL DATA SHEET

IVORY TEMPORARY CEMENT		
REFERENCE	01551	
MANUFACTURER	MEDENTAL INTERNATIONAL INC (CONAMCO S.A. DE C.V.) José María Bustillos No. 28 Bis, 1er y 2do piso. Colonia Algarín, Delegación Cuauhtémoc, CP. 06880. Ciudad de México, México.	
CLASSIFICATION	Class IIa Medical Device, according to Directive 93/42/EEC.	
DESCRIPTION	Ivory temporary filling cement for temporary fillings in endodontics. Features: Self-curing. For occlusal temporary fillings. For temporary filling after endodontic treatments.	
COMPOSITION	Zinc oxide, zinc stearate and cotton oil.	
PRESENTATION	38 g.	
INTENDED USE	They are self-hardening temporary filling materials for the temporary filling of cavities.	
APPLICATION	By means of an instrument, apply the required amount into the wet cavity. The material hardens within a few minutes. Avoid occlusion loads until 2 hours after application. Deep cavities do not need to be filled to the bottom. When obturation of a healing, care should be taken not to exert pressure during application. Use rotary instruments to remove the temporary filling.	
INSTRUCTIONS FOR USE	See product instructions.	
STORAGE AND TRANSPORT	Do not store at a temperature above 24°C. Do not use after the expiration date has passed. Once opened, consume the bottles with screw cap as soon as possible.	