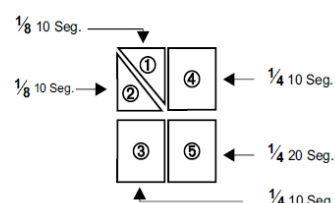
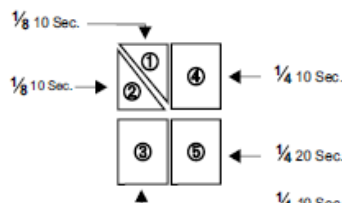
FICHA TÉCNICA

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TECHNICAL DATA SHEET

BESTDENT PHOSPHATE ZINC CEMENT		
REFERENCE	20494	
MANUFACTURER	MEDENTAL INTERNATIONAL INC (CONAMCO S.A. DE C.V.) José María Bustillos No. 28 Bis, 1er y 2do piso. Colonia Algarín, Delegación Cuauhtémoc, CP. 06880. Ciudad de México, México.	
CLASSIFICATION	Class IIa Medical Device, according to Directive 93/42/EEC.	
DESCRIPTION	Zinc oxide and aluminum phosphate based cement for permanent cementation of inlays, crowns, bridges and orthodontic bands. Ivory color.	
COMPOSITION	Zinc Oxyde and aluminum phosphate hydrated.	
PRESENTATION	32 g of powder + 15 ml of liquid.	
INTENDED USE	Cementation in inlay onlyas, crowns, bridges and orthodontic bands.	
MIXING TECHNIQUE	MIXING TECHNIQUE SETTING - TIME CONTROL (A) Powder-Liquid ratio. Thick mixes set faster than thin mixes. (B) The faster the powder is added to the liquid, the quicker the set. (C) Prolonged mixing tends to lengthen setting time. 	
INSTRUCTIONS FOR USE	See product instructions. INSTRUCTIONS FOR USE: (A) Use a cool mixing slab (72°F, 22°C) and a non-corrosive spatula, for mixing ZINC PHOSPHATE CEMENT. (B) Use a ratio of 1.3 to 1.4 gr. of POWDER to 0.85 g. of LIQUID. (C) Divide POWDER into four (4) equal portions. Divide one (1) of these quarter portions into two (2) parts. (D) ① Spatulate 1/8 of the portions into the entire amount of the liquid for 10 seconds. ② Add the other 1/8 of the portions and mix for 10 seconds. ③ ④ Add consecutively two (2) of the quarter portions, spatulating 10 seconds after each addition. ⑤ Add the final quarter portion and mix the entire amount for 20 seconds. This will give 1 minute total mixing time.	
STORAGE AND TRANSPORT	Do not storage in temperatures above 24°C. Close containers after use.	