FICHA TÉCNICA

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TECHNICAL DATA SHEET

PROTECTIVE FILM BLUE BESTDENT		
REFERENCES	20980	
MANUFACTURER/ RESPONSIBLE	Shanghai Dochem Industries Co., Ltd. Suite E2014, 2nd floor, No. 3558, Zhenbei rd. Putuo district, shanghai 200060, China	
CLASSIFICATION	Medical Device Class I, according to Regulation (EU) 2017/745 on medical devices.	
DESCRIPTION	Blue adhesive film. The product is non sterile and for single use. Easily removable	
SPECIFICACIONS	1.200 pre-cut sheets in one roll. Single sheet: 10x15 cm	
PACKAGING	Box with 1 roll.	
INTENDED USE	For dental use. Protection of instruments and materials.	
INSTRUCTIONS FOR USE	• Remove dispensing-slot cover. • Leave the film protector in the box. Insert the end of the film protector into the slot. • Tear off film protector and open its end. Place film protector on turbine and tubing. • Remove and dispose film protector after each patient.	
SAFETY INFORMATION	The product can ignite when in contact to a flame. If it burns, it will cause tear-producing smoke that could lead to suffocation.	
HANDLING AND STORAGE CONDITIONS	It does not require special storage and transportation conditions.	