FICHA TÉCNICA

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TECHNICAL DATA SHEET

MODELLING SPECIAL GYPSUM TYPE IV/4 - PINK		
REFERENCES	25660	
MANUFACTURER	MADESPA, S.A. C/ Río Jarama, 120, 45007, Toledo Spain	
CLASSIFICATION	Non-Sanitary Product (Prosthetic Class).	
DESCRIPTION	Low expansion and high surface hardness. Thixotropic. Its optimum fluidity under vibration allows to reproduce with fidelity all the details of the printing. Compatible with all printing materials. Conforms to I.S.O. 6873 standard.	
SPECIFICATION	- Mixing ratio water/powder: 22 ml/100 gr. - Working time: 6'-7'. - Setting expansion (2h): ~0.09 %. - Compressive strength (1h): ~30 MPa.	
COMPOSITION	Calcium sulfate.	
PACKAGE	1 unit of 6 kg.	
SHELF LIFE	3 years.	
INSTRUCTIONS FOR USE	Put water in a cup and slowly add the powder according to the recommended proportion. The use of a vibrator is recommended.	
INTENDED USE	Type IV hard plaster, pink color, suitable for both clinical and laboratory work, for the realization of models in fixed prosthesis, skeletal and removable prosthesis.	
STORAGE AND TRANSPORT	Store containers in a dry, well-ventilated place, protected from humidity.	