FICHA TÉCNICA

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TECHNICAL DATA SHEET

BESTDENT SELF-ETCH DENTAL ADHESIVE		
REFERENCE	30503	
MANUFACTURER	MEDENTAL INTERNATIONAL INC. (CONAMCO S.A. DE C.V.), José Ma. Bustillos No. 28-Bis, Col. Algarin, Deleg. Cuauhtemoc D.F., 06880 México, Ciudad de México	
CLASSIFICATION	Class IIa Medical Device, Directive 93/42/EEC.	
DESCRIPTION	Single-component, self-etching and light-curing adhesive that provides strong adhesion to enamel and dentin. Allows selective etching or self-etching techniques.	
COMPOSITION	Water, udma, hema phosphate, pmgam, highly dispersed silicon dioxide, catalyst, stabilizers and ethanol.	
PACKAGING	5 ml.	
CONTRAINDICATIONS	- Do not use if the patients are allergic to the components of this product. - Do not use in combinations with a dual or multi-cure or chemical cure cements and composites. - Do not use to direct pulp capping. - Do not use when the application techniques described are not possible. - Do not use if is eugenol coating present in the cavity.	
INTENDED USE	This product is a single-component, self-etching, light-curing, adhesive system light cured for enamel and dentin bonding.	
INSTRUCTIONS FOR USE	See instructions on the product. 1. A dry operating field must be ensured. Ideally, a rubber dam should be used. 2. In very deep cavities, areas close to the pulp should be selectively coated with a calcium hydroxide liner and subsequently covered with pressure resistant cement (glass ionomer cement typelliner). 3. Apply a thick layer of this product on the surface for at least 30 seconds. All cavity surfaces should be thoroughly coated. The total reaction time should be no shorter than 30 seconds.	

TECHNICAL DATA SHEET

	4. Disperse excess amounts of this product with a strong stream of air until there is no longer any movement of the material. 5. Polymerize this product for 10 seconds at a light intensity of more than 500mW/cm².
HANDLING AND STORAGE CONDITIONS	Do not storage at temperature above 24° C. close the packages immediately after to dispense the product.