FICHA TÉCNICA

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TECHNICAL DATA SHEET

DISINFECTANT DETERGENT INSTRUMENTS 2%		
REFERENCES	49890: (1L). 49891: (5L)	
MANUFACTURER	D.M.D. 1, rue de l'Espoir F-59260 Lezennes, France Tel : + 33 3 20 67 67 67 / Fax : + 33 3 20 67 67 68	
CLASSIFICATION	Class IIb Medical Device, Directive 93/42/EEC.	
DESCRIPTION	- Concentrated product. - Economical: dilution at 2% or 1L = 50L. - Broad spectrum of antimicrobial activity starting at 5 min. - Non-corrosive and compatible with a wide range of metals and materials. - Can be used in ultrasonic tank. - Effective formula in hard and fresh water. - pH of pure product: ≈ 7 - Light blue color / lemon odor. - Free of CMR* substances, free of aldehydes. - Not classified as environmentally hazardous at the use dose.	
COMPOSITION	Didecyltrimethylammonium chloride, chlorhexidine digluconate, non-ionic surfactants, perfume, colouring, excipients	
PACKAGE	Drum of 1L and 5L.	
INTENDED USE	Disinfectant detergent solution for cleaning and disinfection of dental instruments prior to sterilization.	
INSTRUCTIONS FOR USE	Concentrated solution. Prime the associated pump before _rst use. Use for soaking or in ultrasonic bath at the required dilution, respecting a minimum contact time of 5 min. Totally immerse the medical device. Keep the soaking bath covered during the contact time. Brush if visible soiling is present. Thoroughly rinse treated medical devices with tap water to remove all traces of product and organic soiling. Replace the solution as soon as it is visibly soiled and at least once per day to maintain the e_ectiveness of the product. Scienti_c dossier (e_cacy, compatibility, etc.) available on request.	

TECHNICAL DATA SHEET

MICROBIOLOGICAL PROPERTIES	Bactericidal* according to EN 13727/EN 14561 in 5 min, 0.5%. Yeasticidal* according to EN 13624/ EN 14562 in 5 min, 2%. Tuberculocidal* according to EN 14348/ EN 14563 in 60 min, 2%. Active on enveloped viruses* according to EN 14476/EN 17111 in 5 min, 1%. *=dirty conditions
STORAGE AND TRANSPORT	Store at a temperature between 5°C and +35°C.