FICHA TÉCNICA

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TECHNICAL DATA SHEET

DISINFECTANT WIPES		
REFERENCES	49898: Wipes bottle 100 u. 49899: Wipes Refill 100 u.	
MANUFACTURER	D.M.D. 1, rue de l'Espoir F-59260 Lezennes, France Tel : + 33 3 20 67 67 67 / Fax : + 33 3 20 67 67 68	
CLASSIFICATION	Class IIb Medical Device, Directive 93/42/EEC.	
DESCRIPTION	- Ready-to-use wipes. - Broad spectrum antimicrobial action in only 30 seconds. - Does not leave stains or residues. - Excellent compatibility with metals. - Wipe size: 130 mm x 190 mm. - Lemon scent. - pH of the impregnation solution: ≈ 5.5. - Percentage of alcohol in the impregnation solution: 60% v/v. - Free of aldehydes, free of CMR* substances..	
COMPOSITION	Impregnation solution: Ethanol, N,N-Didecyl-N-methyl-poly(oxyethyl) ammonium propionate, perfume.	
PACKAGE	- 49898: Wipes bottle 100 u. - 49899: Wipes Refill 100 u.	
INTENDED USE	Disinfectant wipes for rapid surface disinfection of medical devices. Ideal for use between two patients.	

TECHNICAL DATA SHEET

INSTRUCTIONS FOR USE	See product instructions. Ready-to-use wipes. Use on previously cleaned medical devices. Apply to the whole area to be treated, using as many wipes as necessary. Respect the indicated contact time for the required antimicrobial activity. Thoroughly rinse the treated areas intended to come into contact with the skin or mucosa with tap water. Repeat application as often as necessary or refer to the hygiene plan in place. Close the packaging properly after each time it is opened. Do not reuse the wipe, disinfection can no longer be guaranteed. Remarks: Before application, check the compatibility of medical devices treated with alcohol. Electrical devices being disinfected must be switched off. <u>Microbiological properties:</u> Bactericidal and yeasticidal according to with EN 16615* in 30 sec. Impregnation solution: Bactericidal according to EN 13727* in 30 sec. Yeasticidal according to EN 13624* in 30 sec. Fungicidal according to EN 13624* in 5 min. Tuberculocidal according to EN 14348* in 30 sec. Mycobactericidal according to EN 14348* in 5 min. Active against enveloped viruses: Vaccinia – according to EN 14476* in 30 sec. Limited spectrum virucidal according to EN 14476* in 5 min. (Adenovirus in 30 sec/Norovirus (MNV) in 5 min). *=clean conditions.
PRECAUTIONS FOR USE:	DANGER. Highly flammable liquid and vapour. Causes serious eye irritation. Keep away from heat, hot surfaces, sparks, open flames and other sources. No smoking. Wear eye/face protective equipment. Dispose of the product and its container as hazardous waste. Medical Device. Reserved for professional use.
STORAGE AND TRANSPORT	Store at a temperature between 5°C and +25°C.