FICHA TÉCNICA

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TECHNICAL DATA SHEET

DESINFECTANT DETERGENT ASPIRATION		
REFERENCES	49900 : Desinfectant detergent aspiration (1L.) 49901 : Desinfectant detergent aspiration + dispenser (5L.)	
MANUFACTURER	D.M.D. 1, rue de l'Espoir F-59260 Lezennes, France Tel : + 33 3 20 67 67 67 / Fax : + 33 3 20 67 67 68	
CLASSIFICATION	Class IIa Medical Device, in accordance with Directive 93/42/EEC, complying with regulation 607/2023 (EU), regarding new transitional orders for the application of regulation 2017/745 (EU).	
DESCRIPTION	- Concentrated product. - Non foaming formula. - Broad spectrum of antimicrobial activity from 5 minutes. - Economical: 2% dilution, i.e. 1L = 50L. - Triple action: cleans, descaling and disinfects. - Compatible with all types of suction systems. - Can be used for spittoons and amalgam separators. - Effectiveness tested on biofilm according to ISO/TS 15883 - 5: 2006. - pH of the pure product: ≈ 10.5. - Red color / unscented. - Not classified as dangerous for the environment at the dose of use.	
COMPOSITION	N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine, surfactant association, chelating agent, colorant and excipients.	
PACKAGE	- 1 liter container. - 5 liter decanter with dosing pump.	
INTENDED USE	Disinfectant detergent solution for daily cleaning and disinfection of dental suction systems and amalgam separators.	

TECHNICAL DATA SHEET

INSTRUCTIONS FOR USE	Be sure to wear appropriate gloves. 1. Dilute 2% in a bucket to have 2 liters of solution (40 ml for 2 liters of water). 2. Aspirate the diluted solution. 3. Allow the solution to act for at least 15 minutes. 4. Rinse the system by aspirating at least 1 liter of clean water. <u>Microbiological properties:</u> - Bactericidal: EN 1040, EN 13727, EN 14561 - ~15 minutes. - Tuberculocidal: EN 14348, EN 14563 - 60 minutes. - Levuricide: EN 1275, EN 13624, EN 14562 - 15 minutes. - Active according to EN 14476 on BVDV (HCV model virus), Herpes virus, Vaccinia virus - 5 minutes.
STORAGE AND TRANSPORT	Store at a temperature between 5°C and +25°C.