FICHA TÉCNICA

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TECHNICAL DATA SHEET

ROOT CANAL PREPARATION (EDTA)		
REFERENCES	49933	
MANUFACTURER	J.RIPOLL S.L. C/RIO TIETAR Nº20 POLIGONO EL NOGAL ALGETE (MADRID) España	
CLASSIFICATION	Class IIa Medical Device, Directive 93/42/EEC.	
DESCRIPTION	EDTA (tetra acetic diamine ethylene tetra acetic acid). Root canal widener, helps to dissolve the pulp tissue and removes the layer of detritus, in addition, its lubricating effect makes easier the use of files. It allows the removal of pathological mineral content (pulpitis, calcifications...), has antibacterial action and is easily removed thanks to its hydrosolubility.	
COMPOSITION	EDTA (Na2) 15% , Carbamide Peroxide 10% and Excipient.	
PACKAGE	4 syringes of 1.5 ml + tips.	
INSTRUCTIONS FOR USE	See instructions on the product.	
INDICATIONS	- Ethylenediamine tetraacetic acid and its salts have a strong chelating power of various ions. This property gives them a multitude of applications. In dentistry they are used to soften dentin and facilitate its extraction. - After grinding a tooth, the resulting smear layer is more easily removed if an EDTA product is used more easily if an EDTA product is used. - In the instrumentation of canals, the prior placement of the product in the file produces a softening of the adjacent dentin, which facilitates and accelerates the instrumentation of the canal. - The product also includes carbamide peroxide so that once the canal is instrumented, when irrigated with hypochlorite. Once the canal is instrumented, when it is irrigated with hypochlorite, a bubbling effect is produced that helps to eliminate the filing debris.	
PRECAUTIONS	- A rubber dam should be used to isolate the tooth to be endodontized. - No traces of the product should be left inside the canal and the product	

TECHNICAL DATA SHEET

	should be prevented from passing through the apex. If this occurs, no toxic effect should appear. • After instrumentation, irrigate to rinse thoroughly and then dry with paper points. • Caution to allergic reactions to EDTA or carbamide peroxide. • Keep the product tightly closed.
PURPOSE	Root canal preparer for endodontics for professional use.
STORAGE AND TRANSPORT	Keep jars closed and store refrigerated between 4-18°C.