FICHA TÉCNICA

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TECHNICAL DATA SHEET

DISINFECTION FOR MEDICAL DEVICES BESTDENT		
REFERENCES	60157: Disinfection for Medical Devices BESTDENT 1L. 60158: Disinfection for Medical Devices BESTDENT 5L.	
MANUFACTURER	PRISMAN GmbH Otto-Hahn-Ring 6-18 D-64653 Lorsch Phone: +49 (0)6251-866 980-0	
CLASSIFICATION	Class IIb Medical Device, Directive 93/42/EEC.	
DESCRIPTION	BESTDENT Medical Device Disinfectant (1L and 5L).	
COMPOSITION	100 g of product contains: - 43.0 g ethanol; 8.0 g 1-Propanol. - 0.055 g didecyltrimethylammonium chloride.	
PACKAGE	Available in 1L spray bottle and 5L canister.	
INTENDED USE	Ready-to-use alcoholic rapid disinfectant for prophylactic surface disinfection of alcohol-resistant medical devices in the medical sector. Not suitable for final (terminal) disinfection of invasive medical devices.	
INSTRUCTIONS FOR USE	Apply the quick surface disinfectant so that it covers the entire surface. Then allow the disinfectant to act for the specified contact time. For wipe disinfection, thoroughly wet a cloth with the disinfectant solution and wipe the areas or apply the solution directly to the area and then wipe. All surfaces to be disinfected must be thoroughly wet and covered. Suitable for the medical sector. Only suitable for use on alcohol-resistant materials (not suitable e.g. for acrylic glass). <u>Contact times:</u> - Bacteria according to EN 13727 and EN 13697: Conc. 1 min. - Candida albicans according to EN 13624 and EN 13697: Conc. 1 min. - Inactivation of TB (Tuberculosis) (M. terrae) according to DGHM: Conc. 1 min. - Enveloped viruses (e.g. HBV, HIV, HCV) according to RKI/DVV and EN 14476 under clean conditions: Conc. 2 min. - Norovirus (EN 14476, clean conditions): Conc. 1 min. - Contact times for disinfection by wipe application: Bacteria and Candida albicans according to VAH and EN 16615: Conc. 5 min.	
SAFETY INSTRUCTIONS	H226 Flammable liquid and vapors. Keep away from heat/sparks/flames/hot surfaces. Do not smoke. Keep container tightly closed. IF IN EYE CONTACT: Rinse cautiously with water for several minutes. If contact lenses are worn and easy to remove, remove contact lenses. Continue rinsing. Store container tightly closed in a well-ventilated area. Dispose of contents/container in accordance with national/international regulations.	
STORAGE AND TRANSPORT	Store in a cool and dry place. Keep away from direct sunlight and heat sources. Protect from humidity.	