FICHA TÉCNICA

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TECHNICAL DATA SHEET

PROCLINIC MATRIX STRIPS IN ROLLS			
REFERENCES	65870: 0,03 x 5 mm (3 m). 65871: 0,03 x 6 mm (3 m). 65872: 0,03 x 7 mm (3 m).		
MANUFACTURER	HOPF RINGLEB & CO GMBH &CIE Gardeschützenweg 82 2203 Berlin, Alemania		
CLASSIFICATION	Class I medical device, according to Regulation (EU) 2017/745 on medical devices.		
DESCRIPTION	Dies in rolls of 3 meters, with a thickness of 0.03 mm.		
COMPOSITION	Stainless steel.		
SPECIFICATION	Thickness: 0.03 mm Width: 5,0, 6,0, 7,0 mm Length: 3.0 m		
PRESENTATION	1 roll of 3 m.		
USE INTENDED	For dental use. Matrix strips replace one or more missing tooth walls and are ideal for promoting proximal contour and maintaining the point of contact, being sufficiently rigid so that they do not come loose during amalgam fixation or insertion of restorative material. They wrap around the tooth, forming a proximal wall.		
INSTRUCTIONS FOR USE	(a) Surround the tooth with the matrix band to replace the missing wall. (b) Apply the amalgam or dental resin and remove the matrix band.		
SAFETY INFORMATION	Product exclusively for dental use by duly trained professionals. Non sterile product. Sterilize, before use, according to standard procedure.		
STORAGE AND TRANSPORTATION	Store in a cool and dry place. Keep away from sunlight. Protect from humidity.		