FICHA TÉCNICA

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TECHNICAL DATA SHEET

BESTDENT STAINLESS STEEL MATRIX BANDS		
REFERENCES	65873: 0,05x5 mm (3m) 65874: 0,05x6 mm (3m) 65875: 0,05x7 mm (3m)	
MANUFACTURER	HOPF RINGLEB & CO GMBH &CIE Gardeschützenweg 82 2203 Berlin, Alemania	
CLASSIFICATION	Class I medical device, according to Regulation (EU) 2017/745 on medical devices.	
DESCRIPTION	Stainless Steel matrix bands, 3 m length.	
COMPOSITION	Stainless steel AISI 304.	
SPECIFICACIONS	Thickness: 0,05 mm Width: 5,0, 6,0 or 7,0 mm Length: 3,0 m	
PACKAGING	1 roll of 3 m.	
INTENDED USE	Dental use. The matrix bands replace one or more walls absent from the tooth and are ideal for promoting the proximal contour and maintaining the point of contact, being rigid enough not to displace during the amalgam condensation or insertion of restorative material. They envelop the tooth, forming a proximal wall.	
INSTRUCTIONS FOR USE	(a) Wrap the tooth with the band matrix to replace the missing wall. (b) Apply the amalgam or dental resin and remove the band matrix.	
SAFETY INFORMATION	Personal safety: none. Only dental surgeons should use this product because they have scientific information necessary for the proper use. Non sterile product. Sterilize before use, according to the standard procedure.	
HANDLING AND STORAGE CONDITIONS	Store in a cool dry place, away from direct sunlight. Protect from humidity.	