FICHA TÉCNICA

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TECHNICAL DATA SHEET

RESIN CROWNS AND BRIDGES		
REFERENCES	69402: A2 69403: A3 69405: A1	
MANUFACTURER	G-PHARMA 35, avenue du Gros Chêne B.P. 10279 HERBLAY 95617 CERGY-PONTOISE Cedex FRANCE	
CLASSIFICATION	Class IIa Medical Device, according to Directive 93/42/EEC on medical devices.	
DESCRIPTION	Self-curing resin with excellent mechanical properties for the production of bridges and temporary crowns.	
SPECIFICATIONS	This product is available in the following shades: A1 - A2- A3. Total time (working time + setting time) at 22°C, is 2'35" to 2'45".	
COMPOSITION	Glass fillers (30-40%) - Dimethacrylate resins (40-60%) - Catalysts (<1%) - BHT (<1%) - Pigments (<1%).	
PACKAGING	1 cartridge of 50 ml per package + 10 Tips.	
SHELF LIFE	This product has a shelf-life of 24 months.	
PRECAUTION	The use on eugenol-containing products with based composite-based dental materials should be avoided since they interfere with the curing process.	
INTENDED USE	Due to its special consistency, its rapid setting and excellent mechanical qualities, Bestdent is ideally suited for temporary crowns or small aesthetic bridges, made from a pre-impression only.	
INSTRUCTIONS FOR USE	See instructions on the product.	
HANDLING AND STORAGE CONDITIONS	Store at a temperature between 5°C and 25°C. When the product is stored in the refrigerator, it should be brought back to room temperature 15 minutes minimum before use, to restore its normal viscosity and curing time.	