FICHA TÉCNICA

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TECHNICAL DATA SHEET

BESTDENT HOME WHITENING		
REFERENCES	74250: HOME PC WHITENING 10%. 74251: HOME PC WHITENING 16%.	
MANUFACTURER/ RESPONSIBLE	DentalSur 2003 S.L Avda. Ortega y Gasset 152 296006 – Málaga. Spain	
CLASSIFICATION	Cosmetic.	
DESCRIPTION	Bestdent whitening kits for clinic and home use, products with full quality guarantee and excellent results. - The home kits include three 3 cc syringes of carbamide peroxide, with color guide and instructions, and both are CPNP registered products. 1. Carbamide Peroxide 10% concentration, ideal for home treatment of sensitive patients, with excellent results. Daily application at home for three weeks of treatment, 50 to 60 minutes per day. 2. Carbamide Peroxide concentration at 16% for home treatments of only 30 to 40 minutes per day, for three weeks. Excellent results are achieved.	
COMPOSITION	Oxygenated water. Carbamide peroxide bleaching gel.	
PACKAGE	3 syringes of 3 ml.	
INTENDED USE	Carbamide peroxide teeth whitening kit.	
INSTRUCTIONS FOR USE	See the instructions inside the product. <u>Instructions From Home</u> - Once we have the splint made, both the upper and the inner arch, pour the product on the vestibular side of the splint (the one that will be in contact with the dental surface to be whitened).	

TECHNICAL DATA SHEET

	- Repeat the same with the other splint for the other arch. - Place the splints without biting excessively. Avoid swallowing the product. - Do not eat or drink during the process. - After 30-40 minutes with the splints on, remove and rinse with plenty of water. - Repeat the process daily until the product is used up. Apply just before going to sleep.
WARNING	In each cycle of use, the first application should be made by a dentist or under direct supervision, provided that an equivalent degree of safety is guaranteed. Once the first application has been made, it can be supplied to the consumer to complete the cycle of use.
HANDLING AND STORAGE CONDITIONS	Store in a cool, dry place, away from moisture and direct light, as these factors may affect its composition.