

FICHA TÉCNICA

DAPPEN AMALGAMA		
REFERENCIAS	80300	
FABRICANTE / RESPONSABLE	CHL MEDICAL SOLUTIONS, Srl Via Felice Cavallotti, 15 20122 Milán - Italia.	
CLASIFICACIÓN	Producto no Sanitario (Clase consumo).	
DESCRIPCIÓN	Dappen amalgama.	
ESPECIFICACIÓN	Diámetro 3cm y altura 2cm.	
COMPOSICIÓN	Acero inoxidable con base de goma antideslizante.	
VIDA ÚTIL	La vida útil del instrumento depende de la frecuencia de uso, el cuidado del usuario y los métodos de reprocesamiento adecuados.	
PRESENTACIÓN	1 unidad.	
FINALIDAD PROPUESTA	Diseñado para manipular, mezclar y contener materiales dentales durante procedimientos odontológicos.	
ESTERILIZACIÓN	Todos los instrumentos se deben examinar, limpiar y esterilizar antes de cada uso. Usar detergentes y desinfectantes adecuados para instrumentos médicos o dentales. El final de la vida útil de los instrumentos viene determinado por el desgaste y los daños durante el uso, y los instrumentos deben inspeccionarse para detectar defectos durante el proceso de limpieza preliminar. Esterilizar en autoclave o con calor seco.	
ALMACENAMIENTO Y TRANSPORTE	No requiere condiciones especiales de almacenamiento y transporte.	

TECHNICAL DATA SHEET

DAPPEN AMALGAM		
REFERENCE	80300	
MANUFACTURER	CHL MEDICAL SOLUTIONS, Srl Via Felice Cavallotti, 15 20122 Milán - Italia.	
CLASSIFICATION	Non Sanitary Product (Consumer Class).	
DESCRIPTION	Dappen amalgam.	
SPECIFICATION	Diameter 3cm and height 2cm.	
COMPOSITION	Stainless steel with non-slip rubber base.	
SHELF LIFE	The lifetime of the instrument depends on frequency of use, user care and proper reprocessing methods.	
PACKAGE	1 unit.	
FINALITY USE	Designed to handle, mix and contain dental materials during dental procedures.	
STERILIZATION	All instruments should be examined, cleaned and sterilized before each use. Use detergents and disinfectants suitable for medical or dental instruments. The end of life of instruments is determined by wear and damage during use, and instruments should be inspected for defects during the preliminary cleaning process. Autoclave or dry heat sterilize.	
STORAGE AND TRANSPORT	No special storage and transport conditions required.	