FICHA TÉCNICA

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TECHNICAL DATA SHEET

PEESO REAMERS Nº 1-6 BESTDENT		
REFERENCES	81013: Peeso Reamers No. 1 81014: Peeso Reamers No. 2 81015: Peeso Reamers No. 3 81016: Peeso Reamers No. 4 81017: Peeso Reamers No. 5 81018: Peeso Reamers No. 6	
MANUFACTURER	Harald Nordin S.A. Z.I. La Foge CH-1816 Chailly s/Montreux, Switzerland	
CLASSIFICATION	Class IIa medical device, Directive 93/42/CEE.	
SPECIFICATIONS	Length: 32 mm No 1 Ø 0.70 No 2 Ø 0.90 No 3 Ø 1.10 No 4 Ø 1.30 No 5 Ø 1.50 No 6 Ø 1.70	
COMPOSITION	Stainless Steel.	
INSTRUCTIONS	Shape pre-treated, root canal at moderate speed.	
CAPACITY	6 pieces of the same size.	
PRECAUTIONS	When performing a root canal treatment it is of utmost importance that the root-canal has been prepared with a reamer of appropriate size and shape. When using the reamers, start with a reamer number smaller than the diameter number desired and proceed from reamer number to reamer number step by step till desired number reamer can be used. Reamers are numbered 1-6. Reamers should be used at moderate r.p.m. speed and with a careful up- and down movement only. If these precautions are not taken, reamer can get stuck and/or break off in root canal. Reamers should be used “Lege Artis” and with professional judgement. Reamers can be sterilized by all methods. Sterilization is recommended between interventions	
POSSIBLE RISKS	The risks with reamers are mechanical they can break off if improperly used, if stuck in root canal there are several known methods for removal. As reamers are sharp instruments a risk for a cut is the same as with any sharp instrument. The risks with reamers are known and dentists are trained to deal with them and there are no more risks than any normal dental burr routinely used.	
STORAGE AND TRANSPORT	No special storage and transport conditions require	