FICHA TÉCNICA

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TECHNICAL DATA SHEET

BESTDENT TURBINE ARKANSAS TIPS		
REFERENCE	98081	
MANUFACTURER	EVE Ernst Vetter GmbH Neureutstr. 6 - 75210 Keltern Germany Tel.: +49 7231 97 77 -0 Fax: +49 7231 97 77 -99 E-Mail: info@eve-rotary.com	
CLASSIFICATION	Class IIa Medical Device, in accordance with Regulation (EU) 2017/745.	
DESCRIPTION	Abrasive arkansas tips, made of high quality, fine-grained aluminum oxide, with ceramic binder for an ultra-fine finish. Features: - They guarantee vibration-free work. Particle size = extra fine Binder = hard - Sterilizable in autoclave and unaltered by chemical disinfectants. - They pass a dimensional quality control and an optical quality control. - For polishing composites.	
SPECIFICATION	Active part in the shape of a bullet or flame with a diameter of 2.5 mm and 6 mm in length and normal stem. ISO 635 314 243 505 025 Arkansas FG 15.000 RA 7.000 12/100 L mm Size $\varnothing$ 1/10 mm Order No. ISO No. 635 314...     6,0 025 AS01.025 FG 243 505 025	
COMPOSITION	Aluminum oxide.	

TECHNICAL DATA SHEET

PRESENTATION	Contains 10 units.
INTENDED USE	For ultra-fine finishing and finishing Composite Ceramic.
STERILIZATION	Device: Autoclave according to DNI EN 285 or small steam autoclave according to DIN EN 13060, type B process. Process: Steam sterilization with fractionated pre-vacuum, 134 °C, waiting time min. 3 min (in Germany according to KRINKO/BfArM recommendation 134 °C min. 5 min) 0 132 °C min. 3 min (validation parameter). 3 min (validation parameter). Longer waiting times are permissible. 1. Place the packaged products in the sterilization chamber. 2. Start the program. 3. Remove the products at the end of the program and allow them to cool down. 4. Check the packages for damage afterwards. Damaged containers should be considered as non-sterile. The instruments should be repacked and sterilized.
INSTRUCTIONS FOR USE	See instructions on the product.
STORAGE AND TRANSPORT	It does not require special storage and transportation conditions.