FICHA TÉCNICA

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TECHNICAL DATA SHEET

HARD NATURAL GYPSUM TYPE IV/4 IVORY		
REFERENCES	H62017	
MANUFACTURER	MADESPA, S.A. C/ Río Jarama, 120, 45007, Toledo Spain	
CLASSIFICATION	Non-Sanitary Product (Prosthetic Class).	
DESCRIPTION	Extra hard, with optimum fluidity during the preparation of the model. Thixotropic plaster that has a great fluidity under slight vibration. Compatible with all impression materials. Conforms to I.S.O. 6873 standard.	
SPECIFICATION	Ivory color. Type 4. Mixing ratio: 22/23ml-100gr. Working time: 6'-7'. Final setting: 10'-13'. Setting expansion: 0,09%. Compressive strength (1h): 39 MPa Mixing time under vacuum: 30 Manual mixing time: 60".	
COMPOSITION	Calcium sulfate.	
PACKAGE	1 unit of 6 kg.	
SHELF LIFE	3 years.	
INSTRUCTIONS FOR USE	Put water in a cup and slowly add the powder according to the recommended proportion. The use of a vibrator is recommended.	
INTENDED USE	Used for removable dies for crowns and bridges and total or partial removable prostheses made either in conventional muffles or by injection. It is also recommended for skeletal models	
STORAGE AND TRANSPORT	Store containers in a dry, well-ventilated place, protected from humidity.	