FICHA TÉCNICA

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TECHNICAL DATA SHEET

INTRAORAL STEP PLIERS / STEP		
REFERENCE	L9440 L9441 L9442	
MANUFACTURER	DENTAL WORLD S.R.L Via Antichi Pastifici,15 Zona industriale -70056 Molfetta (BA) Italia	
CLASSIFICATION	Class I medical device, EU regulation 2017/745.	
DESCRIPTION	Pliers to create intraoral steps.	
SPECIFICATION	<u>Measurements:</u> L9440: 0.5 L9441: 0.75 L9442: 1 mm.	
COMPOSITION	Nickel and chromium.	
PACKAGING	1 unit per package.	
STERILIZATION AND CLEANING	Orthodontic instruments are supplied “NONSTERILE”: it is recommended that they be washed, rinsed and properly sterilized before each use. Cleaning using an Ultrasonic device is recommended; if not available, instruments can also be rinsed manually: use a stiff brush with plastic bristles (NOT metal bristles) and be sure to use the appropriate washing and decontamination solutions. It is strongly recommended that instruments be sterilized only after they have been thoroughly washed; if the instruments are not perfectly clean, sterilization may be compromised. Please remember to carefully follow the instructions for use provided by the manufacturer of the sterilization equipment. Steam autoclave sterilization is recommended at a temperature approximately between 121°C and 134°C and a pressure between 1.03 bar and 2.3 bar.	
INTENDED USE	Its purpose is to create steps (intraoral steps) in the orthodontic wires or archwires.	
INSTRUCTIONS FOR USE	See instructions on the product.	
STORAGE AND TRANSPORT	It does not require special storage and transport conditions.	