FICHA TÉCNICA

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TECHNICAL DATA SHEET

ROSE PLIERS WITHOUT GROOVES		
REFERENCE	L9444	
MANUFACTURER	DENTAL WORLD S.R.L Via Antichi Pastifici,15 Zona industriale -70056 Molfetta (BA) Italia	
CLASSIFICATION	Class I medical device, EU regulation 2017/745.	
DESCRIPTION	Wire contouring pliers.	
SPECIFICATION	Sizes: For wire up to 0.5 mm max. (.020").	
COMPOSITION	Nickel and chromium.	
PACKAGING	1 unit per package.	
CLEANING AND STERILIZATION	Orthodontic instruments are supplied “NONSTERILE”: it is recommended that they be washed, rinsed and properly sterilized before each use. Cleaning using an Ultrasonic device is recommended; if not available, instruments can also be rinsed manually: use a stiff brush with plastic bristles (NOT metal bristles) and be sure to use the appropriate washing and decontamination solutions. It is strongly recommended that instruments be sterilized only after they have been thoroughly washed; if the instruments are not perfectly clean, sterilization may be compromised. Please remember to carefully follow the instructions for use provided by the manufacturer of the sterilization equipment. Steam autoclave sterilization is recommended at a temperature approximately between 121°C and 134°C and a pressure between 1.03 bar and 2.3 bar.	
INTENDED USE	It is to facilitate adjustment and cutting tasks in orthodontics, with a smooth and precise action.	
INSTRUCTIONS FOR USE	See instructions on the product.	
STORAGE AND TRANSPORT	It does not require special storage and transport conditions.	