Glass ionomer cement

RADIOPAQUE

Instructions for use

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DE

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INSTRUCTIONS FOR USE

EN

CLASSICAL GLASSIONOMER LUTING CEMENT

PRODUCT DESCRIPTION

Proclinc Expert Glass Ionomer Cement is a fluoride containing radiopaque classical strontium glassionomer cement designed for cementing a wide variety of prosthetic appliances and materials. As a glassionomer cement, it features good chemical adhesion and biocompatibility. It bonds to enamel, dentine, non-precious metals and porcelain. The glassionomer cement is suitable for cementing all types of crowns, inlays, onlays, bridges, endodontic posts and orthodontic bands. It can also be used for amalgam bonding and as liner.

Proclinc Expert Glass Ionomer Cement meets the requirements of DIN EN ISO 9917-1 (Specification for Dental Water based Cements).

Indications/intended use

- cementing of all types of crowns, inlays, onlays, bridges, endodontic posts and orthodontic bands
- liner under composite fillings and amalgam

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

- Pulp capping
- Allergic reactions to glass ionomer cements

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

APPLICATION

1. Tooth Preparation

- Prepare tooth in usual manner. Clean prepared tooth with pumice and water only. Rinse thoroughly and dry, but do not desiccate.
- By using a cotton pellet apply a dentine conditioner for 20 sec to remove the smear layer. Rinse the dentine conditioner with water and dry it in an airstream, but do not desiccate.
- Pulp capping with Proclinc Expert Glass Ionomer Cement is contra indicated. To deep areas of possible pulp exposure apply calcium hydroxide liner.

2. Dispensing and Mixing

- The powder/liquid ratio to achieve a suitable consistency is 1.8 / 1.0. This can be obtained by mixing of one spoon (blue colored) of powder and two drops of liquid.
- Note:** The spoon dosage is only an approximate reproduction of the nominal mixing ratio.
- For accurate dispensing of Proclinc Expert Glass Ionomer Cement Powder shake the bottle to loosen the powder. Overfill the spoon with the powder, level the powder by using the collar at the top of the bottle and carry it onto the mixing pad. Avoid compressing powder into the spoon with the inside wall of the bottle.
- For dispensing of Proclinc Expert Glass Ionomer Cement Liquid turn the bottle vertically with the tip about 5cm above the mixing pad. Steady your hand and squeeze the bottle gently to dispense the drops. If any bubbles are present, lightly tap the bottle with the fingers holding it. Discout drops that contain bubbles and are obviously not full-sized.
- Use a small spatula to rapidly mix half the powder into the liquid. Then mix the remaining powder into the mixed paste to achieve a glossy appearance. Total mixing time is 30 sec.
- After use, tightly close both liquid and powder bottles to prevent exposure to moisture.

3. Cementation Technique

- Mix the required amount of Proclinc Expert Glass Ionomer Cement. Coat the prepared bonding surface of the restoration with 1 mm of cement (Do not overfill!) and seat immediately (working time 1:25 min from start of mixing at 23°C). Seat the restoration by using steady, even pressure. Net setting time is about 4:30 min.
- Remove excess cement at the first setting stage.
- Maintain isolation until the set of the cement is verified (approx. 4:30 min).
- Note:** Higher temperatures will shorten the working time, lower temperatures will prolong the working time.
- An overextended working time will cause the loss of adhesion to the dental enamel and the dentin.

CONCLUSIVE NOTES/ WARNINGS

- Do not use Proclinc Expert Glass Ionomer Cement with patients who show an allergy to the material. In case of allergic reactions immediately stop the application, and advise the patient to consult a physician. An operator, who has a history of allergy to glass ionomer-cements should not handle Proclinc Expert Glass Ionomer Cement.
- Do not allow the liquid or cement mixture to contact the oral tissues or skin. In case of contact, remove the material with absorbent cotton soaked in alcohol and rinse with water.
- Avoid eye contact of the liquid or cement mixture. In case of contact, immediately flush with water and seek medical treatment.
- Do not mix the powder or liquid of Proclinc Expert Glass Ionomer Cement with any other glass-ionomer product.
- Shade: universal
- Keep away from children!

STORAGE

Store Proclinc Expert Glass Ionomer Cement in a cool and dark place at 4-25 °C (39-78 °F). Temperature should not exceed 25 °C (78 °F). Store protected from moisture. Do not use after expiry date.

COMPOSITION

Dental glass, polyalkenoate acids.

DISPOSAL

Disposal of the product according to local authority regulations.

REPORTING OBLIGATION

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

NOTE

The summary of safety and clinical performance of the medical device can be found in the European database on

medical devices (EUDAMED - <https://ec.europa.eu/tools/eudamed/>).

WARRANTY

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

LIMITATION OF LIABILITY

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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