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INSTRUCTIONS FOR USE

LIGHT CURE RESIN MODIFIED GLASSIONOMER LUTING CEMENT

Product description

Proclinic Expert Reinforced Glass Ionomer Cement is a light cured resin reinforced glassionomer luting cement. Beside its excellent fluoride release and the excellent biocompatibility as glassionomer cement Proclinic Expert Reinforced Glass Ionomer Cement has also good chemical bonding to dentine and enamel and to the restoration and a tight seal at the dentinal margins. Because of its radiopacity it ensures easy postoperative diagnosis. Proclinic Expert Reinforced Glass Ionomer Cement meets the requirements of DIN EN ISO 9917-2.

Indications/Intended use

- Cementing of crowns, bridges, inlays and onlays (all types: metal, resin, ceramic fused to metal and ceramic)
- Liner under composite fillings

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used further on.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Interaction with other materials

Avoid direct contact with products containing eugenol since eugenol impairs the setting of Proclinic Expert Reinforced Glass Ionomer Cement.

Application

1. Tooth Preparation

Prepare tooth in usual manner.

Apply Proclinic Expert Reinforced Glass Ionomer Cement Primer with a suitable brush onto the moist enamel and dentine surfaces for 30 seconds with agitation. The material should build a homogeneous layer. Air thin gently for 10 seconds to remove the volatile components and to disperse the adhesive. Then light cure with a suitable dental light unit for 10 seconds.

Pulp capping with Proclinic Expert Reinforced Glass Ionomer Cement is contraindicated. To deep areas of possible pulpal exposure apply a small amount of a calcium hydroxide liner.

2. Dispensing and Mixing

The powder/liquid ratio to achieve a suitable consistency is

2.0/1.0. This can be obtained by mixing one level (blue) scoop of powder and 3 drops of liquid.

Note:

The spoon dosage is only an approximate reproduction of the nominal mixing ratio.

For accurate dispensing of Proclinic Expert Reinforced Glass Ionomer Cement powder shake the bottle to loosen the powder. Overfill the spoon with the powder, level the powder for using the collar at the top of the bottle and carry it onto the mixing pad. Avoid compressing powder into the spoon with the inside wall of the bottle.

For dispensing of Proclinic Expert Reinforced Glass Ionomer Cement liquid turn the bottle vertically with the tip about 5 cm above the mixing pad. Steady your hand and squeeze the bottle gently to dispense one drop. If any bubbles are present, lightly tap the bottle with the fingers holding it. Discard drops that contain bubbles and are obviously not full-sized.

Use a small spatula to rapidly mix all the cement powder into the liquid. The mixed cement should be thixotropic and have a smooth consistency and glossy appearance. Total mixing time is 30 sec.

After use, lightly close both liquid and powder bottles to prevent exposure to moisture.

3. Cementing

Avoid water and saliva contamination during application and setting of the cement. Rubber dam is the best way of achieving tooth isolation.

Prepare the restoration according to manufacturer instructions.

Mix the required amount of Proclinic Expert Reinforced Glass Ionomer Cement and apply a thin layer (1 mm or less) to the prepared bonding surface of the restoration and seat immediately (working time 2:30 min from start of mixing at 23°C). Net setting time without any light is approx. 4:30 min.

Note: Higher temperatures will shorten the working time, lower temperatures will prolong the working time. An overextended working time will cause the loss of adhesion to enamel and dentine.

Remove excess cement at the first setting stage.

Maintain isolation until the set of the cement is verified (approx. 4:30 min). Setting can be optimized with a 20 sec light cure.

4. Additional Notes/Warnings

- The material can also be used as liner under composite restorations.
- Unpolymerized material may have an irritant effect and can lead to sensitization against methacrylates.
- Avoid contact with skin, mucous membrane and eyes
- If the material comes into contact with skin, in case of contact, remove the material with absorbent cotton soaked in alcohol and rinse with water. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

5. Storage

Store Proclinic Expert Reinforced Glass Ionomer Cement in a cool and dark place at 4-25 °C (39-77 °F). Temperature should not exceed 25 °C (77 °F). Do not use after expiry date.

Composition

Powder: Dental glass, photo initiators
Liquid: polyalkenoate acid, methacrylates

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices

Warranty

P.L. Superior Dental Materials GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Materials GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Materials GmbH's sole obligation shall be repair or replacement of the P.L. Superior product.

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Materials GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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PROCLINIC S.A.U.