- de Gebrauchsanweisung.
- en Instructions for use.
- es Instrucciones de uso.
- fr Mode d'emploi.

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INSTRUCTIONS FOR USE

Material description:
Chemical special bonding agent to improve the adhesion between denture base materials and highly cross-linked acrylic teeth.

Applications:
The product is suitable for all areas of dental prosthetics and ensures a reliable, durable bond between highly cross-linked, prefabricated acrylic teeth and PMMA denture base resins. It also provides significantly improved adhesion in denture repairs, extensions, additions, and the rebonding of detached teeth.

Contraindications:
If an allergy to one or more components of the product is proven, the material must not be used.

Properties:

- Easy and safe handling
- Creates a strong bond between teeth and denture base
- Also ideal for denture repairs
- Transparent, invisible color
- Durable bond

Dosage and application:
Roughen the areas of the teeth to be bonded with acrylic using a diamond bur or sandblast with aluminum oxide. A particle size between 50 µm and 120 µm is recommended. Afterwards, clean the teeth by steaming off wax residues, grease, insulation, or blasting media. Apply the product once or twice in a thin layer to the roughened, clean, and dry tooth surfaces. After a short reaction time of approx. 2-3 minutes (the surfaces must not dry completely), further processing may begin. If the tooth surfaces dry completely, the bonding agent must be reapplied to maintain surface activity.

Storage and safety instructions:
Do not use after the expiry date. Store below 25°C. Avoid direct sunlight. Keep the container tightly closed after use. Keep away from heat/sparks/open flames/hot surfaces. Do not smoke. Dispose of contents/container in accordance with local regulations.

Contains Methyl Methacrylate; Dichloromethane. Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May irritate the respiratory tract. Suspected of causing cancer. Avoid breathing dust / fumes / gas / mist / vapour / aerosol. Wear protective gloves / protective clothing / eye protection / face protection. IF ON SKIN: Wash with plenty of water and soap. Refer to the safety data sheet for further information.

Disposal:
Dispose of in accordance with official regulations. This product and its container must be disposed of as hazardous waste.

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PROCLINIC, S.A.U.
retect® Kunststofftechnik GmbH