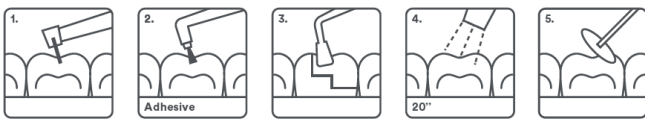
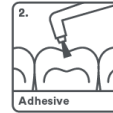
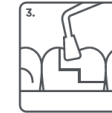
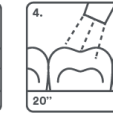
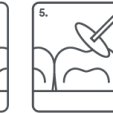
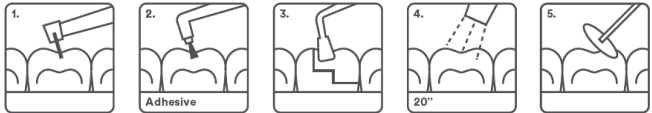
## FICHA TÉCNICA

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## T E C H N I C A L   D A T A   S H E E T

<b>PROCLINIC EXPERT NANO FLOW COMPOSITE</b>		
<b>REFERENCES</b>	According to Annex 1	
<b>MANUFACTURER</b>	<b>SDI LIMITED</b> 3-15 Brunsdon street Bayswater victoria 3153 Australia	
<b>CLASSIFICATION</b>	Medical Device, Class IIa, Directive 93/42/EEC.	
<b>DESCRIPTION</b>	Combining nano-hybrid and micro filler particle technology together with non-stick handling, the radiopaque light cured, condensable and low shrinkage composite offers polishability and strength on an unprecedented scale for anterior and posterior restorations alike.	
<b>COMPOSITION</b>	Nano Flow: 40% wt multifunctional methacrylic ester. 60% wt inorganic filler	
<b>PACKAGING</b>	2 x 1 g syringes + 10 applicator tips.	
<b>SHELF LIFE</b>	3 years.	
<b>INTENDED USE</b>	For use only by dentists. Direct anterior/posterior restorations for Class I, II, III, IV & V. Veneers. Core build ups. Inlays and onlays. Splinting. Composite and porcelain repairs.	
<b>INSTRUCTIONS FOR USE</b>	<div style="display: flex; justify-content: space-around; align-items: center;">      </div> <p>1. Isolate and prepare tooth. 2. Apply a bonding agent as per manufacturer's instructions. 3. Apply Proclinic Nano Hybrid Composite in increment of 2 mm. 4. Light cure each increment for 20 seconds minimum. 5. Finish and polish using standard techniques.</p>	
<b>PRECAUTIONS</b>	Avoid prolonged contact with skin/soft tissues as it may cause inflammation or sensitisation. Discontinue use if irritation occurs or if have known acrylic resin allergies. When not in use, replace cap.	
<b>CONTRAINDICATIONS</b>	Pulp capping. Product may cause skin irritations to some people. In such cases, discontinue use and seek medical attention. Do not use in conjunction with any eugenol-containing materials.	
<b>HANDLING AND STORAGE CONDITIONS</b>	Store in a cool dry place between 10 and 25°C, away from direct sunlight. Protect from moisture. Keep out of the reach of children	

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