## FICHA TÉCNICA

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<b>AGUJAS DE IRRIGACIÓN CON JERINGA BESTDENT</b>		
<b>REFERENCES</b>	<p><b>87832:</b> IRRIGATION SYRINGES WITH 27G NEEDLE (0.4X32MM).</p> <p><b>87833:</b> IRRIGATION SYRINGES WITH 30G NEEDLE (0.3X25MM).</p>	
<b>MANUFACTURER</b>	<p><b>Changzhou Jinlong Medical Pastic Appliance Co., Ltd.</b> Zhenglu Town, Changzhou Jiangsu 213111 China</p>	
<b>CLASSIFICATION</b>	<p>Medical device Class IIa under Directive 93/42/EEC and Regulation 607/2023 (EU).</p>	
<b>DESCRIPTION</b>	<p>Bestdent syringe irrigation needles are sterile, blunt-tipped and have a lateral outlet, designed for controlled irrigation of root canals in endodontic procedures.</p>	
<b>SPECIFICATION</b>	<ul style="list-style-type: none"> <li>• Syringe capacity: Generally, 3 ml, suitable for use in endodontic procedures.</li> <li>• Tip type: Roma (non-beveled), which prevents damage to the tooth apex.</li> <li>• Side outlet: The side opening allows for controlled, atraumatic flow, improving rinsing and cleaning of the canal.</li> <li>• Available sizes: 27G (0.4 mm) and 30G (0.3 mm), with lengths of 25 mm or 32 mm depending on the model</li> <li>• Connection: Luer-Lock, ensuring that the needle is securely attached to the syringe and preventing detachment during irrigation</li> <li>• Colour coding: 30G yellow, 27G grey, useful for quickly identifying the gauge.</li> </ul>	
<b>COMPOSITION</b>	<ul style="list-style-type: none"> <li>• Medical-grade stainless steel needle</li> <li>• Plastic coating</li> </ul>	
<b>PACKAGING</b>	<p>Box of 100 units, sterile, individually packaged</p>	
<b>INTENDED USE</b>	<p>Irrigation of root canals in endodontics; minimizes the risk of apical extrusion and tissue trauma during cleaning and disinfection.</p>	
<b>INSTRUCTIONS FOR USE</b>	<ol style="list-style-type: none"> <li>1. Select a sterile needle.</li> <li>2. Connect to a Luer-Lock syringe.</li> <li>3. Fill the syringe with the desired irrigation solution.</li> <li>4. Gently insert the needle into the root canal.</li> <li>5. Irrigate without applying excessive pressure.</li> </ol>	

	<p>6. Single use only. Discard after use.</p>
<b>HANDLING AND STORAGE CONDITIONS</b>	<ul style="list-style-type: none"><li>• Store in a cool, dry place, away from direct light and heat.</li><li>• Do not expose to moisture or aggressive chemicals.</li><li>• Do not use if the packaging is damaged.</li></ul>