FICHA TÉCNICA

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TECHNICAL DATA SHEET

ALCOHOL-FREE SURFACE DESINFECTANT DETERGENT		
REFERENCE	49895 (2654241HU): 750 ML 49889 (2654042HU): 5L	
MANUFACTURER	D.M.D. 1, rue de l'Espoir F-59260 Lezennes, France Tel : + 33 3 20 67 67 67 / Fax : + 33 3 20 67 67 68	
CLASSIFICATION	Class IIb Medical Device, Directive 93/42/EEC.	
DESCRIPTION	Alcohol-free surface disinfectant cleaner PROCLINIC 5L. <u>Main characteristics of the product:</u> - Ready-to-use product. - Formula that guarantees a wide compatibility with synthetic materials, plastics. - Formulated without alcohol, perfume or colorants. - Broad spectrum of antimicrobial action from 1 minute. - Product pH: ≈ 6. - Free of CMR substances*, aldehyde-free. - Not classified as environmentally hazardous.	
COMPOSITION	Didecyldimethylammonium chloride, excipients.	
PACKAGE	- 49895: 750 ML spray can. - 49889: 5L decanter.	
PURPOSE	Ready-to-use alcoholic rapid disinfectant for prophylactic surface disinfection of alcohol-resistant medical devices in the medical sector.	
INSTRUCTIONS FOR USE	See product instructions. Be sure to wear suitable gloves. 1. Spray the product on a non-woven cloth or directly on the area to be cleaned and disinfected. 2. Wipe the area thoroughly with a non-woven cloth, making sure to treat the area completely. 3. Leave to act according to the contact time indicated for the desired antimicrobial activity. Rinsing is not necessary unless there is a possibility of the medical device coming into contact with mucous membranes. Microbiological properties: • Bactericidal: EN 1040, EN 13727, EN 13697 - 2 minutes. • MRSA: EN 13727, EN 13697 - 2 minutes. • Levuricidal: EN 1275, EN 13624, EN 13697 - 5 minutes.	

TECHNICAL DATA SHEET

	Active according to EN 14476 on Herpesvirus, PRV (HBV model virus), BVDV (HCV model virus), Vaccinia virus, HIV-1 - 1 minute. Active according to EN 14476 on Polyomavirus SV40 - 5 minutes.
SAFETY INSTRUCTIONS	Dangerous - observe precautions for use.
STORAGE AND TRANSPORT	Store at a temperature between 5°C and +35°C.

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