FICHA TÉCNICA

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TECHNICAL DATA SHEET

BESTDENT IRRIGATION NEEDLES WITHOUT SYRINGE 27G		
REFERENCES	87831: Irrigation Needles without Syringe 27G (0,4x32mm)	
MANUFACTURER	Shandong Wuzhou Medical Equipment Co., Ltd. Dingtao (yantai) Industrial Area, 274100 Heze, Shandong Province, P.R. China	
CLASSIFICATION	Medical device Class IIa under Directive 93/42/EEC and Regulation 607/2023 (EU)	
DESCRIPTION	Sterile endodontic irrigation needle, 27G (0.4 mm outer diameter) and 32 mm length. Rounded, atraumatic tip and lateral exit for controlled irrigation, Luer-Lock connection. Grey color for quick identification. Used for canal irrigation during endodontic procedures.	
SPECIFICATION	• Gauge: 27G (0.406 mm) • Length: 32 mm • Tip: Rounded / non-beveled • Lateral exit for safe irrigation • Connection: Luer-Lock • Indicator color: Grey • Sterile unit, individually packaged • Single-unit blister	
COMPOSITION	• Medical-grade stainless steel needle • Plastic coating	
PACKAGING	Box of 100 units, sterile, individually packaged	
INTENDED USE	Irrigation of root canals in endodontics; minimizes the risk of apical extrusion and tissue trauma during cleaning and disinfection.	
SHELF LIFE	See packaging	
INSTRUCTIONS FOR USE	1. Select a sterile needle. 2. Connect to a Luer-Lock syringe. 3. Fill the syringe with the desired irrigation solution. 4. Gently insert the needle into the root canal. 5. Irrigate without applying excessive pressure. 6. Single use only. Discard after use.	
HANDLING AND STORAGE CONDITIONS	• Store in a cool, dry place, away from direct light and heat. • Do not expose to moisture or aggressive chemicals. • Do not use if the packaging is damaged.	