FICHA TÉCNICA

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TECHNICAL DATA SHEET

4DESIGN PREMIUM KOLINSKY OPAQUE BRUSH 1		
REFERENCES	H21116	
MANUFACTURER	East Rich Enterprise Co Ltd 3F, No. 576, Sec. 1 Minsheng N. Rd Guishan Township, Taoyuan County, Taiwan https://www.eastrich.tw/	
CLASSIFICATION	Non-medical product (prosthetic class).	
DESCRIPTION	Dental/prosthetic laboratory instrument for applying ceramic in dental prostheses. High-precision brush with natural Kolinsky sable hair, designed for detailed application of dental ceramics. It has a fine and flexible tip that allows excellent control during layering and makeup of dental porcelain.	
SPECIFICATIONS	• Size: 1 • Hair type: Natural Kolinsky sable • Recommended use: Dental ceramic makeup and glazing <u>Key features:</u> ◦ Fine and flexible tip ◦ Excellent liquid holding capacity ◦ High elasticity and precision	
COMPOSITION	• Hair: Natural Kolinsky sable • Handle: Lacquered metallic, violet color • Ferrule: Nickel-plated metal	
PACKAGING	Single unit in protective packaging to preserve the shape and quality of the brush.	
INTENDED USE	Specialized tool for precise ceramic application in the production of dental prostheses, enabling fine detail and high-quality finishes.	
INSTRUCTIONS FOR USE	1. Before first use, slightly moisten the brush in distilled water. 2. Load the ceramic according to the needs of the procedure. 3. Apply the ceramic to the dental structure with smooth and precise movements. 4. After each use, clean the brush with distilled water and dry it gently.	
HANDLING AND STORAGE CONDITIONS	Store in a dry place protected from dust. Avoid direct sunlight and sources of heat. Transport in its protective case to prevent deformation of the tip.	